






COWELL® Implant Solution

Help your daily practice superior



THE OLDEST IMPLANT CASE IN KOREA

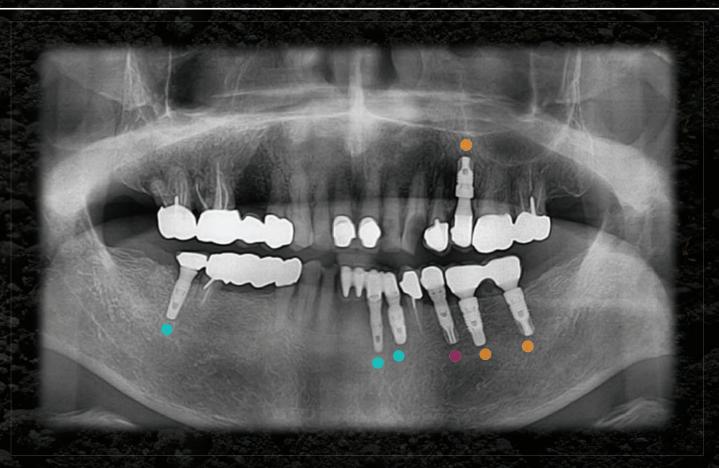







NNO Implant™

#35: BIOPLANT™, 1st generation of the COWELL® Implant, Korea's first dental implant developed in 1994.

#25, 36 & 37: ATLAS Implant System™, 2nd generation of the COWELL® Implant, Korea's first ASD treated Implant.


#32, 33 & 47: INNO Implant System™, 3rd generation of the COWELL® Implant, Nano/CaP soaking technology applied.

Help your daily practice superior

Contents

REID (Research & Education in Implant Dentistry)

COWELL® Implant System

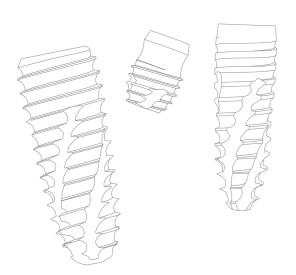

Introduction **Process Flow Chart** 800 COWELL® Warranty 010 Package System 011 SLA-SH® Surface Treatment 016 COWELL® CLASS 1000 021 • INNO SLA-SH® Implant System

004

006

202

INNO-Fixture Design 024 Abutment Prosthetic Protocol 026 INNO Submerged Implant 028 INNO Submerged Short Implant 033 INNO Submerged Narrow Implant 060 INNO Internal Implant 076 **INNO External Implant** 096 110 Surgical Kit

• Mini Plus® Implant System Mini Plus® Implant 127 132 Surgical Kit



• Digital Guided Surgery Kit InnoFit® Lodestar Plus Kit 136 InnoFit® Lodestar Kit 150 • Digital Prosthesis SFIT Ti-Base System · Sub. SFIT Ti-Base System 160 · Sub. SFIT Absolute Ti-Base System 166 · Sub. & Sub-N. Multi SFIT Ti-Base System 172 · Int. SFIT Ti-Base System 180 InnoFit® Hybrid Ti-Base System · Sub. Hybrid Ti-Base System 184 · Sub. & Sub-N. Multi Hybrid Ti-Base System 190 · Sub. Lock Hybrid Ti-Base System 194 · Sub-N. Hybrid Ti-Base System 198 · Int. Hybrid Ti-Base System















COWELL® Expert Instruments

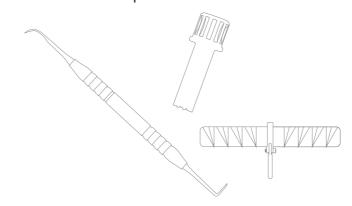

• MFS Kit	208
Sinus Lift Kit	216
• MFR Kit	222
• InnoGenic™ GBR Kit	226
• InnoGenic™ Autobone Harvester	236
• COWELL® BMP Trephine Kit	241
Atraumatic Extraction Kit	244
Direct Surgical Guide Kit	251
AO4 Surgical Stent	260
• Volume-up™ Guide System	262

COWELL® Regenerative Solution

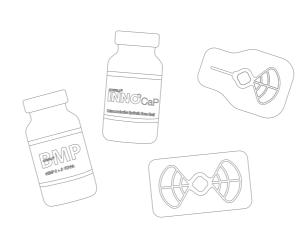

COWELL® BMP Series	268
INNO-CaP	276
INNO OSS & INNO OSS Allo	278
DiaBone	279
MEGA DERM™ Plus	280
Diaderm [®] M	282
InnoGenic™ Non-resorbable Membranes	283

used by a word-processing pr idard against which to check the spell COWELLMEDI [kavəl:medi] Noun, singular 1. Cowellmedi is a manufacturer of dental implants, regenerativ materials, instruments and The company is well known as the manufacturer that developed and launched Korea's first dental implant, called Bioplant™. As one of key players in dental healthcare industry, the company was founded in wing obtained a patent in Korea and USA for its rhB implant surface coating technology, Cowellm tic attention from the world dental com

COWELLMEDI HISTORY

For the first time in Korea,

Beginning with Korea's First Dental Implant, the COWELLMEDI has been leading the way to the future biom industry with the fusion technology to its E.rhBMP-2 developed for the first time in the world

1994 •	• Developed Korea's First Dental Implant, the BIOPLANT™ and localized it for the first time in Korea.
---------------	-------------------------------------------------------------------------------------------------------

• Founded Asrahi Medical.

• Established R&D corporation with PNU's Oral and Biotechnology Research Center.

Converted to COWELLMEDI corporation (Cowellmedi Co., Ltd.) and obtained ISO9001 certificate.

Developed ASD surface treatment technology for dental implant and applied for the first time in Korea.

• Obtained US-FDA approval for the BIOPLANT™ Implant System.

Medaled for contribution of developing Korea's first dental implant, the BIOPLANT™, from Korea Government.

Obtained GMP, ISO13485 certificate, US FDA approval for the ATLAS Implant System™ and CE (EC) certificate.

• Established subsidiaries in USA (COWELLMEDI USA) & Taiwan (COWELL Biotech) and COWELLMEDI Tissue Engineering Institute for Growth Factors.

· Obtained a patent for E.rhBMP-2 coated dental implant in Korea.

• Established COWELL Training Center in L.A, USA.

• Obtained MFDS approval for clinical test of E.rhBMP-2, the COWELL® BMP.

Obtained MFDS manufacturing & sales approval for the COWELL® BMP and held Korea's First BMP Symposium in Seoul, Korea.

Obtained US patent for E.rhBMP-2 Coated Implant.

Obtained MFDS Approval for E.rhBMP-2 Spinal Fusion Clinical Test Plan and launched the INNO SLA Implant System™.

Obtained US FDA approval for the INNO SLA Implant System™.

Established a R&D and Education Organization, REID (Research & Education in Implant Dentistry).

dental implant manufacture

 Applied SLA-SH® surface treatment on the INNO SLA Implant System™ → INNO SLA-SH® Implant System. (*SLA-SH® Surface Treatment: the world's first superhydrophilic surface made by dry process using Nano/CaP soaking technology. For the details, refer to the pages 16 to 21).

Established a subsidiary in China (COWELLMEDI China) and a training course, ACID(Advance Course in Implant Dentistry)

 Developed Collagen based bone graft materials and launched Sonator™ 80's System. 2017

Launched the InnoGenic™ Wifi-Mesh and InnoGenic™ PTFE-Mesh.

Launched the InnoFit Lodestar Plus Kit, InnoGenic Autobone Harvester Kit, InnoGenic Bone Carrier

Launched the InnoGenic™ GBR Kit and Spread Fit™ SFIT Ti-Base System


004 COWELLMEDI History



The REID is a global institute, standing for Research and Education in Implant Dentistry.

The REID has been dedicated to researching technology and knowledge from the past to the future, to creating more predictable concepts of treatment, and to developing cutting edge products in implant dentistry and related fields as its first objective of the establishment.

As its second objective of the establishment, the REID also has been committed to training dental professionals with world-class clinicians, lecturers and education curricula.

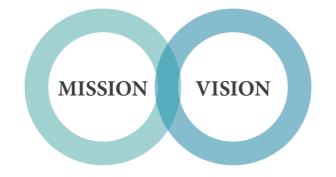

To improve how the world dental community treats implant dentistry by providing dental professionals with internationally multidisciplinary education service and state of the art treatment concepts as well as comprehensive quality research for the benefits of patients.

- Constructing the future of implant dentistry and related fields.
- The world-class education provider and research
- Sharing more know-hows to have better ideas by expending worldwide network of members.
- Providing training system accessible to any dental professional across the globe.

006 REID

Process Flow Chart

CNC Machining



Precise machining process using state of the art computer numerical control system fused to the COWELL® Class 1000, operated by world-class technical unit.



Surface Treatment



The SLA-SH® Surface treatment with biologically active materials to achieve the ideal Osseointegration.



Inspection



Absolutely accurate test and quality control system with cutting edge equipment such as optical profiling measurer, stereoscopic microscope, micrometer scope and other specialized devices for dental implant manufacturing.

Cleansing



Cleansing process by ultrasonic wave using the 3rd stilled distilled, vacuum dry and heating dry sterilization enable ultimately sterilized products and no residues are left on products.



Packing / Sterilization



Sanitarily packed products at Class 10,000 clean room are sterilized by gamma-ray using radiation isotope.



Shipping Warehouse



Finished products are sorted and stored in warehouse for immediate delivery.

008 Process Flow Chart

COWELL® Warranty

* For more details, visit our website at www.cowellmedi.com

1. Guarantee beneficiary and scope

Implant	Lifetime	Replacement with equivalent Implant	The period shall begin from the sale date

- 2. Scope of Warranty
- 1) Quality benefits
- > In case the product material or the manufacture process is flawed.
- 2) Surgical benefits
- > In case implants fail to be grafted to the bone.
- 3. Claim Procedure
- 1) In case certain faults occur after transplanting implants (procedure), the staff in charge shall be contacted within 30 days thereafter.
- 2) When such contact is made, the Customer Complaint Report shall be written out, and shall be submitted together with the concerned product.
- 4. Exclusions from Warranty Service
- 1) In case implants are transplanted onto patients with diabetics and alcohol addiction.
- 2) In case implants are transplanted onto patients for whom surgical procedures are difficult to perform due to the history of systematic disease.
- 3) In case implants are transplanted onto patients who depend on habitual medications.
- 4) In case the procedure is not conducted according to the protocol of the COWELLMEDI.
- 5) In case the procedure is not performed in compliance with biological indication: (E.g. distance between buccal wall and implant should be at least 2mm).
- 6) In case the procedure is conducted using contaminated surgical devices.
- 7) In case implants are transplanted onto patients who sustain or are infected with cell issue contamination.
- 8) In case other materials from other companies are mix used with Implants, prosthetic parts and instruments of the COWELLMEDI.
- 9) In case the result of investigation by COWELL R&D Institute, Div. of QA and QC shows the issue is not related to the products manufactured and provided by the COWELLMEDI.
- 10) In case the information hereby requested, especially, product Lot no., Serial no. or X-ray photos, is missing.
- 11) In case that the concerned products are not returned.
- 12) In case the product is damaged due to negligence of handling.
- 13) In case the product is opened and fails to remain sterilized.
- 14) In case that the expiry date of the concerned product (not opened products only) is not longer than 1/4.

Package System

1. Color classification (Coding) by fixture type and external label marking

A. Color classification by fixture type

Fixture type	Submerged (Sub.)	Submerged Short (Sub.)	Internal (Int.)	External (Ext.)	Submerged Narrow (Sub-N.)	Mini Cement (1P-C.)	Mini Ball (1P-B.)
Package		INN Q	INT. OCTAGES SISTEM	INNO	SUB. BARROW HERAGON STSTEM	10 110 110 110 110 110 110	MINIPAL
Connection	HEX	UB. AGON STEM	INT. OCTAGON SYSTEM Orange	EXT. HEXAGON SYSTEM Green	SUB-N. HEXAGON SYSTEM Emerald	MINI PINE SYSTEM PINE	

B. External label marking and color coding by fixture diameter & fixture type

- > Color coding by diameter on the external label.
- > Reuse is prohibited after opening as the product is sterilized.
- > After the ampule is opened, care should be taken from dropping which may cause by incomplete fastening.

Ø2.5

- > Care should be taken from infection after the product is opened and store at room temperature and in a dry place.
- > Discard expired products.



INNO Fixture (No-Mount)
PRODUCT NAME: COWELL® INNO Implant System
CATALOG No. [REF]: ST4010SM
SIZE : Ø4.0X10mm(Sub.Hex.Taper)
LOT NO. LOT: 20A060040A
DATE OF MANUFACTURE [M]: 2020-01-06
USE BY ☐: 2025-01-05
PACKING UNIT : 1EA
STORAGE CONDITION: Store at room temperature and in a dry place.
Do Not Caution 1°C 30°C
STERILE R Sterilized Using Irradiation Consult Instructions for Use

Manufacturer Countries Co., Ltd. Cowellmedi Co., Ltd. 48, Hakgam-daero 221beon-gil, Sasang-gu, Busan, 46986, Republic of Korea TEL.: +82-51-312-2027~8 D/T: 218 Trianon LN Villanova PA 19085-1442 USA EC-Representative CREP: Certification Experts B.V. Amerlandseweg 7, 3621 ZC Breukelen, The Netherlands



(21) 0004	
Only	CE (
EVICE	CWM-L-004 (

Ex.) INNO SLA-SH® Sub. Fixture (No-Mount) Dimension: Ø4.0X10mm

Fixture										
Type(abbr.)	Bisque	Green	Burgundy	Orange	Yellow	Emerald	Red	Pink	Violet	Gray
Submerged (Sub.)	-	_	_	_	*	*	~	*	-	*
Submerged Short (Sub.)	-	-	-	_	-	~	4	*	~	4
Internal (Int.)	-	-	-	_	4	*	~	*	-	4
External (Ext.)	-	_	-	-	*	~	4	~	-	*
Submerged Narrow (Sub-N.)	_	_	*	~	-	_	_	_	-	-
Mini Cement (1P-C.)	*	4	-	-	-	-	-	_	_	_
Mini Ball (1P-B.)	*	4	_	_	_	_	_	_	_	_

Ø3.3

2. Fixture user guide (Embedded in the packaging)

COWELL® IMPLANT SYSTEM INSTRUCTIONS FOR USE

1. Device Description

The COWELLMEDI implant system includes a variety of precision-machined fixtures manufactured from titanium. These implants are surgically inserted into a mandible (the lower jawbone) or a maxillary bone (the upper jawbone) and serve as a replacement for patient's tooth root providing a stable foundation for restoration.

2. Intended for use

To support dental prosthesis as a dental device, which is implanted into alveolar bone to recover masticatory function and give better esthetics in patients with partially or full edentulous jaws.

3. Directions for use

1) Surgery - The first stage

- a. According to the patient's condition, appropriate dental cleaning operations may be performed and preventive antibiotics may be administered prior to implant operation.
- b. Clean and disinfect the operative site, administer local anesthesia in the area and expose the alveolar bone by making appropriate incisions and reflecting the gingival tissues along the alveolar crest in the area from where teeth were extracted.
- c. Drill into the gum in order to implant a fixture into the planned place with various dental operation tools. The speed of revolution of the drill should be adjusted by the condition of the bone and the kinds of operation tools. Saline solution should be poured onto the area so that necrosis doesn't occur by heating of the bone (The speed for all drilling should be less than 1,200 rpm).
- d. Remove the external sterile package cover sheet: open the cap of the ampule: affix the Fixture Driver (in case of No-mount Fixture) or the Mount Driver (in case of Pre-mount Fixture) to the Hand-piece and connect it to the fixture: move the assembled piece to the osteotomy site for the implant using care to prevent the assembled piece from being separated or contaminated with foreign materials.
- e. A fixture is implanted into the bone as planned depth by turning (25~30 rpm) a
 hand-piece clockwise with 15~50 N.cm torque. In event that it is hard to insert,
 extend width of bone by Tap Drill or Countersink (less than 1,200 rpm) in order to
 facilitate better implantation.
- f. After finishing implantation, the treated part should be sutured by using a hex driver to connect to the Cover Screw with torque 5 N.cm to prevent the intrusion of a foreign substance in the fixture.

2) Surgery - The second stage

 a. Incise gingival of upper part of fixture subsequent to bone fusion and remove Cover Screw, tighten up Healing Abutment and start gingival curing for prosthesis.
 b. In general, surgery is done by a method that makes prosthesis.

4. Contraindication

The operation should be reconsidered when the patient has any of the following conditions.

- a. Patient with oral infection or inflammation.
- b. In the case of low quality bone which will result in an unstable implant.
- c. Patients who have a drinking problem or mental disease or substance or medicine abuse.
- d. Internal disease such as hematodyscrasia or diabetes and undernourishment. e. Any patient who is not suitable for operation.

o. Any patient with

Implant surgery and restoration involves complex dental procedures. For safe and effective use of the COWELLMEDI fixtures, it is strongly suggested that specialized training be undertaken since the surgical techniques required to place dental implants are highly specialized and complex procedures. Improper patient selection and technique can contribute to fixture failure and/or loss of supporting bone. the COWELLMEDI fixtures are intended for use only in the indicated applications. Dental fixtures must not be altered in any way. The use of electro-surgical instruments or lasers around metallic fixtures and their abutments is not recommended due to the risk of electric shock and/or burns. Fixture mobility, bone loss, or chronic infection may indicate fixture failure. The treatment should be done in an aseptic condition by an operator who wears an aseptic costume. If the fixture becomes contaminated by the patient's body fluids in any way, the fixture cannot be used in any other patient.

6. Precautions

The surgical techniques required to place endosseous dental fixtures require specialized and complex procedures. Formal training for placement of fixtures is recommended.

Important: Determine local anatomy and suitability of the available bone for fixture placement. Thorough screening of prospective fixture candidates must be performed. Visual inspection as well as panoramic and periapical radiographs are essential to determine anatomical landmarks, occlusal conditions, periodontal

status, and adequacy of bone. Lateral cephalometric radiographs, CT scans and tomograms may also be beneficial. Adequate radiographs, direct palpation and visual inspection of the fixture site are necessary prior to treatment, planning and use of the COWELLMEDI fixtures.

7. Adverse Effects

Some of the complications (loss of fixture anchorage, prosthesis etc.) are possible occurrences after surgery. Lack of quantity or poor quality of remaining bone, infections, poor patient oral hygiene or cooperation, patient discomfort, fixture mobility, local soft tissue degeneration, and unfavorable fixture placement or alignment are some potential causes for loss of anchorage.

8. Surgical complications

The implant procedure has risk, including localized swelling, dehiscence, tenderness of short duration, edema, hematoma or bleeding. Numbness of the lower lip and chin region following lower jaw surgery, and of the tissue beside the nose following upper jaw surgery, is a possible side-effect of the surgery. Though it would most probably be of a temporary nature, in very rare cases, the numbness has been permanent. Gingival-mucosal (gum tissue) ulceration, tissue reaction or infection may occur, but generally responds to local care.

9. How to manage after implanting the Product

- a. The upper jaw requires a healing period of 6~8 months depending on the bone quality and the lower jaw requires a healing period of 3–5 months depending on the bone quality. If pressures such as mastication would be applied to the fixture during the healing period, early fixation may not be obtained or osseointegration may not occur for the fixture within the healing period.
- D. Once the operator clinically determines that sufficient osseointegration has been achieved, he/she should begin the stages to produce the dental prosthesis.
- c. The Lot number identification tag should be attached to a patient's chart and X-ray film in order to pursue the product when needs be.
- d. The operator should determine the osseointegration status of the implant through X-ray and clinical methods such as percussion and/or reverse torquing.

10. Storage / Sterilization and Handling

- a. Store at room temperature and in a dry place.
- b. The fixture, fixture mount and cover screw have been cleaned and sterilized by radiation (gamma irradiation) and are ready to use.
- c. Product packages should be opened just before their use during operation and any product beyond the expiration date should not be used.
- d. Only appropriate sterilized surgical tools made specifically for dental implants should be used during the operation.

11. Expiration date

Expiration date of the product is 5 years from manufacturing.

12. Cleaning & Sterilization

Cleaning of surgical instruments supplied non-sterile should be performed according to current dental standard practices. Select a suitable method of cleaning that removes all visible contamination from the product in sterilized and distilled water. After cleaning, package the product appropriately and then sterilized by autoclave at the minimum condition of 250°F (121°C/15 mins).

3. Caution

- a. As this product is sterilized by Gamma radiation, should not be used under any circumstances if is open.
- b. Every product is disposable. It should not be reused.

COWELLMEDI Co., Ltd.

48, Hakgam-daero 221beon-gil, Sasang-gu, Busan, 617-801, Republic of Korea Tel. +82-51-312-2027~8 Fax. +82-51-316-2628

https://www.cowellmedi.com

D/T

COWELLMEDI USA INC. 218 Trianon LN Villanova PA 19085-1442 USA

Tel. +1-623-939-1344 Fax. +1-623-939-1472

EC-REPRESENTATIV

Certification Experts B.V. Amerlandseweg 7, 3621 ZC Breukelen, The Netherlands











2020. 01. 30 / CWM-I-007 (Ver.3)

3. Fixture packaging opening and the sequence of the product extraction



Taking out the ampule



To open, press the upper dotted area and take out the sterilized blister.



Remove the moisture-resistant paper on the back of the blister, and drop the ampule lightly on the palm of a practitioner or surgical clothes.

Fixture separation



Hold the ampule with both hands, twist it 45°, and separate the middle part.
Care should be taken so that the fixture will not fall.



Fasten in two ways.
1) No Mount -> Fasten the Fixture Driver.
2) Pre Mount -> Fasten the Mount Driver.

Cover Screw separation



Separate the upper part of the ampule.



Fasten the Hex Driver to the Cover Screw completely. Care must be taken to prevent patient from swallowing the Cover Screw at the time of placing.

012 Package System

Package System 013

4. Abutment packaging and external label marking



5. Surgical Kit packaging and external label marking





Implant Innoyation
When INNOVATION meets Dental Implant.

Experience the difference of SLA-SH®

made by "Nano/CaP soaking technology"



Superhydrophilicity, Uniform micro-surface geometry, Maximized BIC and Acceleration of osseointegration

Aspiring for 100% perfection with SLA-SH®

by Nano/CaP soaking technology

SLA-SH® Surface Treatment

by Nano/CaP soaking technology

SLA-SH®:


Sandblasted, Large-grit, Acid-etched and Super-Hydrophilised


- > Hydrophilicity by activation with neutralization solution & Nano/CaP soaking technology.
- > Sandblasted with biocompatible grits unlike majority of other implants in market are done with Alumina.
- > Macro-pore & micro-pore of Ti-oxide layer mimicking the etched enamel rod of tooth.
- > Even distribution of roughness through the whole portion of implant surface.
- > No destruction or alternation of the surface are caused even with torque force of 120 N.cm.
- > Acceleration of osseointegration and maximization of BIC.
- > The SLA-SH® is applied for all types of the COWELL® Implant Systems.

1. Evaluation using SEM (Scanning Electron Microscope) Images


A. SLA-SH® Surface magnified X300, 1,000 and 3,000



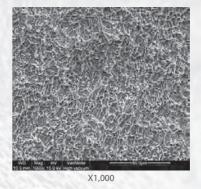

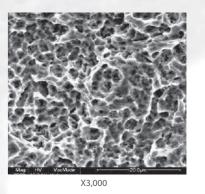


B. Comparison to other SLA treated implants currently sold in the market

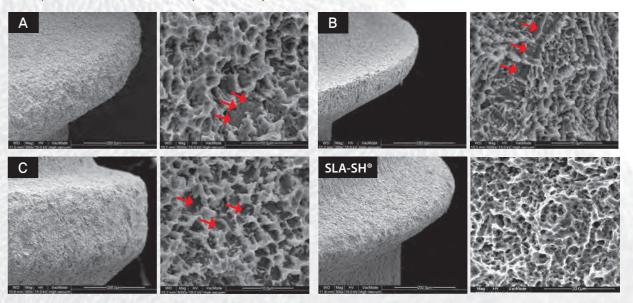

- > Surface treatment patterns were observed on electron microscope photographs of 5,000 magnifications for top parts of the implants.
- > Sand-blasted surface conditions were observed in the product A, B and C due to insufficient acid etching patterns in deep parts as the SLA-SH® is sandblasted with biocompatible grits with even particle size unlike others are done with alumina.
- > The entire surface of the SLA-SH® treated implant showed uniform acid etching patterns. This implies that the acid etching of the SLA-SH® surface is perfect.

2. Evaluation using SSEM (Stereo Scanning Electron Microscope) 3D images

A. SLA-SH® Surface

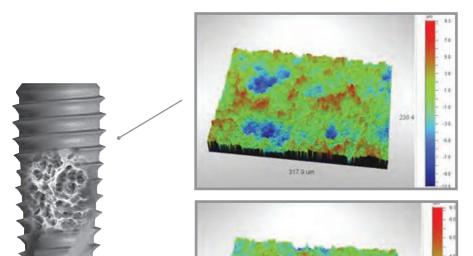

Surface Stats

Ra: 1.80 μm Rq: 2.27 μm Rt: 1849 μm

Measurement Info

Magnification : 19.93 Measurement Mode : VSI Sampling : 496.74 nm Array Size : 640 X 480


Surface Stats

Ra: 1.95 μm Rq: 2.52 μm Rt: 19.83 μm

Measurement Info

Magnification: 19.93 Measurement Mode: VSI Sampling: 496.74 nm Array Size: 640 X 480

B. Comparison to other SLA treated implants currently sold in the market

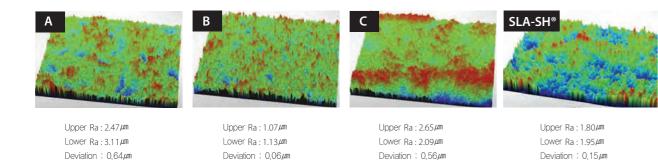


317.9 um

- > Uniform distribution of Macro-pore and micro-pore.
- > Roughness of the SLA-SH® showed 1.90um while the others were 1.08~3.11um.

3. The surface activity increased due to the great surface wetness

A. Contact angle measurement evaluation result for the saline solution

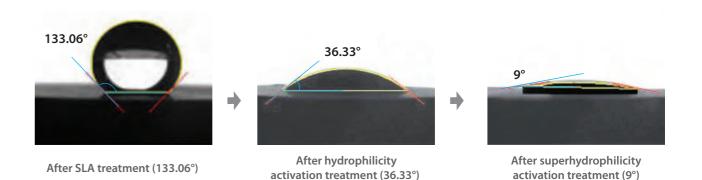


After neutralization process and Nano/CaP soaking treatment, the sample became extremely hydrophilic and the surface energy increased, which facilitated expedition of osteoblast activation to be fused to the bone faster.




Capillarity in the actual clinical setting, which accelerated the penetration of blood.

** Quoted from the website of Cowellmedi Clinical Research Group (www.e-cowellmedi.com)

B. Relation between surface wetness and roughness

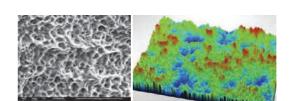

After SLA treatment (Ra: 1.78 / m)

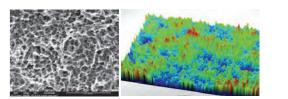

After super-hydrophilicity activation treatment (Ra: 1.90 /411)

Peak BE

End BE

- > There was almost no difference of surface roughness and micro-geometry, and the difference of surface wetness took place in the same physicochemical properties as surface energy increased by hydrophilicity activation treatment.
- C. Physicochemical alternation of surface by hydrophilicity activation treatment

Name	Start BE	PeakBE	End BE
C1s	290	284.6	280.5
O1s	535.3	530.42	525.6
Ti2p	468.1	458.78	450.4

458.78	450.4	Ti2p

 C1s
 290.46
 284.6
 284.6

 O1s
 538.8
 533.73
 529.3

 Ti2p
 468.2
 456.76
 453.4

Start RF

After SLA treatment


After hydrophilicity activation treatment

- > Surface wetness was improved by increased surface energy of C1s, O1s and Ti2p after hydrophilicity activation treatment.
- > To maintain and even to enhance surface wetness, superhydrophilicy activation treatment was carried out and contamination by carbon in the atmosphere is prevented during packing and sterilization.

Name

4. Its safety has been proven through perfect cleaning with an automated system

A. Comparison of surface element tests through X-ray diffraction

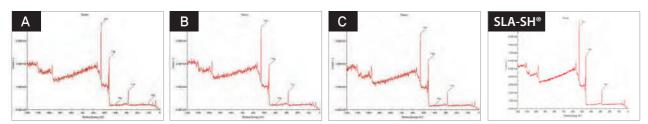

> Cutting-edge automated system that produces the 3rd distilled water.

B. Comparison of surface element tests (X-ray Photo-electron Spectroscopy, XPS)

					Unit:%
Sample	C1s	O1s	Ti2p	Si2p	N1s
Α	34.12	45.05	15.11	5.24	0.47
В	31.84	46.49	15.22	4.87	1.57
C	32.19	47.58	17.58	2.65	N.D
SLA-SH®	27.19	50.81	17.61	N.D	N.D

- > Quantitative analysis of each surface element found 30% carbon, 47% oxygen, 16% titanium and 4% silicon in all products.
- > For the SLA-SH®, they only consisted of carbons(C1s), oxygen(O1s) and titanium(Ti2p).
- > Sodium hydroxide, the main element of the alkali washing solution, combined with silicon(Si) to form water-soluble Na₂SiO₂(OH)₂·4H₂O(water glass), which removed the other elements.

C. Comparison of elution tests using combustion ion chromatography


							Unit:ppm
Sample	F-	Cl-	NO ₂ -	SO ₄ ²⁻	Br-	NO ₃ -	PO ₄ ³⁻
Α	N.D	0.024	0.027	0.002	N.D	0.031	N.D
В	N.D	0.027	0.019	0.002	N.D	0.030	N.D
С	N.D	0.071	0.020	N.D	N.D	0.023	N.D
SLA-SH®	N.D	N.D	N.D	N.D	N.D	0.032	N.D

- > Similar ions were detected in all the products, but they are not harmful to human because their elements and quantities do not affect the human body and those have been proven in many studies.
- > For the SLA-SH®, no other elements except for NO₃ were detected. Alkali washing completely removed the SO₄²⁻ and Cl⁻ ions of sulfuric acid and hydrochloric acid, which are used for heated acid etching because they form water-soluble salts of Na₂SO₄ and NaCl.
- > No elements that interfere with osteo anagenesis were found from both the surface and elution elements, which showing that the cleansing process was perfectly carried out.

COWELL® CLASS 1000


A SUBTLE DIFFERENCE MAKES THE DENTAL IMPLANT OR NOT


1. Fixture manufacturing tolerance evaluation

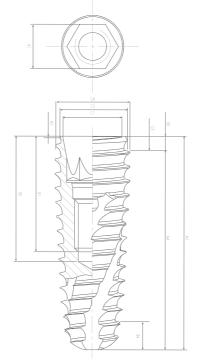

Evaluation Item		Mai	nufacturing Toleran	ce				
Method	a. The specimen was fixed in Jig.b. Each dimensional difference of 3 inner hexagonal connection sides (Hex-1, Hex-2, Hex-3) of 5 specimens was measured.							
Used Equipment	Measuring Micr	oscope and Jig						
Criteria	Each dimensional difference of 3 inner hexagonal sides should be no more than ±0.001mm (1.000/411) from 2.500mm.							
Specimen	INNO SLA-SH® Submerged Fixture (5 Pieces of ST4510S)							
эресппеп	#1	#2	#3	#4	#5			
Hex-1	2.499	2.500	2.500	2.500	2.500			
Hex-2	2.500	2.500	2.501	2.500	2.500			
Hex-3	2.500	2.500	2.500	2.501	2.499			
Average	2.500	2.500	2.500	2.500	2.500			
Total Average			2.500					
Result (Pass/Fail)			Pass					
Manufacturing Tolerance	No more than ±0.001mm (1.000 /≠□)							

2. Prosthetic component manufacturing tolerance evaluation

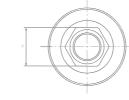

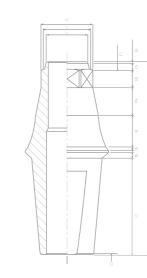

Evaluation Item	Manufacturing Tolerance						
Method	 a. The specimen was fixed in Micro-Measuring Instrument. b. Each dimensional difference of 3 outer hexagonal connection sides (Hex-1, Hex-2, Hex-3) of 5 specimens was measured. 						
Used Equipment	Micro-Measurir	ng Instrument					
Criteria		Each dimensional difference of 3 outer hexagonal connection sides should be no more than ± 0.001 mm (1.000μ m) from 2.490mm.					
Specimen	INNO SLA-SH® Sub. Cemented Abutment (5 Pieces of 2SCH4515)						
Specimen	#1	#2	#3	#4	#5		
Hex-1	2.489	2.490	2.490	2.490	2.490		
Hex-2	2.490	2.490	2.490	2.490	2.490		
Hex-3	2.490	2.490	2.490	2.490	2.491		
Average	2.490	2.490	2.490	2.490	2.490		
Total Average			2.490				
Result (Pass/Fail)			Pass				
Manufacturing Tolerance	No more than ±0.001mm (1.000 /≠□)						

020 SLA-SH®

COWELL® CLASS 1000 021

COWELL® IMPLANT SYSTEM

Help your daily practice superior

Designed for anterior esthetic zone with narrow alveolar ridge. Double tapered thread acquires higher primary stability

Volume-up™ Healing Abutment

Devised for preventing food penetration and forming aesthetical cervical area by restoring contracted buccal alveolar bone & gingiva to the original shape and width.

COWELL® IMPLANT SYSTEM

Meta G UCLA Abutment

Modification to angulated abutment, customized abutment and telescopic abutment.

Easy Temporary Abutment

Temporary restoration for anterior esthetic zone. For simpler and speedier chair-side process.

Angulated Abutment

Simple solution for anterior esthetic zone.

Milling Abutment

Block abutment to customize contouring.

INNO Submerged Short Fixture

Designed for severe bone resorption. Wide and deep upper thread prevents the compressive necrosis of the cortical bone.

INNO Submerged Narrow Fixture

through a wedge action.

| Miniplus®

For mandible anterior spaces and edentulous arch.
Semi-permanent or temporary solution for anterior spaces with extremely narrow ridge.

INNO External Fixture

Stable engraftment of periosteum in the boundary surface of bone and implant.

INNO Submerged Fixture

Designed for all clinical cases including immediate implant placement, immediate loading, implant depth adjustment, maxillary sinus and etc. Simply doing more for your implant treatment.

INNO Internal Fixture

FULL SURGICAL KIT

One stage restoration with state of the art design. 4 spiral round cutting edges maximize the efficiency of self tapping with a sharp edge and accommodates bone chips as ideal cutting edge pocket space.

Cemented Abutment

CWM

Cutting surface is designed for anti-rotation of prosthesis.

Sonator 80's S&A Abutment

Allows you to treat your patients with minimum standard of care of an implant supported overdenture at affordable cost with great satisfaction.

Beauty-up[™] Abutment

Specially designed to solve esthetical and functional challenges when SCRP with angulated screw channel is required in the anterior portion.


Ball Abutment

Designed for use with removable overdentures or partial dentures retained in whole or in part by endosseous implants in the maxilla and mandible.

Multi S&A and Lock Abutment
Designed for both edentulous and
partially edentulous arches.
A broad range of prosthetic

options allows to meet individual requirements of your patients.

INNO-Fixture Design

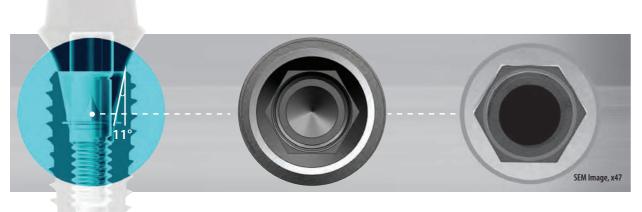

11° Tapered and 2.5 Hex Connection


- > Allows the ideal cold welding between the fixture and the abutment.
- > Prevents micro-sinking of the abutment.
- > Excellent compatibility.

Wide and deep upper thread

- > Prevents the compressive necrosis of the cortical bone.
- > Minimizes the need for Countersink Drills.
- > Reinforces mechanical strength by reinforcing

Double Tapered Thread

- > Secures early fixation even for an alveolar socket or parts with weak bone quality.
- > Allows placement to be completed with only 2-4 rotation with half the length of the fixture inserted in the drill hole.
- > Acquires higher primary stability through a wedge action even with an additional half turn.

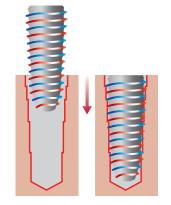

Shortens the placement time with 5mm or more of already entered depth as well as double thread.

Platform Neck

- > Stable engraftment of periosteum in the boundary surface of bone and implant.
- > Prevents possible infections around the implant.

Open Thread

> Possible to place deeper even without additional

4 spiral round cutting edges

- > Maximize the efficiency of self tapping with a sharp edge.
- > Accommodate bone chips as ideal cutting edge
- > Allow soft placement but higher initial stability (refer to the test table below).

Apex Thread with a sharp cutting edge

- > Prevents schneiderian membrane from being ripped.
- > Allows soft placement but higher initial stability (refer to the test table below).

₩ Placement torque force (Unit: for 4pcs of Ø4.5X10mm implants from 4 different manufacturers in 5.0 & 5.5mm depth hole of type 2 bone quality test block.

Classification	INNO	Α	В	C
Depth 5.0mm	26.2 N.cm	29.2 N.cm	26.8 N.cm	28.4 N.cm
Depth 5.5mm	44.0 N.cm	38.0 N.cm	34.4 N.cm	38.5 N.cm

An advantageous design for all clinical cases including immediate implant placement & immediate loading, implant depth adjustment, maxillary sinus and etc.

Fixture type	Submerged (Sub.)	Submerged Short (Sub.)	Internal (Int.)	External (Ext.)	Submerged Narrow (Sub-N.)
Fixture Design					
Connection	H	SUB. XXAGON YSTEM	INT. OCTAGON SYSTEM	EXT. HEXAGON SYSTEM	SUB-N. HEXAGON SYSTEM

Simpler, Speedier and Safer Surgical Kit

Sub. / Sub-N. / Int. / Ext. / provide different types of exclusive kits, respectively.











All in One Drill: Minimal drilling frequency with Initial and Final Drill

Chair time for implantation is shortened because the fixture can be implanted with just three drillings for general bone quality (when implanting Ø3.5-Ø4.5 fixtures).















COWELL® DIGITAL PRODUCTS

Abutment Prosthetic Protocol

> For digital procedure, refer to the COWELL® Digital Products.

1. Fixture Level Impression - Prosthesis Fabrication

* Two Piece Screw Retained Abutment

Submerged & Submerged Short: Temporary | Easy Temporary | SFIT S | SFIT A


Internal: SFIT S | SFIT A **External:** Temporary

* Two Piece Screw-Cement Retained / Cement Retained Abutment


Submerged & Submerged Short: Cemented | Angulated | Beauty-up™ | Milling | Meta G UCLA | Plastic UCLA Hybrid S | Hybrid L | Hybrid A | Ti-Block


Submerged Narrow: Cemented | Angulated | Temporary | Meta G UCLA | Hybrid S | Hybrid L | Hybrid A

Internal: Cemented | Angulated | Meta G UCLA | Hybrid S | Hybrid L External: Cemented | Angulated | Temporary | Meta G UCLA | Plastic Sleeve

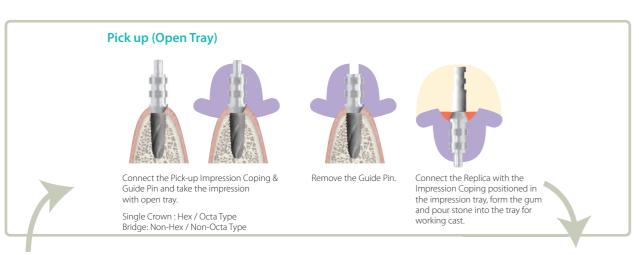



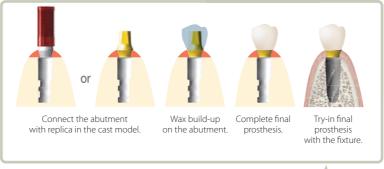

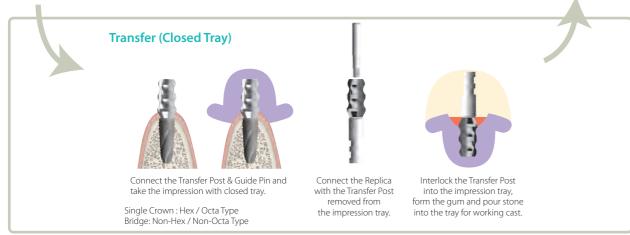

2. Abutment Level Impression - Prosthesis Fabrication

*Two / One Piece Screw Retained Abutment

Submerged & Submerged Short: Multi S | Multi A | Lock | SFIT Absolute Submerged Narrow: Multi S | Multi A

* One Piece Cemented Retained Abutment

Submerged & Submerged Short : Absolute | Straight (Direct)

Submerged Narrow: Straight Internal: Solid | Shoulder External: Shoulder

*Two / One Piece Attachment Retained Abutment

Remove the

The working cast

with the gum.

Submerged & Submerged Short: Sonator S | Sonator A | Ball

Internal: Sonator S | Ball


Form the gum and pour

stone into the tray for

working cast.

External: Ball

Indirect Impression Technique (No Abutment Modification Applied)

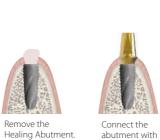

abutment with the fixture.



Fasten the Plastic Coping

(Absolute).



Ø4.5 Abutment Ø5.5 Abutment Ø6.5 Abutment Multi Pick-up Impression Coping

Fasten the Impression Cap

on the Abutment.

Wax build-up on the Lab Analog on the abutment



Take impression

with closed tray.

Complete final prosthesis.



Sonator Impression Coping


Connect the Lab Analog

with the Impression Cap

positioned in the impression

with the fixture.

Direct Impression Technique (Abutment Modification Applied)

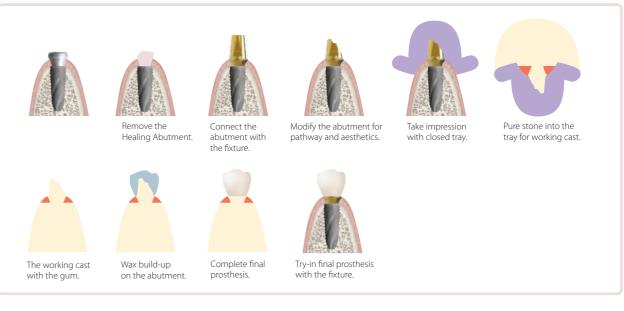

026 Abutment Prosthetic Protocol

Abutment Prosthetic Protocol 027

INNO SUBMERGED IMPLANT (Sub.)

System Flow




INNO Submerged Implant



Submerged Fixture Surface Treatment: SLA-SH®

> Interchangeable with hexagonal morse tapered fixture

> Internal hex connection (Taper 11°/ Hex 2.5)

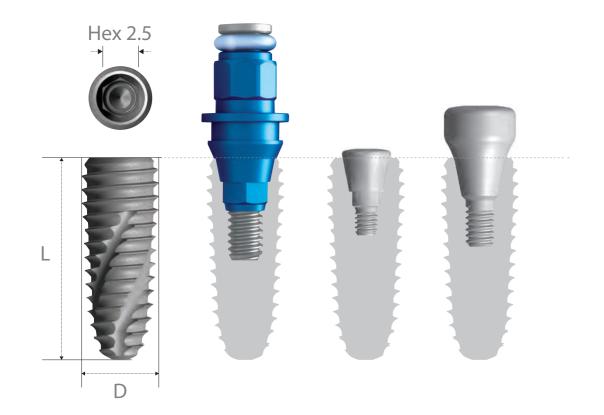

INNO Fixture Code


S	Т	40	10	S	M	*Ex.)	
Туре	body	Diameter	Length	Surface Treatment	Mount	SLA No-Mount	ST 4010 SM
S ubmerged	Taper	Ø 4.0	10 mm	SLA	No- M ount		
-	_	4.0	10	C		*_ ,	
S		40	10	5		*Ex.)	
Туре	body	Diameter	Length	Surface Treatment	Mount	SLA Pre-Mount	ST4010S
S ubmerged	T aper	Ø4.0	10 mm	SLA	Pre-Mount		

No-Mount > Packing unit: 1 Fixture + 1 Cover Screw.


14

18

ST4014SM ST4016SM ST4018SM

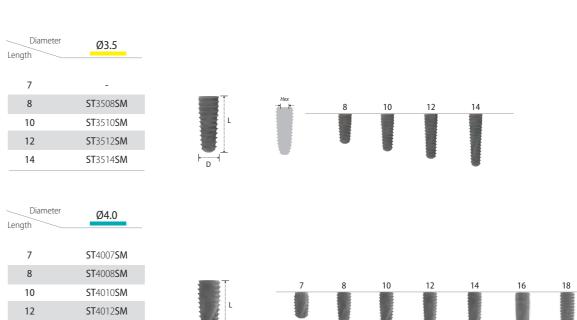

Length	Ø4.5											
7	ST 4507 SM											
8	ST4508SM		■ T	7	0	10	12	1.4	16	10		
10	ST 4510 SM				8	10	12	14	16	18		
12	ST 4512 SM		L				Ŧ	7			- 80	
14	ST 4514 SM					•	-					
16	ST 4516 SM	۱ '	D '					•		ŧ		
18	ST 4518 SM											

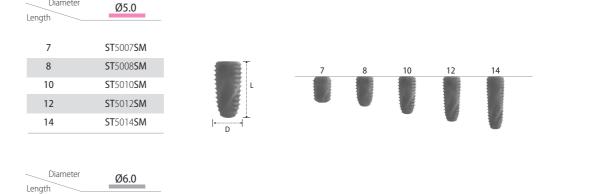

|--|

Pre-Mount > Packing unit: 1 Fixture + 1 Cover Screw + 1 Mount.


Diameter	Ø3.5								
Length				•	40	40			
7	_			8	10	12	14		
8	ST 3508 S	= +	Hex						
10	ST 3510 S								
12	ST 3512 S			-					
14	ST 3514 S	D			-	-			
		D							
Diameter									
Length	Ø4.0								
		-	_		4.0	40			40
7	ST4007S		7	8	10	12	14	16	18
8	ST4008S								
10	ST 4010 S								
12	ST 4012 S	L L							
14	ST4014S	#			#				
16	ST4016S	ŀ D T				_	-		
18	ST 4018 S								•
Diameter	04.5								
Length	Ø4.5								
			7	8	10	12	14	16	18
7	ST 4507 S		_	_	_	_	_	_	_
8	ST 4508 S	= T	1						
10	ST 4510 S								
12	ST 4512 S		-		1/2			- 80	1
14	ST 4514 S				•	*		- 10	
16	ST 4516 S	r D →						w	ŧ
18	ST 4518 S								
Diameter	Ø5.0								
Length	25.0								
J			_	_					
			7	8	10	12	14		
7	ST 5007 S	₫-	7	8	10	12	14		
7	ST5007S ST5008S	—	7	8	10	12	14		
7 8 10	ST5007S ST5008S ST5010S		7	8	10	12	14		
7 8 10 12	ST5007S ST5008S ST5010S ST5012S		7	8	10	12	14		
7 8 10	ST5007S ST5008S ST5010S		7	8	10	12	14		
7 8 10 12 14	ST5007S ST5008S ST5010S ST5012S ST5014S		7	8	10	12	14		
7 8 10 12 14	ST5007S ST5008S ST5010S ST5012S		7	8	10	12	14		
7 8 10 12 14	ST5007S ST5008S ST5010S ST5012S ST5014S						14		
7 8 10 12 14 Diameter	\$T5007\$ \$T5008\$ \$T5010\$ \$T5012\$ \$T5014\$						14		
7 8 10 12 14 Diameter Length	\$T5007\$ \$T5008\$ \$T5010\$ \$T5012\$ \$T5014\$						14		

INNO Submerged Short Implant




Submerged Short Fixture
Surface Treatment: **SLA-SH**®

- > Interchangeable with Hexagonal Morse Tapered Fixture.
- > Internal hex connection (Taper 11°/ Hex 2.5).

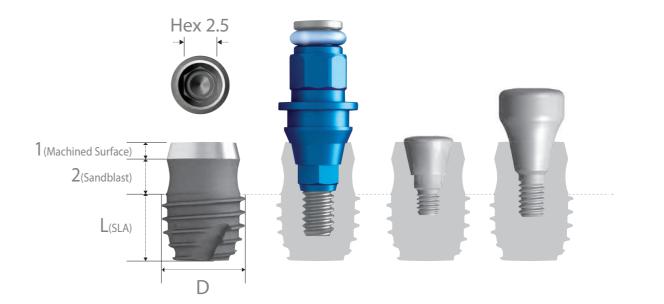

Pre-Mount > Packing Unit: 1 Fixture + 1 Cover Screw + 1 Mount.

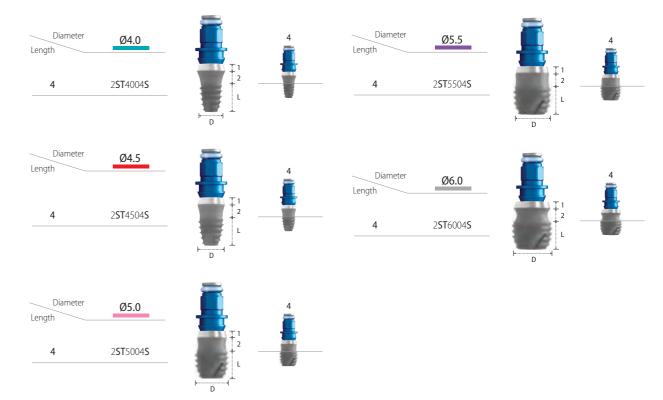

032 INNO-SUBMERGED IMPLANT

INNO-SUBMERGED IMPLANT 033

Fixture Mount



Length	5.4
	2 SMHR 001

- > Packing unit: 1 Mount + 1 Mount Screw.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force: 5~10 N.cm.

Cover Screw	D
T 17	


Diameter Length	Ø3.35	Ø3.75	Ø4.15
3	2 SCS 000		
4.2		* 2 SCS 001	
5.2			* 2 SCS 002

*Extra Product

1mm

- > Packing unit: 1 Cover Screw.
- > To seal the conical interface of fixture.
- > The longer Cover Screw for deeply inserted fixture.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force : 5~10 N.cm.


Healing Abutment

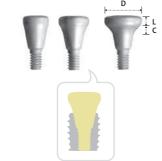

Diameter	Ø4	1.5	Ø5	5.5	Ø6	.5
Length Cuff	1	2	1	2	1	2
1	2 HS 4511		2 HS 5511		2 HS 6511	
2		2 HS 4522		2 HS 5522		2 HS 6522
3		2 HS 4532		2 HS 5532		2 HS 6532
4		2 HS 4542		2 HS 5542		2 HS 6542
5		2 HS 4552		2 HS 5552		2 HS 6552
7		2 HS 4572		2 HS 5572		2 HS 6572
Diameter	<i>a</i> -		a	0.5	a	0.5
Diameter	Ø7	.5	Ø	8.5	Ø	9.5
Length Cuff	2	2		2		2
3	2 HS 7	7532	2HS	8532	2HS	9532

- > Packing unit: 1 Abutment.
- > For remodeling gingival contour during soft tissue healing.
- > Recommended to use with the Volume-up™ Guide System.
- > Select according to gingival height and abutment type.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force : 5~10 N.cm.

Volume-up™ Healing Abutment

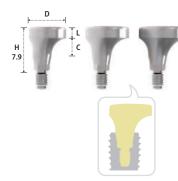


Diameter	Ø6.5	Ø7.5	Ø8.5
Length Cuff	2	2	2
3	VUHN6532	VUHN7532	VUHN8532

- > Packing unit: 1 Abutment(Inbuilt Abutment Screw).
- > Used for an implant procedure to form the gingival tissue and alveolar bone in the form of natural teeth and gums by prevention or minimizing the food penetration.
- > Extremely effective when used with the COWELL® BMP.
- > Recommended to use with the Volume-up™ Guide System.
- ${\color{red}>}$ Select according to gingival height and abutment type.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force: 25~35 N.cm.

Prosthetic Procedure I

Components Selection Guide for Cemented and UCLA Abutment

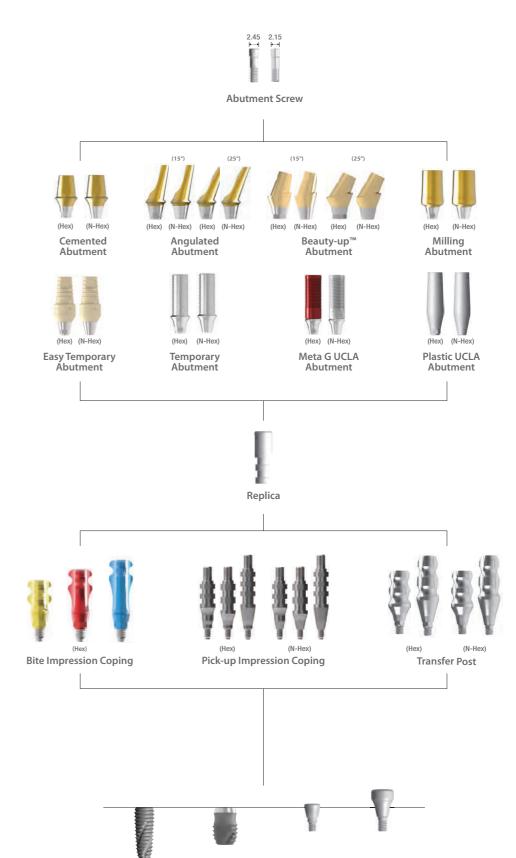

Fixture


034 INNO-SUBMERGED IMPLANT

Healing

Abutment

Cover Screw

Cemented Abutment

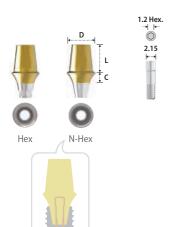


Туре		Hex							
Diameter		Ø4.5			Ø5.5			Ø6.5	
Length Cuff	4	5.5	7	4	5.5	7	4	5.5	7
1	2 SCH 4514	2 SCH 4515	2 SCH 4517	2 SCH 5514	2 SCH 5515	2 SCH 5517	2 SCH 6514	2 SCH 6515	2 SCH 6517
2	2 SCH 4524	2 SCH 4525	2 SCH 4527	2 SCH 5524	2 SCH 5525	2 SCH 5527	2 SCH 6524	2 SCH 6525	2 SCH 6527
3	2 SCH 4534	2 SCH 4535	2 SCH 4537	2 SCH 5534	2 SCH 5535	2 SCH 5537	2 SCH 6534	2 SCH 6535	2 SCH 6537
4	2 SCH 4544	2 SCH 4545	2 SCH 4547	2 SCH 5544	2 SCH 5545	2 SCH 5547	2 SCH 6544	2 SCH 6545	2 SCH 6547
5	2 SCH 4554	2 SCH 4555	2 SCH 4557	2 SCH 5554	2 SCH 5555	2 SCH 5557	2 SCH 6554	2 SCH 6555	2 SCH 6557

Туре		N-Hex							
Diameter		Ø4.5			Ø5.5			Ø6.5	
Length	4	5.5	7	4	5.5	7	4	5.5	7
1	2 SCN 4514	2 SCN 4515	2 SCN 4517	2 SCN 5514	2 SCN 5515	2 SCN 5517	2 SCN 6514	2 SCN 6515	2 SCN 6517
2	2 SCN 4524	2 SCN 4525	2 SCN 4527	2 SCN 5524	2 SCN 5525	2 SCN 5527	2 SCN 6524	2 SCN 6525	2 SCN 6527
3	2 SCN 4534	2 SCN 4535	2 SCN 4537	2 SCN 5534	2 SCN 5535	2 SCN 5537	2 SCN 6534	2 SCN 6535	2 SCN 6537
4	2 SCN 4544	2 SCN 4545	2 SCN 4547	2 SCN 5544	2 SCN 5545	2 SCN 5547	2 SCN 6544	2 SCN 6545	2 SCN 6547
5	2 SCN 4554	2 SCN 4555	2 SCN 4557	2 SCN 5554	2 SCN 5555	2 SCN 5557	2 SCN 6554	2 SCN 6555	2 SCN 6557

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Prosthesis.
- > Cutting surface for anti-rotation of prosthesis.
- > Gold color for more translucent restoration and classification with N-Hex.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Connected with the Abutment Screw (2SSHR200).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Fixture level impression.

Angulated Abutment

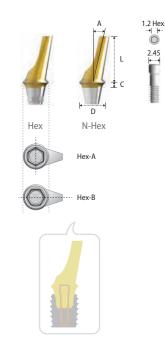


Туре	Hex-A				Hex-B			
Diameter(Angle)	Ø4.5(15°)	Ø4.5(25°)	Ø5.5(15°)	Ø5.5(25°)	Ø4.5(15°)	Ø4.5(25°)	Ø5.5(15°)	Ø5.5(25°)
Length Cuff	8	8	8	8	8	8	8	8
1	2 SAH 45151	2 SAH 45251	2 SAH 55151	2 SAH 55251	2 SAH 45151 B	2 SAH 45251 B	2 SAH 55151 B	2 SAH 55251 B
2	2 SAH 45152	2 SAH 45252	2 SAH 55152	2 SAH 55252	2 SAH 45152 B	2 SAH 45252 B	2 SAH 55152 B	2 SAH 55252 B
3	2 SAH 45153	2 SAH 45253	2 SAH 55153	2 SAH 55253	2 SAH 45153 B	2 SAH 45253 B	2 SAH 55153 B	2 SAH 55253 B
4	2 SAH 45154	2 SAH 45254	2 SAH 55154	2 SAH 55254	2 SAH 45154 B	2 SAH 45254 B	2 SAH 55154 B	2 SAH 55254 B

Туре	N-Hex					
Diameter(Angle)	Ø4.5(15°)	Ø4.5(25°)	Ø5.5(15°)	Ø5.5(25°)		
Length Cuff	8	8	8	8		
1	2 SAN 45151	2 SAN 45251	2 SAN 55151	2 SAN 55251		
2	2 SAN 45152	2 SAN 45252	2 SAN 55152	2 SAN 55252		
3	2 SAN 45153	2 SAN 45253	2 SAN 55153	2 SAN 55253		
4	2 SAN 45154	2 SAN 45254	2 SAN 55154	2 SAN 55254		

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Prosthesis.
- > Solution for anterior esthetic zone.
- > Connected with the Abutment Screw (2SSHR100).
- > Gold color for more translucent restoration and classification with N-Hex.
- > Select Hex-A or Hex-B according to case.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Fixture level impression.

Beauty-up[™] Abutment

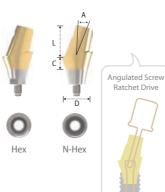

Туре	Hex	N-Hex	Hex	N-Hex
Diameter(Angle)	Ø3.8 (15°)	Ø3.8 (15°)	Ø3.8 (25°)	Ø3.8 (25°)
Length Cuff	5	5	5	5
2	2 SBH 381525	2 SBN 381525	2 SBH 382525	2 SBN 382525


- > Packing unit: 1 Abutment (Inbuilt Abutment Screw). > Tightened with the Angulated Screw Ratchet Drive
- > For Screw-Cement Retained Prosthesis
- with angulated screw channel.
- > Solution for anterior esthetic zone.
- > The gingival line of the Beauty-up™ Abutment allows more esthetic prosthesis.
- > Oval design allows lower incisal application (Mesiodistal diameter: 3.8mm).
- and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Library available for EXOCAD®, 3Shape® & Others. > Use the Scanbody for 3D Work.
- > Fixture level impression.
- * Angulated Screw Ratchet Driver



LDIIVCI	
Height Type	Ratchet
24(Short)	KRBUD15
29(Long)	KRBUD20

- > Stable to internal slip or fracture due to wide contact area of the Angulated Driver and the dedicated Stargrip Abutment Screw.
- > Tightening torque force : 30 N.cm (50 N.cm Max.).

Milling Abutment

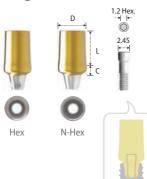

Туре	Hex			N-Hex		
Diameter	Ø4.5	Ø5.5	Ø6.5	Ø4.5	Ø5.5	Ø6.5
Length Cuff	7	7	7	7	7	7
2	2 SMH 4527	2 SMH 5527	2 SMH 6527	2 SMN 4527	2 SMN 5527	2 SMN 6527
4	2 SMH 4547	2 SMH 5547	2 SMH 6547	2 SMN 4547	2 SMN 5547	2 SMN 6547

- > Packing unit: 1 Milling Abutment + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Prosthesis.
- > Block abutment for customized contouring.
- > Gold color for more translucent restoration and classification with N-Hex.
- > Connected with the Abutment Screw (2SSHR100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Fixture level impression.

Easy Temporary Abutment




Туре	He	ex	N-Hex		
Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5	
Length Cuff	10	10	10	10	
2	2 STHA 45 C	2 STHA 55 C	2 STNA 45 C	2 STNA 55 C	

- > Packing unit: 1 Easy Temporary Abutment + 1 Abutment Screw.
- > For Screw Retained Prosthesis.
- > For simpler and speedier chair-side process.
- > Vaneerable polymer material.
- > Temporary restoration for anterior esthetic zone.
- > Titanium core for strength.
- > Connected with the Abutment Screw (2SSHR200).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 20 N.cm.
- > Fixture level impression.

036 INNO-SUBMERGED IMPLANT INNO-SUBMERGED IMPLANT 037


Temporary Abutment

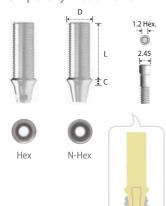

Туре	Hex	N-Hex
Diameter	Ø4.5	Ø4.5
Length Cuff	10	10
1	2 STHA 45	2 STNA 45

- > Packing unit: 1 Temporary Abutment + 1 Abutment Screw.
- > For Screw-Cement Retained Prosthesis.
- > For provisional restoration.
- > Connected with the Abutment Screw (2SSHR100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 20 N.cm.
- > Fixture level impression.


Meta G UCLA Abutment

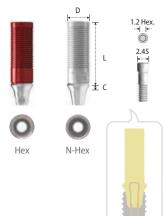

Туре	Hex	N-Hex
Diameter	Ø4.5	Ø4.5
Length Cuff	12	12
1	2 SGH 45 N	2 SGN 45 N
2	2 SGH 452 N	2 SGN 452 N
3	2 SGH 453 N	2 SGN 453 N

- > Packing unit: 1 Meta G UCLA Abutment + 1 Abutment Screw.
- > For Screw-Cement or Screw Retained Prosthesis.
- > Modification to angulated abutment, Customized abutment and telescopic abutment.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the Abutment Screw (2SSHR100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Fixture level impression.

Plastic UCLA Abutment

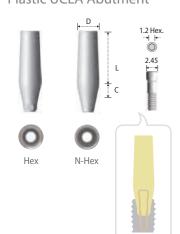

Туре	Hex		N-Hex	
Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Cuff Length	11	11	11	11
3	2 SPHR 001	2 SPHW 001	2 SPNR 001	2 SPNW 001

- > Packing unit: 1 Plastic UCLA Abutment + 1 Abutment Screw.
- > Same purpose of use as the Meta G UCLA Abutment but low accuracy of connection during lab procedure.
- > PMMA material.
- > Connected with the Abutment Screw (2SSHR100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : Finger light force during wax pattern fabrication, 30 N.cm after casting.
- > Fixture level impression.

Abutment Screw



Diameter Height	Ø2.45	Ø2.15
8.5	2 SSHR 100	2 SSHR 200

- > Packing unit: 1 Abutment Screw.
- > 2SSHR100 : Hybrid Block / Scanbody / Angulated / Milling / Temporary / Meta G UCLA / Plastic UCLA Abutment.
- > 2SSHR200 : Hybrid S & L Ti-Base / Cemented / Easy Temporary Abutment.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Replica



Diameter Height	Ø4
12	2 SRHR 001

- > Packing unit: 1 Replica.
- > Mimicking of conical interface of fixture.
- > Analog of fixture for working cast.


Bite Impression Coping



Туре	Hex(Short)	Hex(Long)	Hex(X-Long)
Diameter	Ø4.5	Ø4.5	Ø4.5
Cuff Length	2	4	6
4.0	2 SBIC 45 S	2 SBIC 45 L	2 SBIC 45 X

- > Packing unit: 1 Bite Impression Coping (Inbuilt Guide Pin).
- > Designed to simultaneously take bite and impression.
- > For Closed Tray Impression (Bite Impression).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.


Pick-up Impression Coping

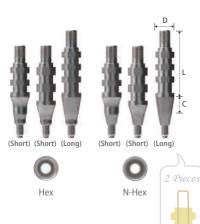

Туре		Hex			N-Hex	
	Ø4.5	Ø5.5	Ø6.5	Ø4.5	Ø5.5	Ø6.5
14 (Short) / 4	2 SIH 454 S	2 SIH 554 S	2 SIH 654 S	2 SIN 454 S	2 SIN 554 S	2 SIN 654 S
14 (Short) / 2	2 SIH 45 S	2 SIH 55 S	2 SIH 65 S	2 SIN 45 S	2 SIN 55 S	2 SIN 65 S
16 (Long) / 4	2 SIH 45 L	2 SIH 55 L	2 SIH 65 L	2SIN45L	2 SIN 55 L	2SIN65L

- > Packing unit: 1 Pick-up Impression Coping + 1 Guide Pin (2SISR001SS / 2SISR001SL).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Transfer Post

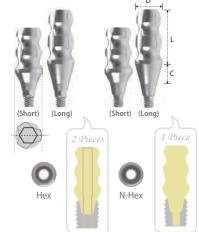


Туре	Hex		N-Hex			
Diameter ogth/Cuff	Ø4.5	Ø5.5	Ø6.5	Ø4.5	Ø5.5	Ø6.5
(Short) / 2	2 STH 45 S	2 STH 55 S	2 STH 65 S	2 STN 45 S	2 STN 55 S	2 STN 65 S
1 (Long) / 4	2 STH 45 L	2 STH 55 L	2 STH 65 L	2STN45L	2 STN 55 L	2 STN 65 L


- > Packing unit: Hex 1 Transfer Post + 1 Guide Pin (2STH001SS / 2STH001SL)
 - N-Hex 1 Transfer Post (Solid Type).
- > For closed tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Prosthetic Procedure II

Component Selection Guide for Multi S&A Abutment

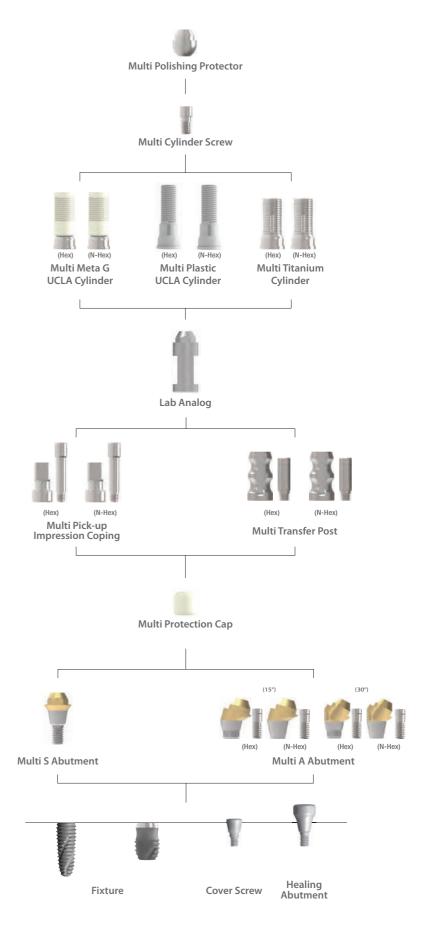


Multi S Abutment

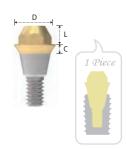

Diameter	Ø4.5	Ø5.5
Cuff Length	2	2
1	2 SMS 451	2 SMS 551
2	2 SMS 452	2 SMS 552
3	2 SMS 453	2 SMS 553
4	2 SMS 454	2 SMS 554
5	2 SMS 455	2 SMS 555

- > Packing unit: 1 Multi S Abutment.
- > For Screw-Retained Prosthesis.
- > Titanium base for the cylinders.
- > Gold color for more translucent restoration.
- > Integrated with screw and abutment.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Use the S Holder for more stable position.
- > Tightened with the S Machine & S Ratchet Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.

Multi A Abutment

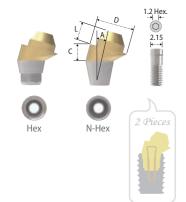


Туре	Hex					
Diameter(Angle)	Ø4.5(15°)	Ø4.5(30°)	Ø5.5(15°)	Ø5.5(30°)		
Length	2	2	2	2		
2	• 2 SMAH 45152					
3	★ 2 SMAH 45153	2SMAH 45303	★ 2 SMAH 55153	★ 2 SMAH 55303		
4	★ 2 SMAH 45154	★ 2 SMAH 45304	★ 2 SMAH 55154	★ 2 SMAH 55304		
5			★ 2 SMAH 55155	★ 2 SMAH 55305		
Туре		N-	Hex			

Туре	N-Hex					
Diameter(Angle)	Ø4.5(15°)	Ø4.5(30°)	Ø5.5(15°)	Ø5.5(30°)		
Length	2	2	2	2		
2	• 2 SMAN 45152					
3	★ 2 SMAN 45153	2SMAN 45303	★ 2 SMAN 55153	★ 2 SMAN 55303		
4	★ 2 SMAN 45154	★ 2 SMAN 45304	★ 2 SMAN 55154	★ 2 SMAN 55304		
5			* 2 SMAN 55155	* 2 SMAN 55305		

- > Packing unit: 1 Multi A Abutment + 1 Abutment Screw.
- > For Screw-Retained Prosthesis.
- > Titanium base for the cylinders.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Use the A Holder for more stable position.
- > Connected with the Abutment Screw (2SSHR300 : ★ / 2SSHR400 : •).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the Multi Scanbody for digital flow.
- > Abutment level impression.

Abutment Screw

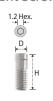

Height Diameter	7.5	6.5
2.15	★ 2 SSHR 300	• 2SSHR400

- > Packing unit: 1 Abutment Screw.
- > To connect the Multi A Abutment.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Multi Protection Cap



Multi S & A Abutment Diameter	Ø4.5	Ø5.5
Diameter Height	Ø5.2	Ø6.2
5	2 SMPC 45	2 SMPC 55

- > Packing unit: 1 Multi Protection Cap.
- $\,>$ Protection from cheek and tongue for gingival healing period.
- > Gingival retraction for prosthodontic margin of abutment.
- > Alternative usage for sub-structure of temporary prosthesis.


Multi Pick-up Impression Coping



Туре	Hex		N-Hex	
Multi S & A Abutment Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
— Diameter Height —	Ø4.65	Ø5.65	Ø4.65	Ø5.65
14.8	2 SMIH 45	2 SMIH 55	2 SMIN 45	2 SMIN 55

- > Packing unit: 1 Multi Pick-up Impression Coping + 1 Guide Pin (2SMGP012).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Multi Transfer Post

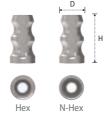



Туре	Hex		N-Hex	
Multi S & A Abutment Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
— Diameter Height —	Ø4.5	Ø5.5	Ø4.5	Ø5.5
8.5	2 SMTH 45	2 SMTH 55	2 SMTN 45	2 SMTN 55

- > Packing unit: 1 Multi Transfer Post + 1 Guide Pin (2SMTHS100).
- > For closed tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

042 INNO-SUBMERGED IMPLANT INNO-SUBMERGED IMPLANT 043

Multi Lab Analog



Multi S & A Abutment Diameter	Ø4.5	Ø5.5
Diameter Length	Ø4.5	Ø5.5
2	2 SMA 45	2 SMA 55

- > Packing unit : 1 Multi Lab Analog.
- > Replacement of abutment shape in working cast.
- > Choose by abutment size.

Multi Titanium Cylinder









Туре	Hex		N-H	lex
Multi S & A Abutment Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Length Cuff	8.5	8.5	8.5	8.5
0.5	2 STCH 45	2 STCH 55	2 STCN 45	2 STCN 55

- > Packing unit: 1 Multi Titanium Cylinder + 1 Cylinder Screw.
- > For Screw, Cement or Screw-Cement Retained Prosthesis.
- > Connected with the Cylinder Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

Multi Meta G UCLA Cylinder



Туре	Hex		N-I	lex
Multi S & A Abutment Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Length Cuff	10.9	10.9	10.9	10.9
0.5	2 SCCH 45	2 SCCH 55	2 SCCN 45	2 SCCN 55

- > Packing unit: 1 Multi Meta G UCLA Cylinder + 1 Multi Cylinder Screw.
- > For Screw, Cement or Screw-Cement Retained Prosthesis.
- > Modification to various types of abutments.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the Multi Cylinder Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.


Multi Cylinder Screw



Diameter Height	Ø2.25
5	2 SMCS 100

- > Packing unit: 1 Multi Cylinder Screw.
- > Connected with the Meta G UCLA, Plastic UCLA and Titanium Cylinder.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

Multi Plastic UCLA Cylinder

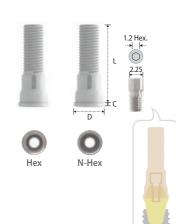

Туре	Hex		N-Hex	
Multi S & A utment Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Length Cuff	11.5	11.5	11.5	11.5
0.5	2 SMPH 45	2 SMPH 55	2 SMPN 45	2 SMPN 55


- > Packing unit: 1 Multi Plastic UCLA Cylinder + 1 Multi Cylinder Screw.
- > For Screw, Cement or Screw-Cement Retained Prosthesis.
- > Same purpose of use as the Meta G UCLA Cylinder but low accuracy of connection.
- > PMMA material.
- > Connected with the Multi Cylinder Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

Multi Polishing Protector



Туре	Hex	
Multi S & A Abutment Diameter	Ø4.5	Ø5.5
Diameter Length	Ø4.5	Ø5.5
2	2 SMPP 45	2 SMPP 55

- > Packing unit: 1 Protector.
- > For polishing work during lab procedure.

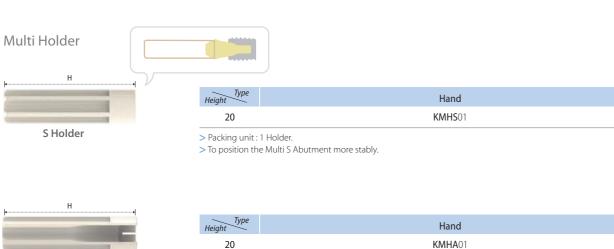


A Holder

1.2 Hex Driver



- > Packing unit: 1 Holder.
- > To position the Multi A Abutment more stably.

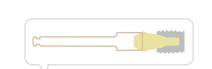


Multi S Machine Driver



Type Height	Machine
27.5	KMMSD21L

- > Packing unit: 1 Multi S Machine Driver.
- > To install the Multi S Abutment by machine.

Multi S Ratchet Driver

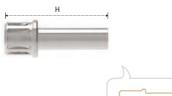


Type Height	Ratchet
22	KRMSD15L


- > Packing unit: 1 Multi S Ratchet Driver.
- > To install the Multi S Abutment by hand.

Prosthetic Procedure III

Component Selection Guide for Lock Abutment

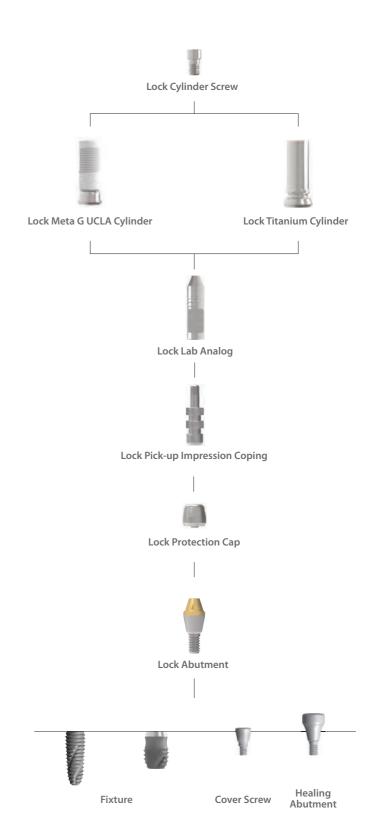

Lock Abutment

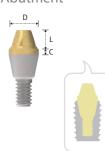

Diameter	Ø4.5
Cuff Length	2.15
0.5	2 SLA 400
1	2 SLA 410
2	2 SLA 420
3	2 SLA 430
4	2 SLA 440

- > Packing unit: 1 Lock Abutment.
- > For Screw-Retained Prosthesis.
- > Titanium base for cylinder.
- > Gold color for more translucent restoration.
- > Integrated with screw and abutment.
- > Tightened with the Lock Ratchet Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.


Lock Protection Cap



Diameter Height	Ø4.5
4.3	2 SLP 45

- > Packing unit: 1 Lock Protection Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Gingival retraction for prosthodontic margin of abutment.

Lock Pick-up Impression Coping

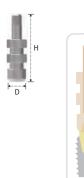

16 2SLIH45	Diameter Height	Ø4.5
	16	2 SLIH 45

- > Packing unit: 1 Lock Pick-up Impression Coping + 1 Guide Pin (2SLIH45S).
- > For open tray impression.


Lock Lab Analog



Diameter Length	Ø4.5
2.15	2 SLA 45

- > Packing unit: 1 Lock Lab Analog.
- > Replacement of abutment shape in working cast.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.


Lock Meta G UCLA Cylinder

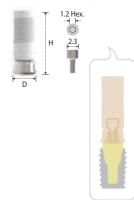

Diameter Height	Ø4.5
11.2	2 SLCH 45

- > Packing unit: 1 Lock Meta G UCLA Cylinder + 1 Cylinder Screw.
- > For Screw, Cement or Screw-Cement Retained Prosthesis.
- > Modification to various types of abutments.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the Cylinder Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

Lock Titanium Cylinder

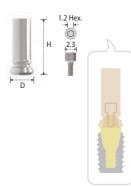

Diameter Height	Ø4.5
10	2 SLTH 45

- > Packing unit: 1 Lock Titanium Cylinder + 1 Cylinder Screw.
- > For Screw, Cement or Screw-Cement Retained Prosthesis.
- > Connected with the Cylinder Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.


048 INNO-SUBMERGED IMPLANT INNO-SUBMERGED IMPLANT 049

Lock Cylinder Screw




Diameter Height	Ø2.3
4.8	2 SLCS 200

- > Packing unit: 1 Cylinder Screw.
- > Connected with the CCM Cylinder or Titanium Cylinder.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

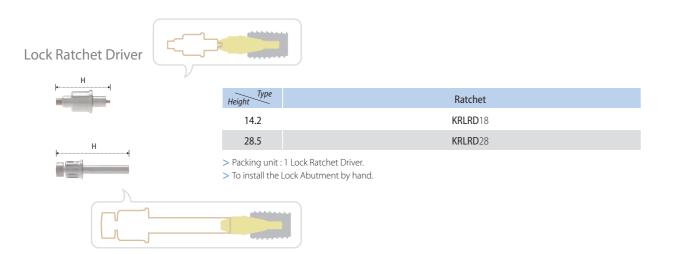

Prosthetic Procedure IV

Component Selection Guide for Absolute Abutment

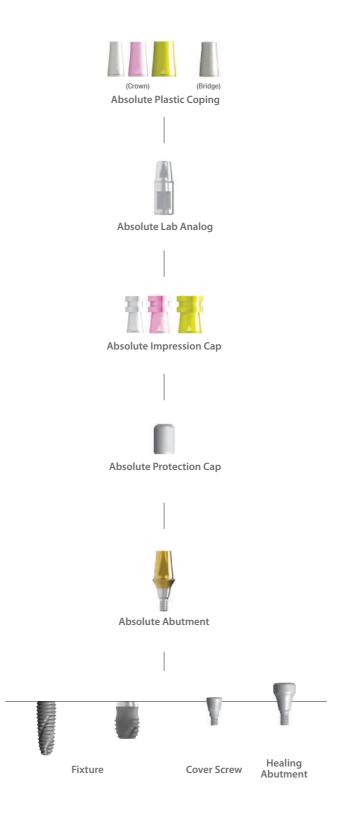

Absolute Abutment



Diameter		Ø4.5			Ø5.5	
Length Cuff	4	5.5	7	4	5.5	7
1	2 SAC 4514	2 SAC 4515	2 SAC 4517	2 SAC 5514	2 SAC 5515	2 SAC 5517
2	2 SAC 4524	2 SAC 4525	2 SAC 4527	2 SAC 5524	2 SAC 5525	2 SAC 5527
3	2 SAC 4534	2 SAC 4535	2 SAC 4537	2 SAC 5534	2 SAC 5535	2 SAC 5537
4	2 SAC 4544	2 SAC 4545	2 SAC 4547	2 SAC 5544	2 SAC 5545	2 SAC 5547
5	2 SAC 4554	2 SAC 4555	2 SAC 4557	2 SAC 5554	2 SAC 5555	2 SAC 5557

Diameter		Ø6.5	
Length Cuff	4	5.5	7
1	2 SAC 6514	2 SAC 6515	2 SAC 6517
2	2 SAC 6524	2 SAC 6525	2 SAC 6527
3	2 SAC 6534	2 SAC 6535	2 SAC 6537
4	2 SAC 6544	2 SAC 6545	2 SAC 6547
5	2 SAC 6554	2 SAC 6555	2 SAC 6557

- > Packing unit: 1 Absolute Abutment + 1 Production Cap.
- > For Cement Retained Prosthesis.
- > Cutting surface for anti-rotation of prosthesis.
- > Integrated with the Screw and Abutment.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.

Absolute Protection Cap



Abut. Diameter	Ø4.5	Ø5.5	Ø6.5
Diameter Height	Ø5.0	Ø6.0	Ø7.0
6	2 SHPC 454	2 SHPC 554	2 SHPC 654
7.5	2 SHPC 455	2 SHPC 555	2 SHPC 655
9	2 SHPC 457	2 SHPC 557	2 SHPC 657

- > Packing unit: 1 Absolute Protection Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Gingival retraction for prosthodontic margin of abutment.
- > Alternative usage for sub-structure of temporary prosthesis.

Absolute Impression Cap



Abut. Diameter	Ø4.5	Ø5.5	Ø6.5
Diameter Height	Ø5.5	Ø6.5	Ø7.5
10.3	2 SIC 45	2 SIC 55	2 SIC 65

- > Packing unit: 1 Absolute Impression Cap.
- > Confirm locking with abutment by rotation of clockwise and anti-clockwise direction.

Absolute Lab Analog




Diameter Ø4.5 Ø5.5 Ø6.5 4.1 2SHLA454 2SHLA554 2SHLA654 5.6 2SHLA455 2SHLA555 2SHLA655 7.1 2SHLA457 2SHLA557 2SHLA657	Abut. Diameter	Ø4.5	Ø5.5	Ø6.5
5.6 2SHLA455 2SHLA555 2SHLA655	Diameter Length	Ø4.5	Ø5.5	Ø6.5
510	4.1	2 SHLA 454	2 SHLA 554	2 SHLA 654
7.1 2SHLA457 2SHLA557 2SHLA657	5.6	2 SHLA 455	2 SHLA 555	2 SHLA 655
	7.1	2 SHLA 457	2 SHLA 557	2 SHLA 657

- > Packing unit: 1 Absolute Lab Analog.
- > Replacement of abutment shape in working cast.
- > Choose according to width and length of abutment.

Absolute Plastic Coping (Burn Out Cylinder)



Crown

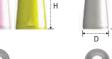

	1
	Н
	1
, D	1



- Туре Crown Bridge Ø6.5 Ø4.5 Ø5.5 Ø6.5 Ø4.5 Ø5.5 Ø5.1 Ø6.1 Ø7.1 Ø5.1 Ø6.1 Ø7.1 2**SHBC**45 2**SHBC**55 2**SHBC**65 2**SHBB**45 2**SHBB**55
- > Packing unit: 1 Absolute Plastic Coping.
- > Connected with the Lab Analog.
- > Burn out and casting for metal framework.

Prosthetic Procedure V

Component Selection Guide for Straight Abutment

Fixture






Cover Screw

Fixture


Straight Abutment

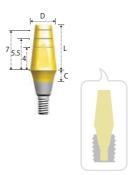

Diameter	Ø3.5	Ø4.5
Length Cuff	8	8
0.5	2 SSCM 308	2 SSCR 408
1	2 SSCM 318	2 SSCR 418
2	2 SSCM 328	2 SSCR 428
3	2 SSCM 338	2 SSCR 438
4	2 SSCM 348	2 SSCR 448

- > Packing unit: 1 Straight Abutment.
- > For Cement Retained Prosthesis.
- > Integrated with screw and abutment.
- > Tightened with the Shoulder Driver.
- > Tightening torque force : 30 N.cm.
- > Direct impression.



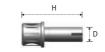


Diameter Height	Ø4.5
19	KRR19L

- > Packing unit: 1 Driver.
- > To install the Straight Abutment.

Prosthetic Procedure VI

Component Selection Guide for Sonator Abutment

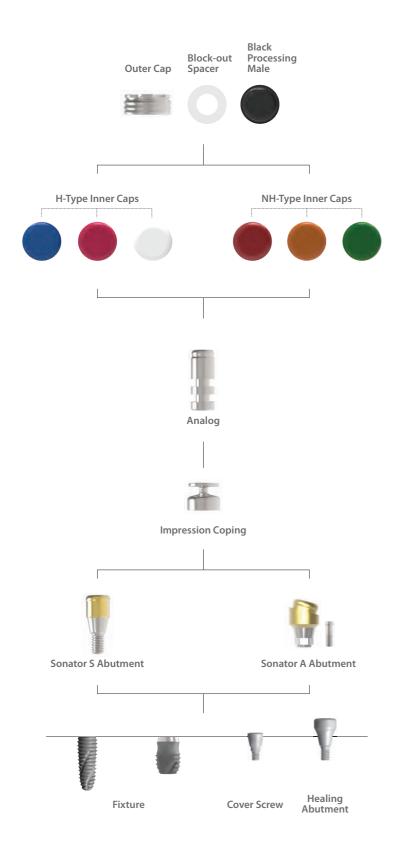

Sonator S Abutment





Diameter			Ø4	1.0		
Length Cuff	1	2	3	4	5	6
1.5	SONS401	SONS402	SONS 403	SONS404	SONS 405	SONS406

- $> Packing\ unit: Sonator\ S\ Abutment + Carrier + 3\ H-Type\ Inner\ Caps + Outer\ Cap + Block-out\ Spacer + Black\ Processing\ Male.$ > For Implant Supported Over-denture Prosthesis.
- > Stable with low vertical height.
- > 6 kinds of the Inner Caps give various holding force (Both, H and NH-Type Inner Caps are used for the Sonator S Abutment).
- > Path compensation up to 20° based on 2 implants.
- > Carrier: Used for delivery of the abutment.
- > Tightened with the Sonator S Ratchet Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.

Sonator A Abutment





Diameter	Ø4.0		
Length	1.5	3.0	
Angle	3	3	
15°	SONA 415	SONA 430	

- > Packing unit: Sonator A Abutment + Abutment Screw + Carrier + 3 NH-Type Inner Caps + Outer Cap + Block-out Spacer + Black Processing Male.
- > For Implant Supported Over-denture Prosthesis.
- > Stable with low vertical height.
- > 6 kinds of Inner Caps give various holding force (Both, H and NH-Type Inner Caps are used for Sonator A Abutment).
- > Path compensation up to 40° based on 2 Implants.
- > Connected with the Abutment Screw (2SSHR300).
- > Carrier: Used for delivery of the abutment.
- > Tightened with the 1.2 Hex Driver and Torque wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.

Abutment Screw



Diameter Height	Ø2.15	
7.5	2 SSHR 300	

054 INNO-SUBMERGED IMPLANT INNO-SUBMERGED IMPLANT 055

Outer Cap



Black Processing Male

Diameter Height	Ø5.4
2.25	SONOC01

- > Packing unit: 2 Outer Caps and 2 Black Processing Males.
- > Black Processing Male: Inserted and removed with the I&R Driver.

H-Type Inner Cap



SONIC01

- > Packing unit: 3 Block-out Spacers + 3 Inner Caps (1 Blue, 1 Pink and 1 White).
- > Path compensation up to 20° based on 2 implants.
- > Mainly used for the Sonator S Abutment.
- > Inner Caps: Inserted and removed with the I&R Driver.



NH-Type Inner Cap

Block-out

Spacer

Code

SONIC02

- > Packing unit: 3 Block-out Spacers + 3 Inner Caps (1 Red, 1 Orange and 1 Green).
- > Non-humped design.
- > Path compensation up to 40° based on 2 implants.
- > Mainly used for the Sonator A Abutment.
- > Inner Caps : Inserted and removed with the I&R Driver.



Inner Cap

Sonator Impression Coping



Diameter Length	Ø4.8
3	SONIP04

- > Packing unit: 4 Impression Copings and 4 Black Processing Males.
- > Connected over the Sonator Abutment after placing the Block-out Spacer.
- > For close tray impression.


Sonator Lab Analog



Diamete Length	Ø4
1.4	SONLA04

- > Packing unit: 4 Lab Analogs.
- > Replacement of abutment shape in working cast.


Sonator S Ratchet Driver

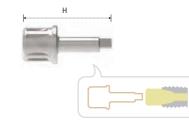

Type Height	Ratchet
18	SONRD19L

> Used to tighten and untighten the Sonator S Abutment.


I & R Driver

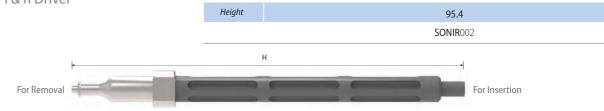


> Used to insert and remove the Inner Caps and Block Processing Male.

Prosthetic Procedure VII

Component Selection Guide for Ball Abutment

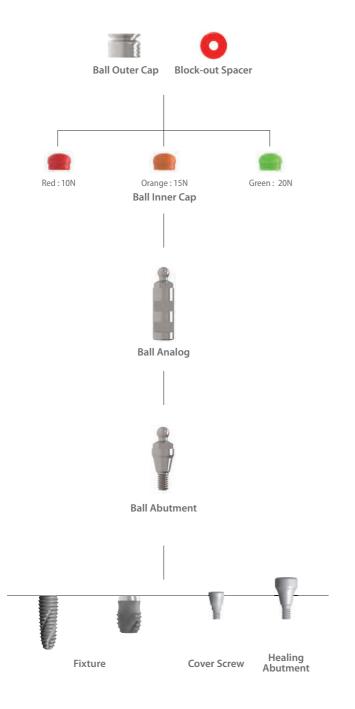

Ball Abutment

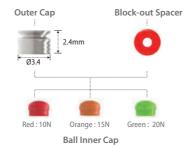



> Packing unit: 1 Ball Abutment + 3 Inner Caps (1 per each colour) + 1 Block-out Spacer + 1 Outer Cap.

Ø4.0

- > For Implant Supported Over-denture Prosthesis.
- > Tightened with the Ball Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Direct impression.

Diameter

Ball Outer Cap



Diameter Height	Ø3.4
2.4	BATC003C

> Packing unit: 2 Outer Caps.

Ball Inner Cap



Ball Abutment

> Packing unit: 2 Blo	ock-out Spacers + 6 Inner	Caps (2 per each color).
-----------------------	---------------------------	--------------------------

> Retention force: Red 10N, Orange 15N & Green 20N.

Ball Analog



	Ø4.0
4	SBAL400

BATC003I

- > Packing unit: 4 Lab Analogs.
- > Replacement of abutment shape in working cast.



٠,		
	Type Height	Ratchet
	19	KRB19L

> Packing unit: 1 Ball Driver.

> Used with the Torque Wrench to tighten and untighten the Ball Abutment.

*Extra Product

I&R Driver




- > Packing unit: 1 I&R Driver.
- > Used to insert and remove the Inner Caps into and out of the Outer Cap.

058 INNO-SUBMERGED IMPLANT

INNO SUBMERGED NARROW IMPLANT (Sub-N.)

System Flow

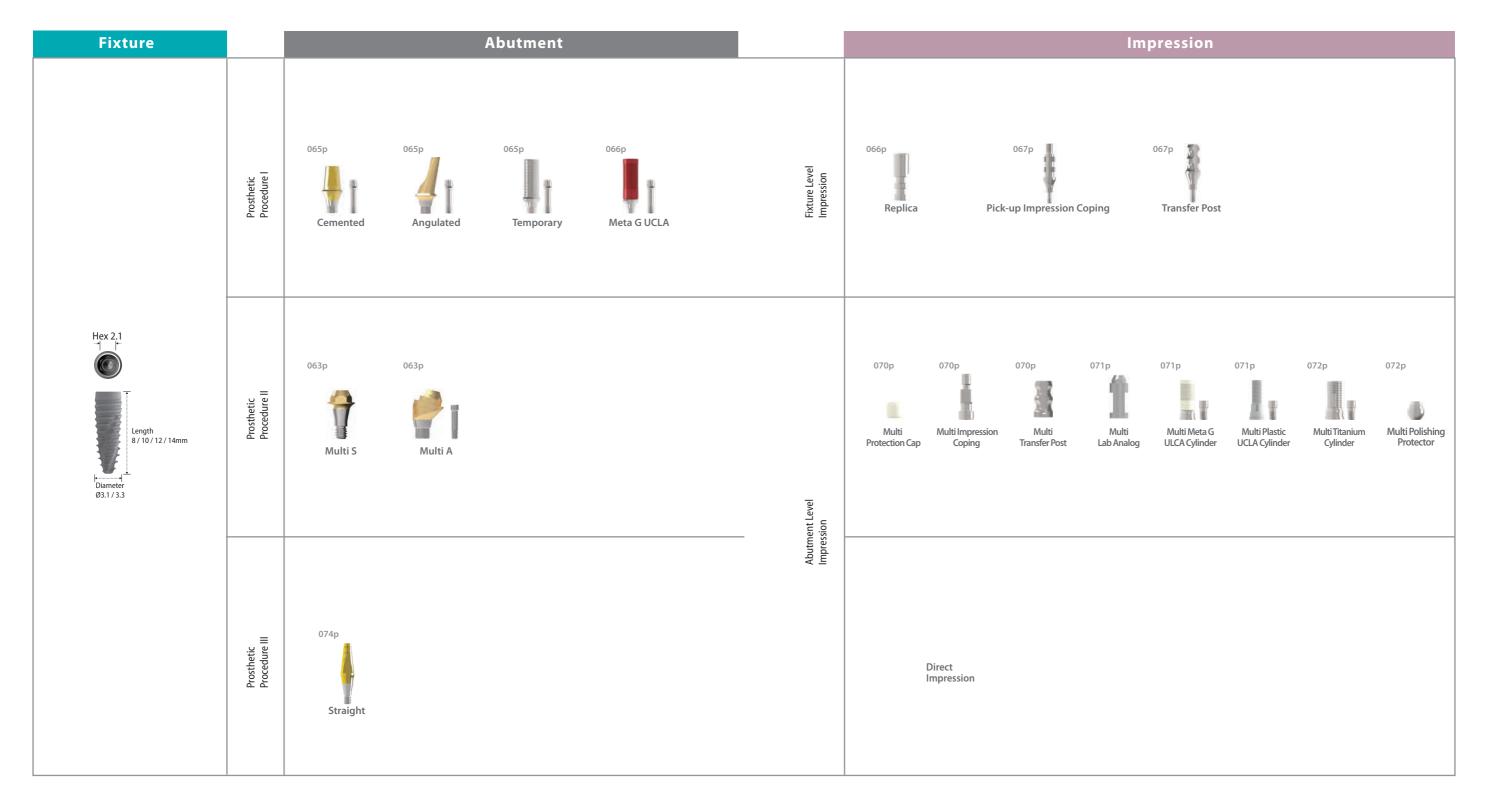

INNO Submerged Narrow Implant (Sub-N.)

SUB-N. HEXAGON SYSTEM

Submerged Fixture Surface Treatment: SLA-SH®

- > Interchangeable with hexagonal morse tapered fixture.
- > Internal hex connection (Taper 11°/ Hex 2.1).

% Note


> The INNO Sub. Narrow System is not compatible with the INNO Submerged System as hexagon size is different.



No-Mount > Packing unit: 1 Fixture + 1 Cover Screw.






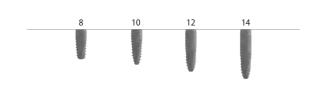


Diameter Length	Ø3.3
Lengui	
8	SR 3308 NSM
10	SR 3310 NSM
12	SR 3312 NSM
14	SR 3314 NSM

Diameter




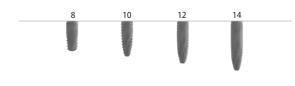

Pre-Mount > Packing unit: 1 Fixture + 1 Cover Screw + 1 Mount.

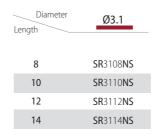




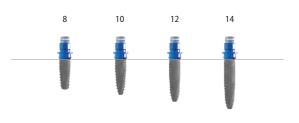





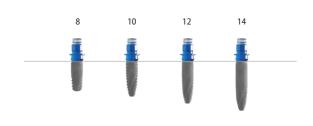

Fixture Mount



Length	5.4
	RSM 001

- > Packing unit: 1 Mount + 1 Mount Screw.
- > Tightened with the 1.2 Hex Driver. > Tightening torque force : 5~10 N.cm.

Cover Screw



	Ø2.85	Ø3.25	Ø3.6
1.7	RCS000		
2.7		RCS001	
3.7			RCS002

- > Packing unit: 1 Cover Screw.
- > To seal the conical interface of fixture.
- > The longer the Cover Screw for deeply inserted fixture.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force: 5~10 N.cm.

1000	P D →
	\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\

Healing Abutment


Diameter	Ø3.5		Ø4	4.5
Cuff Length	1	2	1	2
0.5	HR 3501			
1	HR 3511		HS 4511 N	
2		HR 3522		HS 4522 N
3		HR 3532		HS4532N
4		HR 3542		HS4542N
5				HS4552N
7				HS4572N

- > Packing unit: 1 Abutment.
- > For remodeling gingival contour during soft tissue healing.
- > Select according to gingival height and abutment type.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force: 5~10 N.cm.


062 INNO-SUB. NARROW IMPLANT INNO-SUB. NARROW IMPLANT 063

Prosthesis Procedure I

Components Selection Guide for Cemented and UCLA Abutment

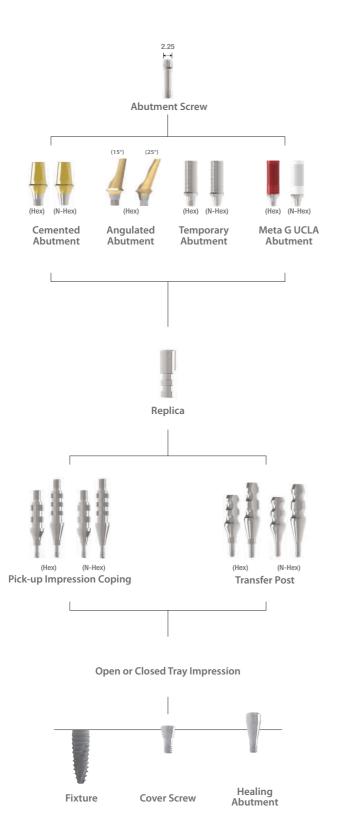


Cemented Abutment

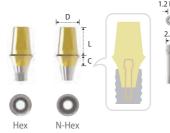

Hex.	Туре			N-Hex Ø4.5			
))	Diameter						
.25 •	Length Cuff	4	5.5	7	4	5.5	7
t	1	SCH4514N	SCH 4515 N	SCH 4517 N	SCN4514N	SCN 4515 N	SCN4517N
	2	SCH4524N	SCH4525N	SCH4527N	SCN4524N	SCN4525N	SCN4527N
	3	SCH4534N	SCH4535N	SCH4537N	SCN4534N	SCN4535N	SCN4537N
	4	SCH4544N	SCH4545N	SCH4547N	SCN4544N	SCN4545N	SCN4547N
	5	SCH4554N	SCH4555N	SCH4557N	SCN4554N	SCN4555N	SCN4557N

- > Packing unit: 1 Cemented Abutment + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Prosthesis.
- > Cutting surface for anti-rotation of prosthesis.
- > Gold color for more translucent restoration and classification with N-Hex.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Connected with the Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Fixture level impression.


Angulated Abutment

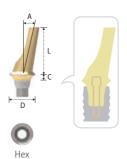

	Туре	Hex		
	Diameter(Angle)	Ø4.5(15°)	Ø4.5(25°)	
lex.	Length Cuff	8	8	
5	1	SAH 45151 NA	SAH 45251 NA	
1	2	SAH 45152 NA	SAH 45252 NA	
	3	SAH 45153 NA	SAH 45253 NA	
	4	SAH 45154 NA	SAH 45254 NA	

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Prosthesis.
- > Solution for anterior esthetic zone.
- > Gold color for esthetics.
- > Connected with the Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Fixture level impression.

Temporary Abutment

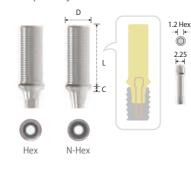


Туре	Hex	N-Hex
Diameter	Ø4.5	Ø4.5
Length Cuff	10	10
1	STHA45N	STNA45N

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Screw-Cement Retained Prosthesis.
- > For provisional restoration.
- > Connected with the Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 20 N.cm.

064 INNO-SUB. NARROW IMPLANT INNO-SUB. NARROW IMPLANT 065


Meta G UCLA Abutment

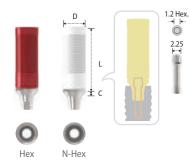

Туре	Hex	N-Hex
Diameter	Ø4.5	Ø4.5
Length Cuff	12	12
1	SGH45N	SGN45N
2	SGH452N	SGN 452 N
3	SGH453N	SGN 453 N

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Screw Retained Prosthesis.
- > Modification to angulated abutment, customized abutment and telescopic abutment.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Fixture level impression.

Abutment Screw

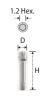

Diameter Height	2.25
10.2	SSHR100N

- > Packing unit: 1 Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

Replica



Diameter Height	Ø4
12.1	SRHR001N

- > Packing unit: 1 Replica.
- > Mimicking of conical interface of fixture.
- > Analog of fixture for working cast.

Pick-up Imp	ression Cop	oing
(Short) (Long)	(Short) (Long)	2 Pieces
	0	
Hex	N-Hev	

	Туре	Hex	N-Hex
	Diameter Length/Cuff	Ø4.5	Ø4.5
	14 (Short) / 2	SIH45SN	SIN45SN
ces	16 (Long) / 4	SIH45LN	SIN45LN

- > Packing unit: 1 Pick-up Impression Coping + 1 Guide Pin.
- > For open tray impression.
- > Connected with the Guide Pin (SIS001SN / SIS001LN).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Transfer Post	
(Short): (Long)	(Short) (Long)
	0
Hex	N-Hex
2 Pieces	1 Piece


Туре	Hex	N-Hex
Diameter Length/Cuff	Ø4.5	Ø4.5
9 (Short) / 2	STH45SN	STN45SN
11 (Long) / 4	STH45LN	STN45LN

- > Packing unit: 1 Transfer Post + 1 Guide Pin (Hex) / 1 Transfer Post (N-Hex, Solid).
- > For closed tray impression.
- > Connected with the Guide Pin (STS001SN / STS001LN).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

066 INNO-SUB. NARROW IMPLANT

Prosthesis Procedure II

Component Selection Guide for Multi S&A Abutment

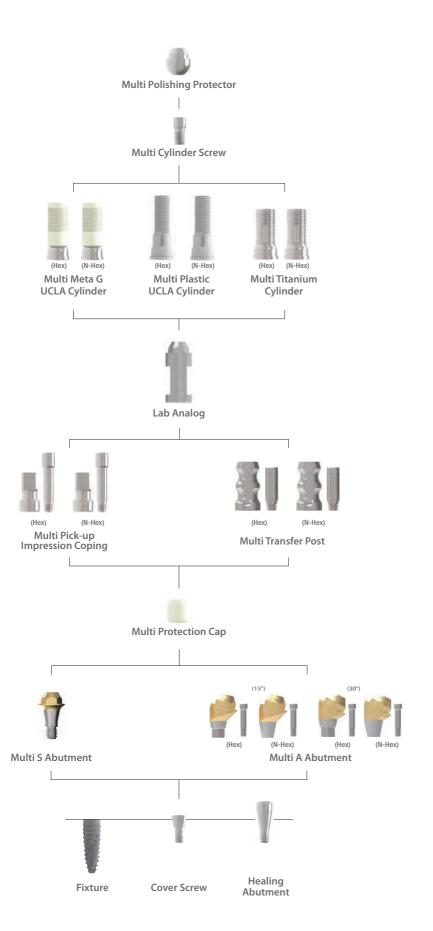

Multi S Abutment



Diameter	Ø4.5
Cuff Length	2
1	SMS 451 N
2	SMS452N
3	SMS 453 N
4	SMS454N

- > Packing unit : 1 Multi S Abutment.
- > For Screw-Retained Prosthesis.
- > Titanium base for the cylinders.
- > Gold color for more translucent restoration.
- > Integrated with screw and abutment.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Use the S Holder for more stable position.
- > Tightened with the S Machine & S Ratchet Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.

Multi A Abutment




Туре	Hex	
Diameter(Angle)	Ø4.5(15°)	Ø4.5(30°)
Length	2	2
2	★ SMAH 45152 BN	
3	• SMAH45153BN	★ SMAH45303BN
4	• SMAH45154BN	• SMAH45304BN

Туре	N-Hex	
Diameter(Angle)	Ø4.5(15°)	Ø4.5(30°)
Length	2	2
2	★ SMAN45152NB	
3	• SMAN45153NB	★ SMAN 45303 NB
4	• SMAN45154NB	• SMAN45304NB

- > Packing unit: 1 Multi A Abutment + 1 Abutment Screw.
- > For Screw-Retained Prosthesis.
- > Titanium base for the cylinders.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Use the A Holder for more stable position.
- > Connected with the Abutment Screw (SSHR200N : ★ / SSHR300N :).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.

Abutment Screw

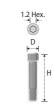

Height Diameter	8.7	9.3
Ø1.95	★ SSHR200N	• SSHR300N

- > Packing unit: 1 Abutment Screw.
- > To connect the Multi A Abutment.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Multi Protection Cap



Multi S & A Abutment Diameter	Ø4.5
Diameter Height	Ø5.2
5	2 SMPC 45

- > Packing unit: 1 Multi Protection Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Gingival retraction for prosthodontic margin of abutment.
- > Alternative usage for sub-structure of temporary prosthesis.


Multi Lab Analog



Multi S & A Abutment Diameter	Ø4.5
Diameter Length	Ø4.5
2	2 SMA 45

- > Packing unit: 1 Multi Lab Analog.
- > Replacement of abutment shape in working cast.

Multi Pick-up Impression Coping

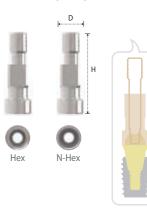

Туре	Hex	N-Hex
Multi S & A Abutment Diameter	Ø4.5	Ø4.5
Diameter Height	Ø4.65	Ø4.65
14.8	2 SMIH 45	2 SMIN 45

- > Packing unit: 1 Multi Pick-up Impression Coping + 1 Guide Pin (2SMGP012).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.


Multi Meta G UCLA Cylinder



Туре	Hex	N-Hex
Multi S & A Abutment Diameter	Ø4.5	Ø4.5
Diameter	Ø4.5	Ø4.5
Length Cuff	10.9	10.9
0.5	2 SCCH 45	2 SCCN 45

- > Packing unit: 1 Multi Meta G UCLA Cylinder + 1 Multi Cylinder Screw.
- > For Screw, Cement or Screw-Cement Retained Prosthesis.
- > Modification to various types of abutments.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the Multi Cylinder Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.


Multi Transfer Post

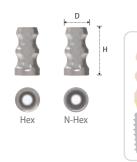

Туре	Hex	N-Hex
Multi S & A Abutment Diameter	Ø4.5	Ø4.5
Diameter Height	Ø4.5	Ø4.5
8.5	2 SMTH 45	2 SMTN 45

- > Packing unit: 1 Multi Transfer Post + 1 Guide Pin (2SMTHS100).
- > For closed tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Multi Plastic UCLA Cylinder

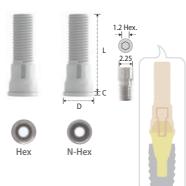

Туре	Hex	N-Hex
Multi S & A Abutment Diameter	Ø4.5	Ø4.5
Diameter	Ø4.5	Ø4.5
Length Cuff	11.5	11.5
0.5	2 SMPH 45	2 SMPN 45

- > Packing unit: 1 Multi Plastic UCLA Cylinder + 1 Multi Cylinder Screw.
- > For Screw, Cement or Screw-Cement Retained Prosthesis.
- > Same purpose of use as the Meta G UCLA Cylinder but low accuracy of connection.
- > PMMA material.
- > Connected with the Multi Cylinder Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

070 INNO-SUB. NARROW IMPLANT INNO-SUB. NARROW IMPLANT 071

Multi Titanium Cylinder







Туре	Hex	N-Hex
Multi S & A Abutment Diameter	Ø4.5	Ø4.5
Diameter	Ø4.5	Ø4.5
Length Cuff	8.5	8.5
0.5	2 STCH 45	2 STCN 45

- > Packing unit: 1 Multi Titanium Cylinder + 1 Cylinder Screw.
- > For Screw, Cement or Screw-Cement Retained Prosthesis.
- > Connected with the Cylinder Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

Multi Cylinder Screw



Diameter Height	Ø2.25
5	2 SMCS 100


- > Packing unit: 1 Multi Cylinder Screw.
- > Connected with the Meta G UCLA, Plastic UCLA and Titanium Cylinder.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

Multi Polishing Protector




Туре	Hex
Multi S & A Abutment Diameter	Ø4.5
Diameter Length	Ø4.5
2	2 SMPP 45

- > Packing unit: 1 Protector.
- > For polishing work during lab procedure.

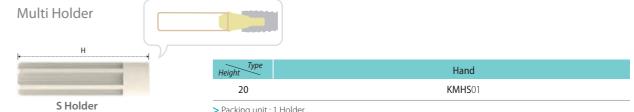

- > Packing unit: 1 Holder.
 - > To position the Multi S Abutment more stably.

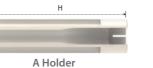


1.2 Hex Driver

Type Height	Hand
20	KMHA01

- > Packing unit: 1 Holder.
 - > To position the Multi A Abutment more stably.



Multi S Machine Driver

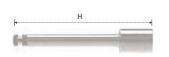

Type Height	Machine
27.5	KMMSD21L

- > Packing unit: 1 Multi S Machine Driver.
- > To install the Multi S Abutment by machine.

Multi S Ratchet Driver






- > Packing unit: 1 Multi S Ratchet Driver.
- > To install the Multi S Abutment by hand.



072 INNO-SUB. NARROW IMPLANT INNO-SUB. NARROW IMPLANT 073

Prosthesis Procedure III

Component Selection Guide for Straight Abutment

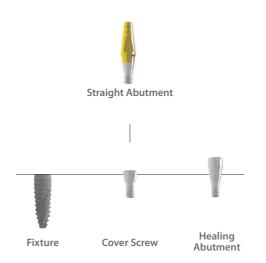

Straight Abutment



Diameter Ø3.5					
Length [Cuff]	8 [0.5]	8 [1]	8 [2]	8 [3]	8 [4]
	SR 308	SR 318	SR 328	SR 338	SR 348


- > Packing unit: 1 Straight Abutment.
- > For Cement Retained Prosthesis.
- > Integrated with screw and abutment.
- > Tightened with the Shoulder Driver.
- > Tightening torque force : 30 N.cm.
- > Direct impression.



4	Н	
		ŤD

Diameter Height	Ø4.5
19	KRR19L

- > Packing unit: 1 Driver.
- > To install the Straight Abutment.

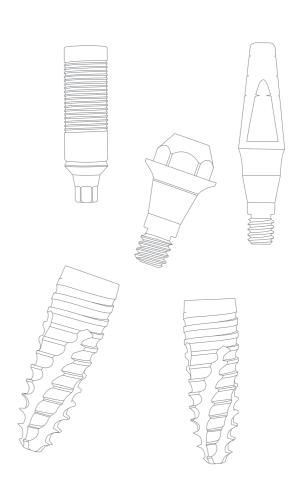

INNO INTERNAL IMPLANT (Int.)

System Flow


Fixture		Abutment		Impression
Octa 3.1 Platform 94.8/5.9	Prosthetic Procedure I	083p 083p 083p Cemented Angulated Meta G UCLA	Fixture Level Impression	084p 084p 084p Replica Pick-up Transfer Post Impression Coping
	Prosthetic Procedure II	086p Solid		O86p 086p 087p 087p Solid Solid Solid Positioning Solid Solid Protection Cap Impression Cap Cylinder Lab Analog Plastic Coping
Platform @4.8/5.9 Cuff 1.8/2.4mm Length 7/8/10/12/14mm Diameter @3.5/4.0/4.5/5.0/6.0	Prosthetic Procedure III	089p Shoulder	Abutment Level	O89p O89p O89p O89p Shoulder Shoulder Positioning Cylinder Lab Analog
Diameter Ø3.5 /4.0 / 4.5 / 5.0 / 6.0	Prosthetic Procedure IV	091p Sonator S Abutment	Abu	092p 092p Impression Coping Sonator Analog
	Prosthetic Procedure V	095p Ball		095p Ball Analog

INNO Internal Impant (Int.)


INT. OCTAGON SYSTEM

Internal Fixture Surface Treatment: SLA-SH®

- > Interchangeable with 1 staged internal fixture.
- > Internal Octa Connection (Taper 8°/ Octa 3.1).
- > No-Mount typ

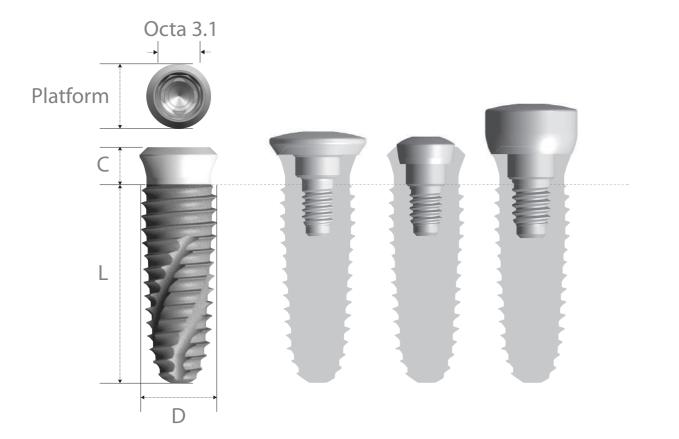

INNO Fixture Code

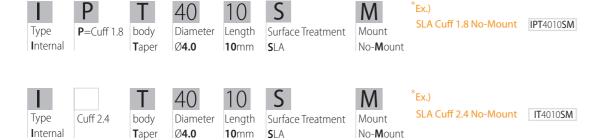

No-Mount Cuff 1.8mm fixture > Packing unit: 1 Fixture + 1 Cover Screw.

IPT6008SM

IPT6010SM

IPT6012SM

10

12

14

* Diameter	62.5						
Length	Ø3.5						
		* Platform : Ø4.8					
7	-	ŢC	* Platform	8	10	12	1
8	IPT3508SM	L		-	-	-	-1
10	IPT 3510 SM		* Fixture ************************************		-	1/4	- 1
12	IPT 3512 SM	₩.		•	**	7	- 1
14	IPT3514SM	D					1
* Diameter Length	Ø4.0						
7	IPT4007SM	* Platform : Ø4.8	7	0	10	12	1
8	IPT4008SM	∓c	7	8	10	12	1
10	IPT4010SM	L L					
12	IPT4012SM			響		1/4	
14	IPT4014SM	 D					-
* Diameter Length	Ø4.5 IPT4507SM	* Platform : Ø4.8	7	0	10	12	
8	IPT4508 SM		7	8	10	12	9
10	IPT 4510 SM	L L	7	A			
12	IPT 4512 SM		•		-	1	
14	IPT4514SM	⊢ D 1				•	1
* Diameter Length	Ø5.0						
7	IPT5007SM	* Platform : Ø5.9	-		10	12	
8	IPT5008SM		7	8	10	12	100
10	IPT5010SM	L	1	Z			
12	IPT5012SM			*	1	1/2	
14	IPT 5014 SM						-
* Diameter Length	Ø6.0						
Lengui		* Platform : Ø5.9					
7	IPT6007SM	TC	7	8	10	12	

No-Mount Cuff 2.4mm fixture > Packing unit: 1 Fixture + 1 Cover Screw.

* Diameter Ø3.5 Length IT3508SM 10 IT3510SM 12 IT3512**SM** 14 IT3514SM






-	
7	IT 4007 SM
8	IT4008SM
10	IT 4010 SM
12	IT 4012 SM
14	IT4014SM

Ø4.0

* Diameter


Length



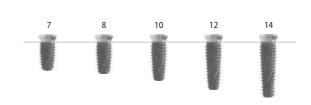

* Diameter Length	Ø4.5
7	IT4507 SM
8	IT4508 SM
10	IT 4510 SM
12	IT4512 SM
14	IT 4514 SM



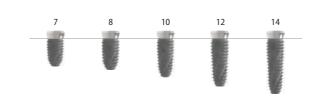





Ø5.0

* Diameter



7	8	10	12	14

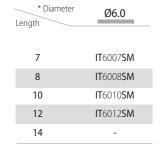

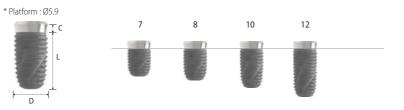

Cover Screw



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Height	Ø5.0	Ø6.0
6.5	ICVR002	ICVW002

- > Packing unit : 1 Cover Screw.
- > To seal the conical interface of fixture.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force : 5~10 N.cm.

Headless Screw



Diameter Height	Ø3.5
6	ICVR001

- > Packing unit: 1 Headless Screw.
- > For narrow mesio-distal distance.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force : 5~10 N.cm.

Healing Abutment




Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
	Ø5.5	Ø6.6
2	IHCR020	IHCW020
3	IHCR030	IHCW030
4.5	IHCR045	IHCW045

- > Packing unit: 1 Abutment.
- > For remodeling gingival contour during soft tissue healing.
- > Select according to gingival height and abutment type.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force : 5~10 N.cm.


080 INNO-INTERNAL IMPLANT INNO-INTERNAL IMPLANT 081

Prosthetic Procedure I

Component Selection Guide for Cemented & UCLA Abutment

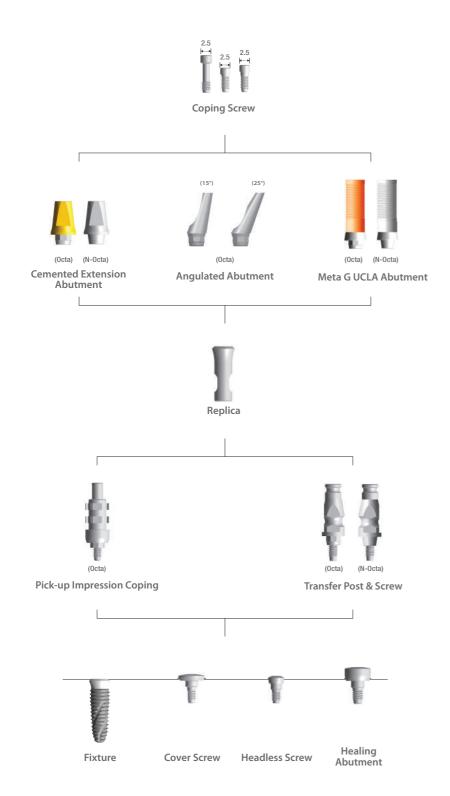

Cemented Extension Abutment

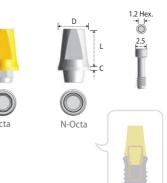

ΙL	it.								
	Туре	Octa							
	Platform [Fixture Dia.]	Ø4.8 [Ø3.5 /	Ø4.0/Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]					
	Diameter	Ø4.8	Ø5.8	Ø5.9	Ø6.9				
	Length Cuff	6	6	6	6				
	0.5	IECR406		IECW506					
	1		IECR416		IECW516				
	2		IECR426		IECW526				
	3		IECR436		IECW536				
	Туре	N-Octa							

Platform [Fixture Dia.]	ire Dia.] 04.8 [03.5 / 04.0 / 04.5]		Ø5.9 [Ø5.0 / Ø6.0]	
Diameter	Ø4.8	Ø5.8	Ø5.9	Ø6.9
Length Cuff	6	6	6	6
0.5	IENR406		IENW506	
1		IENR416		IENW 516
2		IENR426		IENW526
3		IENR436		IENW536

- > Packing unit: 1 Cemented Extension Abutment + 1 Abutment Screw.
- > For Cement Retained or Screw-Cement Retained Prosthesis.
- > Cutting surface for anti-rotation of prosthesis.
- > Connected with the Abutment Screw (ISHR110).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 35 N.cm.
- > Fixture level impression.

Angulated Abutment








Туре	Octa		
Platform [Fixture Dia.]	Ø4.8 & Ø5.9 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø6.0]		
Diameter(Angle) Length	3.8 (15°)	3.8 (25°)	
8	IAAR158A	IAAR258A	

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Cement Retained or Screw-Cement Retained Prosthesis.
- > Solution for anterior esthetic zone.
- > Connected with the Abutment Screw (ISHR100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 35 N.cm.
- > Fixture level impression.

Meta G UCLA Abutment

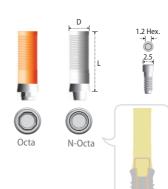

Туре	Octa		N-Octa	
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	Ø4.8 [Ø3.5/Ø4.0/Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Length	Ø5	Ø6	Ø5	Ø6
12	IGOR400N	IGOW500N	IGNR400N	IGNW500N

- > Packing unit: 1 Meta G UCLA Abutment + 1 Abutment Screw.
- > For Screw-Cement or Screw Retained Prosthesis.
- > Modification to angulated abutment, customized abutment and telescopic abutment.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the Abutment Screw (ISHR120).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 35 N.cm.
- > Fixture level impression.

082 INNO-INTERNAL IMPLANT INNO-INTERNAL IMPLANT 083

Diameter Height	Ø2.5	Ø2.5	Ø2.5
6.3		ISHR100	
7.8			ISHR120
9.2	ISHR110		

- > Packing unit: 1 Abutment Screw.
- > ISHR110 : Cemented Abutment.
- > ISHR100 : Angulated Abutment.
- > ISHR120 : Meta G Abutment.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.


Replica



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Height —	Ø4.8	Ø5.9
12	IROR001	IROW001

- > Packing unit: 1 Replica.
- > Mimicking of conical interface of fixture.
- > Analog of fixture for working cast.


Pick-up Impression Coping

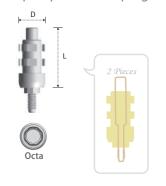

Туре	Octa		
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	
Diameter Length	Ø5.5	Ø6.6	
13.7	IIOR001	IIOW001	

- > Packing unit: 1 Pick-up Impression Coping + 1 Guide Pin (IIOR001S).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Transfer Post

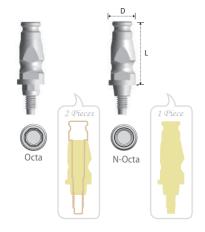


Туре	Octa		N-Octa	
,,	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5] Ø5.9 [Ø5.0 / Ø6.0]		0] Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5] Ø5.9 [Ø5.0 / Ø6	
Diameter Length	Ø4.85	Ø5.95	Ø4.85	Ø5.95
11.6	ITOR400	ITOW500	ITNR400	ITNW500

- > Packing unit: Octa 1 Transfer Post + 1 Guide Pin (Regular: ITOR400S / Wide: ITOW500S) N-Octa - 1 Transfer Post (solid screw).
- > For closed tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Prosthetic Procedure II

Component Selection Guide for Solid Abutment

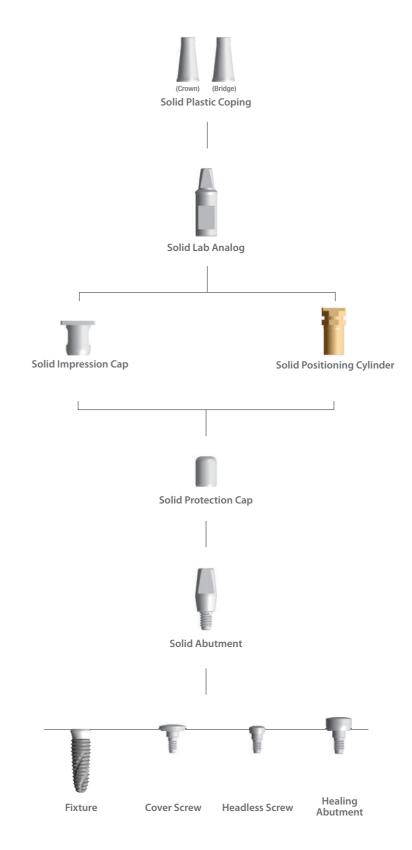

Solid Abutment



Platform [Fixture Dia.]	Ø4.8 & Ø5.9 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø6.0]			
Diameter	Ø3.5			
Length	3	4	5.5	7
	IASR030	IASR040	IASR055	IASR070

- > Packing unit: 1 Solid Abutment + 1 Abutment Cap.
- > For Cement Retained Prosthesis.
- > Cutting surface for anti-rotation of prosthesis.
- > Integrated with screw and abutment.
- > Tightened with the Shoulder Driver.
- > Tightening torque force : 35 N.cm. > Abutment level impression :

Impression cap in platform Ø4.1 fixture and direct impression in platform Ø5.8 fixture.



Solid Protection Cap



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]
Diameter Height	Ø5.4
5.2	IASR130
6.2	IASR140
7.7	IASR 155
9.2	IASR170

- > Packing unit: 1 Solid Protection Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Alternative usage for sub-structure of temporary prosthesis.

Solid Impression Cap



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	
Diameter Height	8	
8	IICR001	

- > Packing unit: 1 Impression Cap.
- > Connected with the Positioning Cylinder.
- > Confirm locking with abutment by rotation of clockwise and anti-clockwise direction.

Solid Positioning Cylinder



Platform [Fixture Di	uJ Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]
Diamete Height	Ø5.7
10.2	IPCR001

- > Packing unit: 1 Positioning Cylinder.
- > Inner cutting surface for anti-rotation on abutment.
- > Insert into the Impression Cap.

G 11 1			
Solid	Lah	Δ nal	α
JUIIU	LUD	/ \IIIui	109



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]			
Diameter Length	3	4	5.5	7
4.8	ILSR030	ILSR040	ILSR 055	ILSR 070

- > Packing unit : 1 Lab Analog.
- > Replacement of abutment shape in working cast.
- > Choose by abutment length.

Solid Plastic Coping










Туре	Crown	Bridge
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]
Diameter Height	Ø5	Ø5
10	IPCC001	IPCB001

- > Packing unit: 1 Plastic Coping.
- > Connect with the Lab Analog.
- > Burn out and casting for metal framework.

086 INNO-INTERNAL IMPLANT

Prosthetic Procedure III

Component Selection Guide for Shoulder Abutment

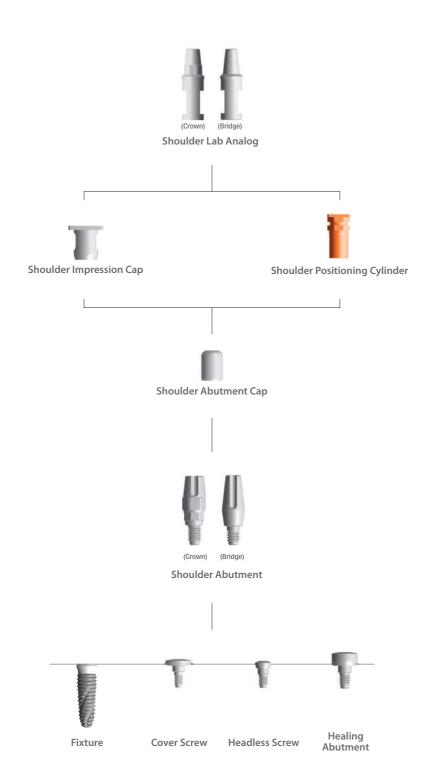

Shoulder Abutment















	Туре	Cro	wn	Brid	ge
_	* Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
)	Diameter Length	Ø3.5	Ø4.5	Ø3.5	Ø4.5
	4	ISAC404	ISAC504	ISAB404	ISAB504
	5.5	ISAC405	ISAC505	ISAB 405	ISAB 505
	7	ISAC407	ISAC507	ISAB407	ISAB 507

- > Packing unit: 1 Shoulder Abutment + 1 Abutment Cap.
- > For Cement Retained Prosthesis.
- $\,>\,$ Dual anti-rotation grip with single crown for prevention of screw loosening.
- > Integrated with the Screw and Abutment.
- > Tightened with the Shoulder Driver.
- > Tightening torque force : 35 N.cm.

> Abutment level i	mpression.	
	Shoulder Ø4.5	KRR19L



Shoulder Abutment Cap



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Height	Ø5.4	Ø5.4
6.2	IASR140	IASW140
7.7	IASR155	IASW 155
9.2	IASR170	IASW 170

- > Packing unit: 1 Shoulder Abutment Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Alternative usage for sub-structure of temporary prosthesis.

Shoulder Impression Cap



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Height	8	9
8	IICR001	IICW001

- > Packing unit: 1 Impression Cap.
- > Connected with the Positioning Cylinder.
- > Confirm locking with abutment by rotation of clockwise and anti-clockwise direction.

Shoulder Positioning Cylinder



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Height	5.7	6.8
10.7	SAPR001	SAPW 001

- > Packing unit: 1 Positioning Cylinder.
- > Inner cutting surface for anti-rotation on abutment.
- > Insert into the Impression Cap.

Shoulder Lab Analog





D	
<u></u>	
98	
ш.	

Bridge


Туре	Cro	wn	Bridg	ge
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Length	Ø4.8	Ø5.9	Ø4.8	Ø5.9
4	SLCR040	SLCW 040	SLBR040	SLBW 040
5.5	SLCR055	SLCW 055	SLBR055	SLBW 055
7	SLCR070	SLCW 070	SLBR070	SLBW 070

- > Packing unit: 1 Lab Analog.
- > Replacement of abutment shape in working cast.
- > Choose according to width and length of abutment.


088 INNO-INTERNAL IMPLANT INNO-INTERNAL IMPLANT 089

Prosthetic Procedure IV

Component Selection Guide for Sonator Abutment

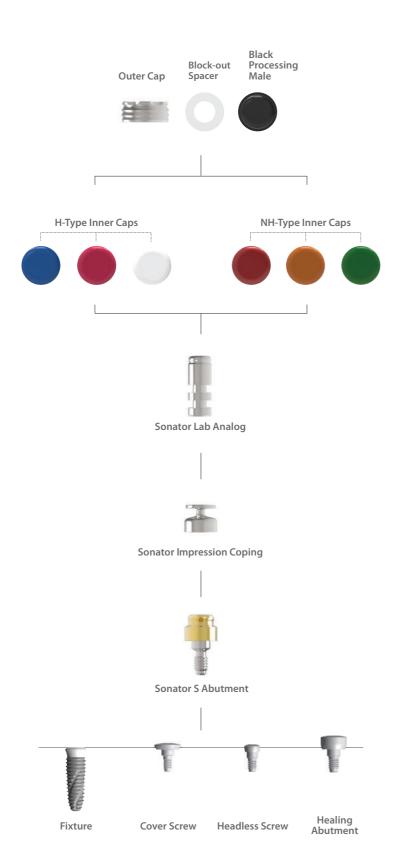

Sonator S Abutment

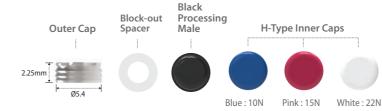

Ratchet Driver





- + Black Processing Male.
- > For Implant Supported Over-denture Prosthesis.
- > Stable with low vertical height.
- > 6 kinds of the Inner Caps give various holding force (Both, H and NH-Type Inner Caps are used for Sonator S Abutment).
- > Path compensation up to 20° based on 2 implants.
- > Carrier: Used for delivery of the abutment.
- > Tightened with the Ratchet Driver and Torque wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.

Outer Cap





- > Packing unit: 2 Outer Caps and 2 Black Processing Males.
- > Black Processing Male: Inserted and Removed with the I&R Driver.

H-Type Inner Cap

Block-out



- > Packing unit: 3 Block-out Spacers + 3 Inner Caps (1 Blue, 1 Pink and 1 White).
- > Path compensation up to 20° based on 2 implants.
- > Mainly used for the Sonator S Abutment.
- > Inner Caps : Inserted and Removed with the I&R Driver.



090 INNO-INTERNAL IMPLANT

INNO-INTERNAL IMPLANT 091

NH-Type Inner Cap

Block-out Spacer Code SONIC02

> Packing unit: 3 Block-out Spacers + 3 Inner Caps (1 Red, 1 Orange and 1 Green).

- > Path compensation up to 40° based on 2 implants.
- > Mainly used for the Sonator A Abutment.
- > Inner Caps: Inserted and Removed with the I&R Driver.

Red: Orange: Green: Retention Force About 10N About 15N About 22N

Inner Cap

Sonator Impression Coping



Diameter Length	Ø4.8
3	SONIP04

- > Packing unit: 4 Impression Copings and 4 Black Processing Males.
- > Abutment level pick-up impression.
- > Connected over the Sonator Abutment after placing the Block-out Spacer.
- > For close tray impression.

Sonator Lab Analog



Diameter Length	Ø4
1.4	SONLA04

- > Packing unit: 4 Lab Analogs.
- > Replacement of abutment shape in working cast.

Sonator S Ratchet Driver




Height 95.4 SONIR002 H For Removal

> Used to insert and remove the Inner Caps and Block Processing Male.

092 INNO-INTERNAL IMPLANT

Prosthetic Procedure V

Component Selection Guide for Ball Abutment

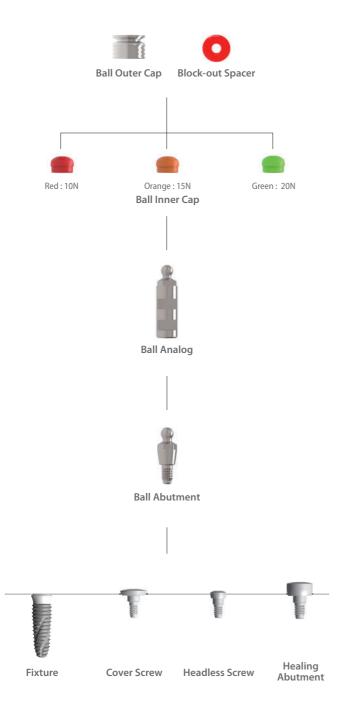

Ball Abutment

Outer Cap Block-out Spacer Length Diameter 4 BAT404R Packing unit: 1 Ball Abutment + 3 Inner Caps (1 per each color) + 1 Block-out Spacer + 1 Outer Cap. For Implant Supported Over-denture Prosthesis. Tightened with the Ball Driver and Torque Wrench.

> Tightening torque force : 30 N.cm.



> Direct impression.

Ball Outer Cap



Diameter Height	Ø3.4
2.4	BATC003C

> Packing unit : 2 Outer Caps.

Ball Inner Cap



- > Packing unit: 2 Block-out Spacers + 6 Inner Caps (2 per each color).
- > Retention force : Red 10N, Orange 15N & Green 20N.


Ball Analog



Diameter Length	Ø4.0
4	SBAL400

BATC003I

- > Packing unit : 4 Lab Analogs.
- > Replacement of abutment shape in working cast.

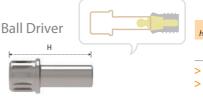

Type Height	Ratchet	
19	KRB19L	
> Packing unit :	1 Ball Driver.	*Extra Product
> Used with the	Torque Wrench to tighten and untighten the Ball Abutment.	



- > Packing unit: 1 I&R Driver.
- > Used to insert and remove the Inner Caps into and out of the Outer Cap.

INNO EXTERNAL IMPLANT (Ext.)

System Flow




INNO External Implant (Ext.)

EXT. HEXAGON SYSTEM

External Fixture
Surface Treatment: **SLA-SH**®

- > Interchangeable with external hexagonal fixture.
- > External hex connection (Hex 2.7 / 3.4).

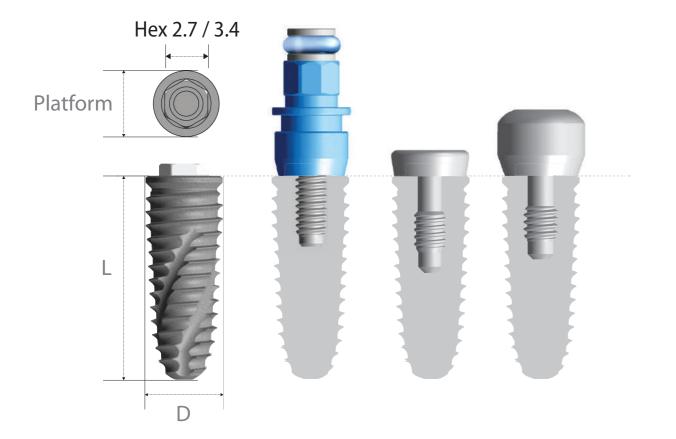

SLA Pre-Mount ET4010S

INNO Fixture Code



Pre-Mount

> Packing unit :1 Fixture + 1 Mount + 1 Mount Screw.

* Diameter	Ø3.5	* Hex 2.7						
Length		* Hex 2.7 * Platform : Ø4.1						
7	_		_		8	10	12	14
8	ET 3508 S							
			Hex Platform	1				
10	ET3510S						7/2	
12	ET3512S	W.L	_		•	W.	1	
14	ET3514S	D						W.
* Diameter	Ø4.0							
Length	2	* Hex 2.7 * Platform : Ø4.1						
_	FT 10076		7	,	8	10	12	14
7	ET4007S		î	i		6		6
8	ET4008S						_	_
10	ET4010S	L			1			
12	ET4012S				-	¥	1/4	1/2
14	ET4014S	F D						•
* Diameter	Ø4.5	* Uov 2.7						
Length	<i>V</i> 4.3	* Hex 2.7 * Platform : Ø4.1						
			7	7	8	10	12	14
7	ET4507S		Ĭ	Ĭ.				
8	ET4508S							
10	ET4510S	L			1			A
12	ET 4512 S	*					1/4	
14	ET 4514 S	D						
* Diameter	ar o							
Length	Ø5.0	* Hex 3.4 * Platform : Ø5.1						
7	ET5007 S		7	7	8	10	12	14
8	ET5008S	<u>. </u>	- 4				•	
10	ET5000S							_
12	ET5010 S		-	4	The second	E		1
14	ET5012S						1	1/2
	2130113	D						•
* Diameter	Ø6.0	* Hex 3.4 * Platform : Ø5.1						
Length	\$0.0			7	8	10	12	
7	ET6007 S			2	<u>-</u>		12	
8	ET60073		1					
10	ET6010S	L L		Sitte				
12	ET6012S			19	12	1	1/2	
14	-	⊢ D · T						
דו		-						

Fixture Mount



Нех	Hex2.7	Hex3.4
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter Length	Ø4.9	Ø5.5
7.2	MER001	MEW002

- > Packing unit: 1 Mount + 1 Mount Screw.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force : 5~10 N.cm.

Cover Screw



Нех	Hex2.7	Hex3.4
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter Height	Ø4.3	Ø5.4
5.8	VNR 001	VNW 001

- > Packing unit: 1 Cover Screw.
- > To seal the conical interface of fixture.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force : 5~10 N.cm.

Healing Abutment




Нех	Hex2.7	Hex3.4
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
	Ø5.0	Ø6.0
2.8	HNR 502	HNW 602
3.8	HNR 503	HNW603
4.8	HNR504	HNW 604
5.8	HNR 505	HNW605
6.8	HNR506	HNW606
7.8	HNR507	HNW607


- > Packing unit: 1 Abutment.
- > For remodeling gingival contour during soft tissue healing.
- > Select according to gingival height and abutment type.
- > Tightened with the 1.2 Hex Driver.
- > Tightening torque force : 5~10 N.cm.

Prosthetic Procedure I

Component Selection Guide for Cemented & UCLA Abutment

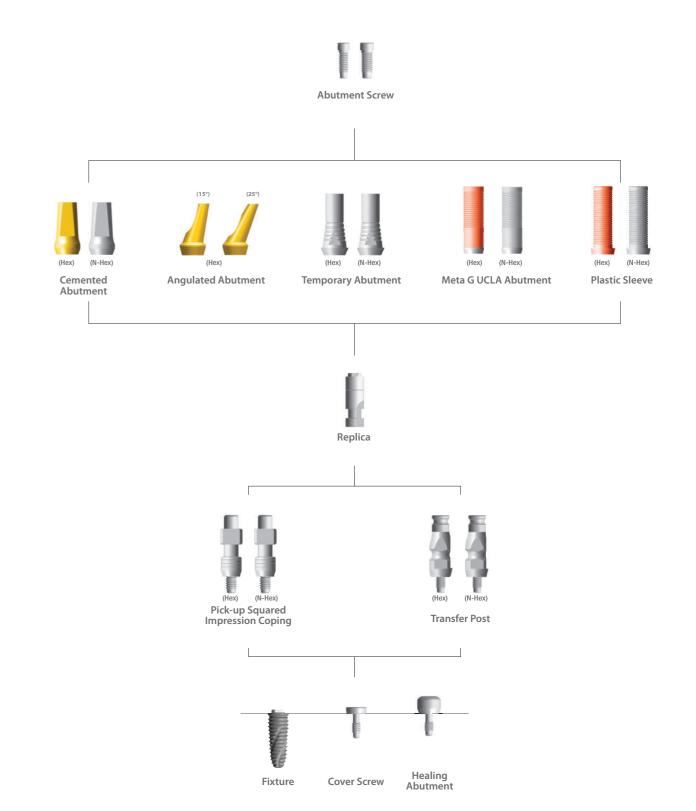

Cemented Abutment

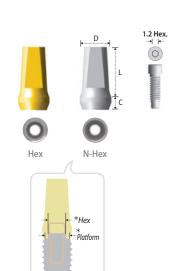


Туре	Hex				
*Hex	Hex	x2.7	Hex3.4		
* Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]		Ø5.1 [Ø5.0 / Ø6.0]		
Diameter	Ø5.0		Ø6.0		
Length Cuff	6	8	6	8	
1	CHR 516	CHR 518	CHW 616	CHW 618	
2	CHR 526	CHR 528	CHW626	CHW628	
3	CHR 536	CHR 538	CHW636	CHW638	
4	CHR 546	CHR548	CHW646	CHW648	

Туре	N-Hex			
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]		Ø5.1 [Ø5.0 / Ø6.0]	
Diameter	Ø5.0		Ø6.0	
Length Cuff	6	8	6	8
1	CNR 516	CNR 518	CNW 616	CNW 618
2	CNR 526	CNR528	CNW626	CNW628
3	CNR 536	CNR 538	CNW636	CNW638
4	CNR 546	CNR548	CNW646	CNW648

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Cement Retained and Screw-Cement Retained Prosthesis. > Tightening torque force: 35 N.cm
- > Cutting surface for anti-rotation of prosthesis.
- > Connected with the Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Fixture level impression.


Angulated Abutment

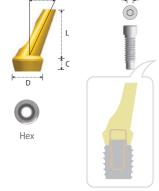

Type[Hex]	Hex2.7	Hex3.4	Hex2.7	Hex3.4
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter (Angle)	Ø5 (15°)	Ø6 (15°)	Ø5 (25°)	Ø6 (25°)
Length Cuff	8	8	8	8
2	AAR 152	AAW 152	AAR 252	AAW 252
3	AAR 153	AAW 153	AAR 253	AAW 253
4	AAR 154	AAW 154	AAR 254	AAW 254

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Prosthesis.
- > Solution for anterior esthetic zone.
- > Conneced with the Abutment Screw.
- ${\color{red}>}$ Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 35 N.cm.
- > Fixture level impression.


Temporary Abutment

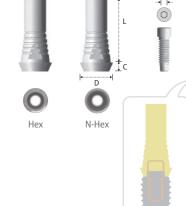

Type[Hex]	Hex[2.7]	Hex[3.4]	N-Hex	N-Hex
atform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter	Ø5.4	Ø5.95	Ø5.4	Ø5.95
Length Cuff	12	12	12	12
1.5	THR 001	THW 001	TNR 001	TNW 001

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Screw-Cement Retained Prosthesis.
- > For provisional restoration.
- > Connected with the Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 20 N.cm.


Meta G UCLA Abutment

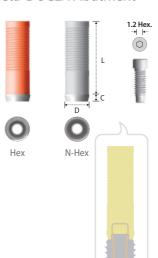

Type[Hex]	Hex[2.7]	Hex[3.4]	N-Hex	N-Hex
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Length Cuff	13	13	13	13
1.2	GHR001N	GHW 001 N	GNR 001 N	GNW 001 N

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > For Screw-Cement or Screw Retained Prosthesis.
- > Modification to angulated abutment, customized abutment and telescopic abutment.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 35 N.cm.
- > Fixture level impression.

Plastic Sleeve

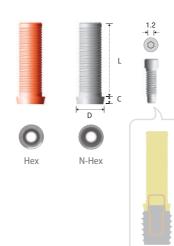

Type[Hex]	Hex[2.7]	Hex[3.4]	N-Hex	N-Hex
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Length Cuff	11.8	11.8	11.8	11.8
1.2	PHR001	PHW 001	PNR 001	PNW 001

- > Packing unit: 1 Abutment + 1 Abutment Screw.
- > Same purpose of use as Meta G UCLA Abutment but low accuracy of connection.
- > PMMA material.
- > Connected with the Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: Finger light force during wax Pattern fabrication, 35 N.cm after casting.



Нех	Hex2.7	Hex3.4
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter Height	Ø2.5	Ø3.0
8	SHR 100	SHW 100

- > Packing unit: 1 Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.


Replica



Type[Hex]	Hex[2.7]	Hex[3.4]
Platform [Fixture Dia.]		Ø5.1 [Ø5.0 / Ø6.0]
Diameter Height	Ø4.1	Ø5.1
12	LHR001	LHW 001

- > Packing unit : 1 Replica.
- Mimicking of conical interface of fixture.Analog of fixture for working cast.


Pick-up Squared Impression Coping

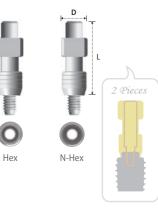

Type[Hex]	Hex[2.7]	Hex[3.4]	N-Hex	N-Hex
Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter Length	Ø5	Ø5.8	Ø5	Ø5.8
17	IHR500	IHW600	INR500	INW600

- > Packing unit: 1 Pick-up Impression Coping + 1 Guide Pin (Regular: UHR115 / Wide: UHW115).
- > For Open Tray Impression.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Transfer Post

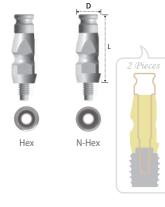


Type[Hex]	Hex[2.7]	Hex[3.4]	N-Hex	N-Hex
atform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.1 [Ø5.0 / Ø6.0]
Diameter ength	Ø4.8	Ø5.8	Ø4.8	Ø5.8
13.1	IHR 510	IHW 610	INR 510	INW 610


- > Packing unit: 1 Pick-up Impression Coping + 1 Guide Pin (Regular: IHR510S, IHR610S / Wide: IHW610S).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Prosthetic Procedure II

Component Selection Guide for Shoulder Abutment

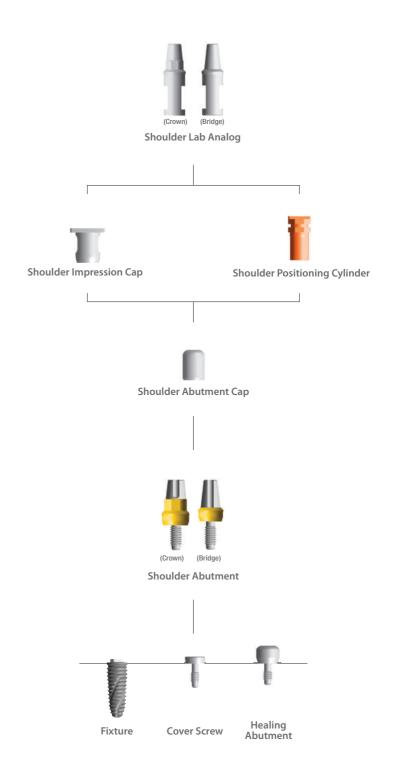

Shoulder Abutment

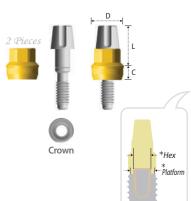

Туре	Crown					
*Hex	Hex2.7				Hex3.4	
* Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]			Ø	5.1 [Ø5.0 / Ø6.	0]
Diameter	Ø4.8				Ø5.9	
Length Cuff	4	5.5	7	4	5.5	7
1	SAC 414	SAC 415	SAC 417	SAC 514	SAC 515	SAC 517
2	SAC424	SAC 425	SAC 427	SAC 524	SAC 525	SAC 527
3	SAC 434	SAC 435	SAC 437	SAC 534	SAC 535	SAC 537
4	SAC444	SAC 445	SAC447	SAC 544	SAC 545	SAC 547

SAB517

SAB527

SAB537

SAB547

	Туре	Bridge					
D	Нех		Hex2.7			Hex3.4	
D	Platform [Fixture Dia.]	Ø4.1 [Ø3.5 / Ø4.0 / Ø4.5]			Ø	5.1 [Ø5.0 / Ø6.	0]
L	Diameter		Ø4.8			Ø5.9	
<u></u> c	Length Cuff	4	5.5	7	4	5.5	
	1	SAB 414	SAB 415	SAB 417	SAB 514	SAB 515	
	2	SAB424	SAB 425	SAB427	SAB 524	SAB 525	
	3	SAB 434	SAB 435	SAB 437	SAB 534	SAB 535	
	4	SAB444	SAB 445	SAB 447	SAB 544	SAB 545	

- > Packing unit: 1 Abutment.
- > For Cement Retained Prosthesis.
- > Dual anti-rotation grip for prevention of screw loosening.
- > Integrated with screw and abutment.
- > Tightened with the Shoulder Driver.
- > Tightening torque force : 35 N.cm.
- > Abutment level impression: Impression cap in platform Ø4.1 fixture and direct impression in platform Ø5.8 fixture.



Shoulder Abutment Cap

Bridge



Нех	Hex2.7	Hex3.4
Diameter Height	Ø5.4	Ø6.5
6.2	IASR140	IASW140
7.7	IASR155	IASW 155
9.2	IASR170	IASW 170

- > Packing unit: 1 Shoulder Abutment Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Alternative usage for sub-structure of temporary prosthesis.

Shoulder Impression Cap



Нех	Hex2.7	Hex3.4
Diameter Height	8	9
8	IICR001	IICW001

- > Packing unit: 1 Impression Cap.
- > Connected with the Positioning Cylinder.
- > Confirm locking with abutment by rotation of clockwise and anti-clockwise direction.

Shoulder Positioning Cylinder



Нех	Hex2.7	Hex3.4
Diameter Height	Ø4.4	Ø5.5
10.7	SAPR001	SAPW 001

- > Packing unit: 1 Cyilnder.
- > Inner cutting surface for anti-rotation on abutment.
- > Insert into the Impression Cap.

Shoulder Lab Analog







ŀ	D	L
1	f	•
ı		ij.


Туре	Crown		Brio	lge
Diameter Length	Ø4.8	Ø5.9	Ø4.8	Ø5.9
4	SLCR040	SLCW 040	SLBR040	SLBW040
5.5	SLCR055	SLCW 055	SLBR055	SLBW 055
7	SLCR070	SLCW 070	SLBR070	SLBW 070

- > Packing unit: 1 Analog.
- > Replacement of abutment shape in working cast.
- > Choose by abutment length.

106 INNO-EXTERNAL IMPLANT INNO-EXTERNAL IMPLANT 107

Prosthetic Procedure III

Component Selection Guide for Ball Abutment

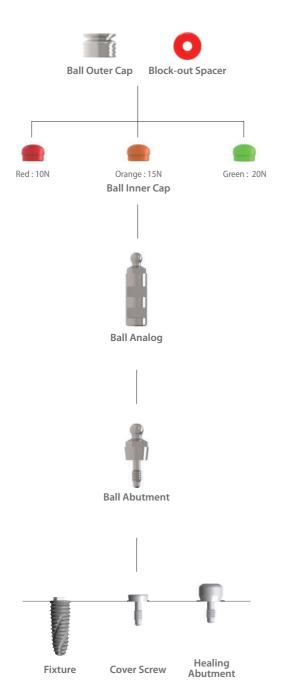

Ball Abutment

Block-out Spacer Outer Cap Ø3.4 Ball Inner Cap

Diameter	Ø5.0	Ø6.0	
Length Cuff	4	4	
1	EBAT411R	EBAT511R	
2	EBAT412R	EBAT512R	
3	EBAT413R	EBAT513R	
4	EBAT414R	EBAT514R	
> Packing unit : 1 Ball Abutment + 3 Inner Caps (1 per each color) + 1 Block-out Spacer + 1 Outer Cap.			

- > For Implant Supported Over-denture Prosthesis.
 - > Tightened with the Ball Driver and Torque Wrench.
 - > Tightening torque force : 30 N.cm. > Direct impression.

Ball Abutment

Ball Outer Cap



Diameter Height	Ø3.4
2.4	BATC003C

BATC003I

> Packing unit: 2 Outer Caps.

Ball Inner Cap



> Packing unit · 2 Block-out Spacers + 6	5 Inner Caps (2 per each color)


> Retention force: Red 10N, Orange 15N & Green 20N.

Ball Analog



Diameter Length	Ø4.0
4	SBAL400

- > Packing unit: 4 Lab Analogs.
- > Replacement of abutment shape in working cast.

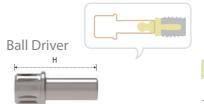


Type Height	Ratchet
19	KRB19L

> Packing unit: 1 Ball Driver.

*Extra Product

> Used with the Torque Wrench to tighten and untighten the Ball Abutment.

I&R Driver

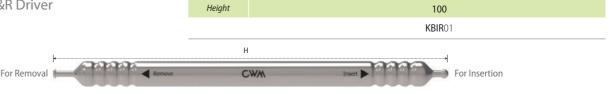


- > Packing unit: 1 I&R Driver.
- > Used to insert and remove the Inner Caps into and out of the Outer Cap.

108 INNO-EXTERNAL IMPLANT

INNO SUB. FULL SURGICAL KIT [KCA010F]

- > For INNO Submerged Implant System (Sub.).
- > All components are for Sub. / Int. / Ext. except for the Fixture Drivers and the Depth Gauge used for Sub. exclusively.

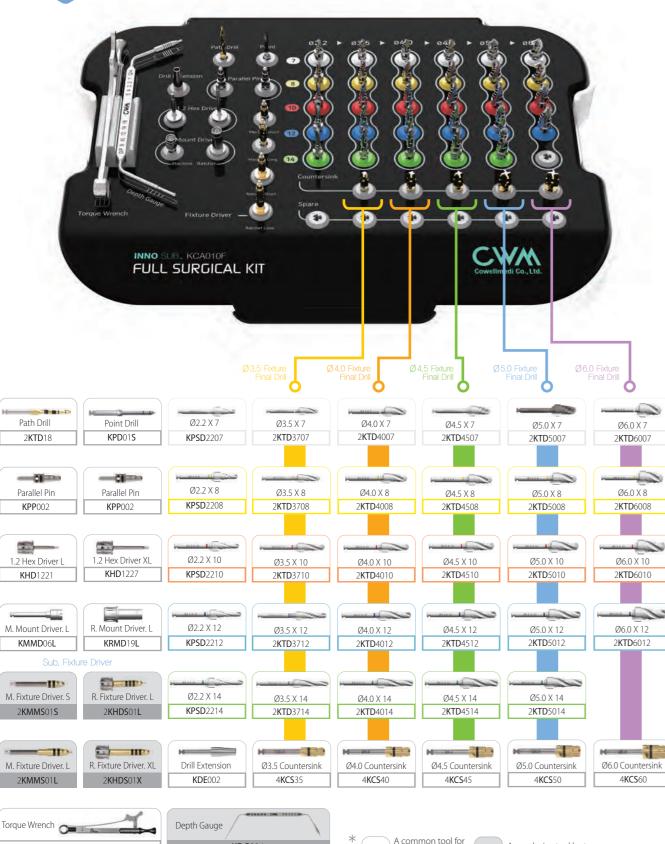


An exclusive tool by type

INNO INT. FULL SURGICAL KIT [KCA010FI]




- > For the selection and use guide of the components for the Sub. Int. & Ext. Full Surgical Kit, refer to pages 113 to 119.
- > For the INNO Internal Implant System (Int.).
- > All components are for Sub. / Int. / Ext. except for the Fixture Drivers used for Int. exclusively.

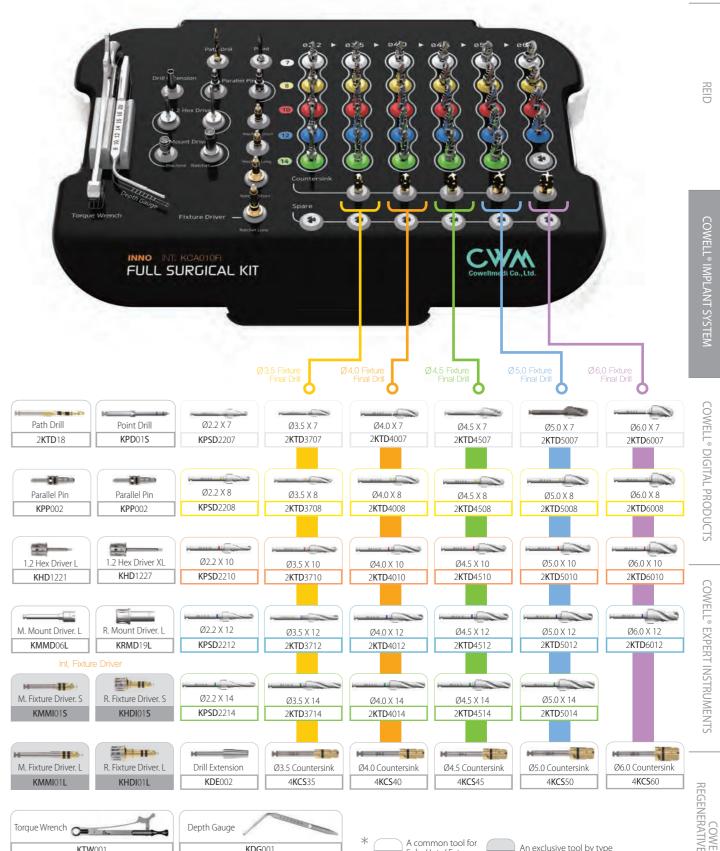


INNO EXT. FULL SURGICAL KIT [KCA010FE]

- > For the selection and use guide of the components for the Sub. Int. & Ext. Full Surgical Kit, refer to pages 113 to 119.
- > For the INNO External Implant System (Ext.).
- > All components are for Sub. / Int. / Ext. except for the Fixture Drivers and the Multi Countersink used for Ext. exclusively.

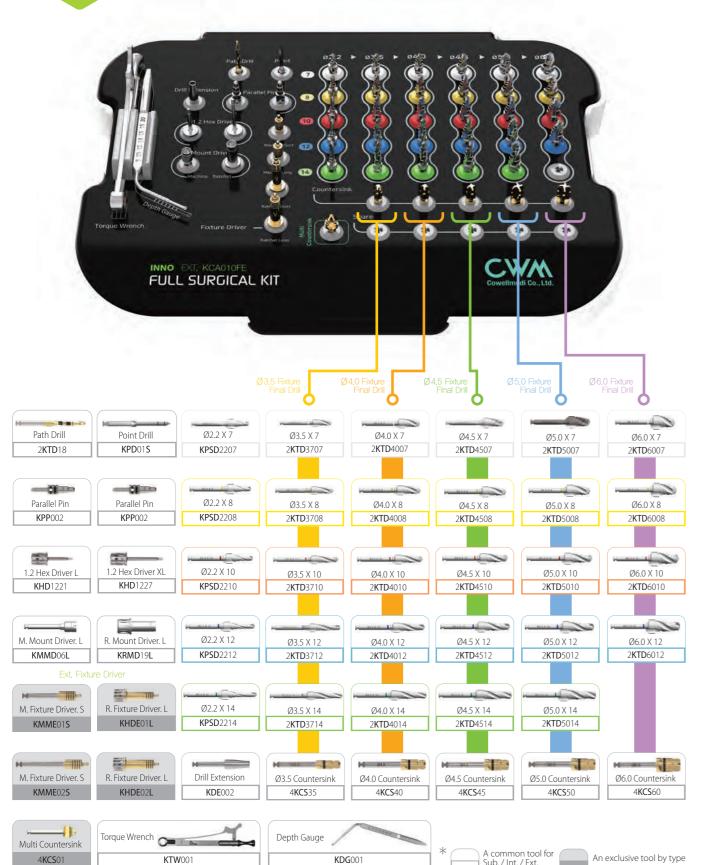

SUBHEXAGON SYSTEM Drill / Surgical Tool


Point Drill

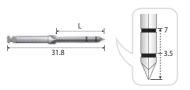

- > Primarily used for marking the implant recipient site and to determine spacing.
- > The point drill has a unique pointed tip, making this an excellent drill for starting the osteotomy through the hard cortical plate.

Length	15
	KPD01S


Path Drill

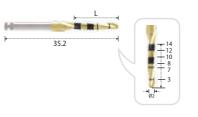

- > Used for the case that path modification is required.
- > Excellent ablation force that does not slip in slanted bone.
- > Easy to drill even in extraction socket without slipping.

	Length	15
2 KTD 18		2 KTD 18

Initial Drill

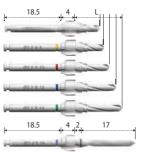

> Initial stepped Drill - Ø2.2 / Ø2.8 / Ø3.3 mm stepped osteotomy at Ø1.8 Drill site.


/]	• /	Length band
()		
\(

☐ 7mm Fixture 8mm Fixture 10mm Fixture 12mm Fixture 14mm Fixture

■ 16&18mm Fixture


Length 13 17&19 KPSD2208 *KPSD2218 KPSD2207 **KPSD**2210 **KPSD**2212 KPSD2214

*Extra product

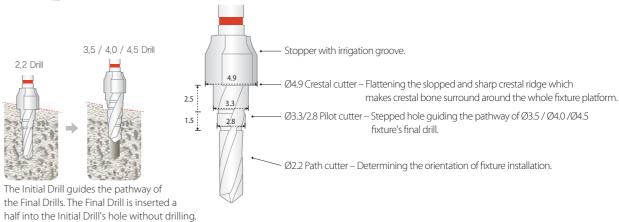

112 SURGICAL KIT SURGICAL KIT 113

Final Drill

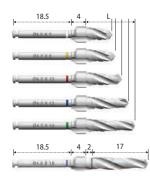

> Ø3.5 / 4.0 / 4.5 / 5.0 / 6.0 fixture's Final Drill. > 7 / 8 / 10 / 12 / 14 / 16 / 18mm fixture's Final Drill.

Fixture Dia. Length	Ø3.5	Ø4.0	Ø4.5	Ø5.0	Ø6.0
8	2 KTD 3707	2 KTD 4007	2 KTD 4507	2 KTD 5007	2 KTD 6007
9	2 KTD 3708	2 KTD 4008	2 KTD 4508	2 KTD 5008	2 KTD 6008
11	2 KTD 3710	2 KTD 4010	2 KTD 4510	2 KTD 5010	2 KTD 6010
13	2 KTD 3712	2 KTD 4012	2 KTD 4512	2 KTD 5012	2 KTD 6012
15	2 KTD 3714	2 KTD 4014	2 KTD 4514	2 KTD 5014	
17&19	*2 KTD 3718	*2 KTD 4018	*2 KTD 4518		

*Extra product

Tap Drill



> Used in preparation of osteotomy in dense bone only.

Fixture Dia.	Ø3.5	Ø4.0	Ø4.5	Ø5.0	Ø6.0
	* 3KMTD35A	*3KMTD40A	*3KMTD45A	*3KMTD50A	*3KMTD60A
					*Extra product

Countersink



- > Bone quality 1 : high compressive placement of fixtures induces the failure of osseointegration
- > Countersink decreases torque force (Ø4.0 Fixture: 80 N.cm -> 45 N.cm / Ø5.0 Fixture: 150 N.cm -> 45 N.cm).


Ø6.0 Fixture line. Ø5.0 Fixture line.

°Ø4.5 Fixture line.


`Ø3.5 / Ø4.0 Fixture line.

> Preventing compressive necrosis of dense cortical bone.

Fixture Dia.	Ø3.5	Ø4.0	Ø4.5	Ø5.0	Ø6.0
Diameter	Ø3.7	Ø4.2	Ø4.6	Ø5.1	Ø6.0
	4 KCS 35	4 KCS 40	4 KCS 45	4 KCS 50	4 KCS 60

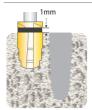

Lower margin of depth marking line indicating fixture platform level.

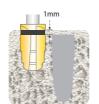

Upper margin of depth marking line indicating 1 mm over fixture platform level.

Parallel Pin




- > Insert the Parallel Pin after Ø 2.2 or 3.5 Drill to check the osteotomy path.
- > In order to prevent losing Parallel Pin in patient's mouth, be sure to tie floss through the hole in the Parallel Pin.


Height	21
	KPP 002





Multi Countersink

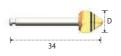

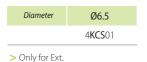





Drill Extension



- > Used for lengthening the Drill when using a Hand-piece.
- > Do not go over recommended torque when using the Drill Extension.

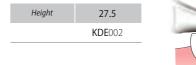

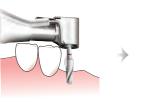



Triangle mark indicating the cutting surface of the drill shaft.

114 SURGICAL KIT

SURGICAL KIT 115

COWELL® EXPERT INSTRUMENTS

Mount Driver



> Used to install Pre-Mount type fixtures.

Type Heiaht	Machine
20.5(Short)	* KMMD06S
26.3(Long)	KMMD06L
32.3(X-Long)	*KMMD12X

*Extra product



Type Height	Ratchet
12(Short)	* KRMD12S
19(Long)	KRMD19L

*Extra product

Hex Driver



> The Machine Drivers are used with contra angle, while the Ratchet Drivers are used with the Torque Wrench.

Туре	Machine		
Height Hex	Hex 0.9	Hex 1.2	
22(Short)	* KMD09S	* KMD12S	
28(Long)	* KMD09L	* KMD12L	

*Extra product



Туре	Ratchet		
Height Hex	Hex 0.9	Hex 1.2	
12(X-Short)	-	* KHD 1212	
17(Short)	* KHD 0915	* KHD 1215	
23(Long)	* KHD 0921	KHD 1221	
29(X-Long)	* KHD0927	KHD1227	

*Extra product



Fixture Driver







Туре	Machine				
System Length	Sub.	Int.	Ext.(Hex 2.7)	Ext.(Hex 3.4)	
28.1 / 26.3 / 26.4 (Short)	2 KMMS 01 S	KMMI01S	KMME01S	KMME02S	
33.3 / 30.5 / 31.4 (Long)	2KMMS01L	KMMI01L	*KMME01L		
40.3 / 35.5 / 36.4 (X-Long)	*2KMM\$01X	*KMMI01X	*KMME01X		

*Extra product

Sub.	
Int.	E III a
Ext.	

Туре		Ratch	et	
System Length	Sub.	Int.	Ext.(Hex 2.7)	Ext.(Hex 3.4)
20.7 / 19.5 / 19.9 (Short)	* 2KHDS01S	KHDI01S	* KHDE01S	
25.7 / 24.5 / 24.9 (Long)	2 KHDS 01 L	KHDI01L	KHDE01L	KHDE02L
30.7 / 29.5 / 29.9 (X-Long)	2 KHDS 01 X	* KHDI01X	*KHDE01X	

*Extra product



> Used to install No-Mount type fixtures.



Torque Wrench

- > Torque control in implant placement and abutment connection.
- > Torque force 10, 25, 30 & 35 N.cm are able to be controlled by pulling the elastic bar.
- > Maximal torque force 120 N.cm with pulling the rigid main bar.







Depth Gauge

- > Measuring the drill depth with scaled rod.
- > Measuring the 5mm space between adjacent fixtures with flat end of the other side.



Code	KDG 001





> One side of the Depth Gauge measures the drilling depth and the other side measures the gingival height from the top of the implant.

Code	KDG 004	※ Exclusive for Sub.
------	----------------	----------------------


Drilling Sequence E.g. 12mm Fixture

> Minimal drill frequency with the Point Drill, Initial Drill and Final Drills (Ø3.5, Ø4.0 and Ø4.5 Fixtures).






Actual length of Drill: Fixture + 1mm

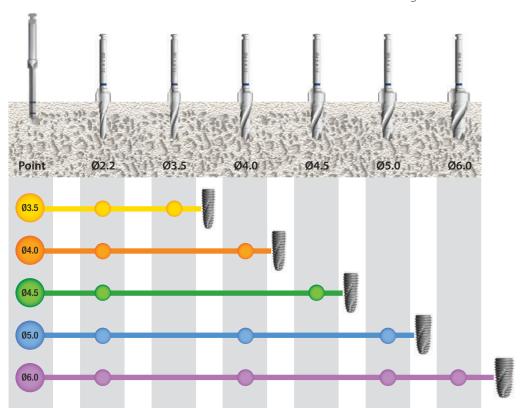


- > Ø5.0 fixture: a series of the Point Drill, Initial Drill, Ø4.0 Final Drill and Ø5.0 Final Drill.
- > Ø6.0 fixture: a series of the Point Drill, Initial Drill, Ø4.0 Final Drill, Ø5.0 Final Drill and Ø6.0 Final Drill.

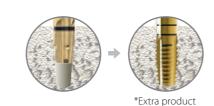


* The Countersink and Tap Drill are used in hard bone quality.

Sloped edentulous ridge adjacent of tooth

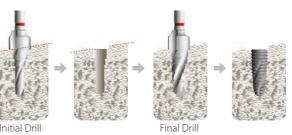

Crestal flatting

> Crestal cutter of the Initial Drill and Final Drill. > Longer drills than fixture's length.

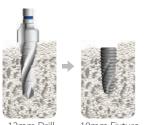

Longer Final Drill

Wide extraction socket


- > Absence of the cortical bone & spongy bone.
- > Narrower diameter than the fixture's diameter.



Ø4.5 Fixture

***** Torque force control


> The 0.5mm deeper placement increases the initial torque force of fixture.

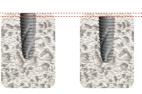



Fixture Placement Level						
Level	Cre	Crestal Level		0.5mm Deeper Leve		r Level
Density	D1	D2	D3	D1	D2	D3
Torque	34.1	29	15.5	44.4	38.4	19.1

- > The pumping action of drill removes the bone chip in the hole.
- > In dense bone, the debriment action decreases the high torque force.

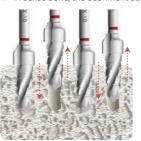


Pumping action of final drill for debriment				
Density	D1	D2	D3	
Non- Debriment	34.1	29	19.6	
Debriment	30	25	15.5	

* In weak bone quality 4 of maxillary tuberosity

: 0.5mm deeper level.

- > No pumping action.
- > 0.5mm deeper placement of fixture.
- > Wider fixture than Final Drill.

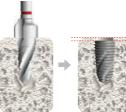

Ø5.0 Fixture

Level	Crestal level		0.5mm Deeper Lev		
Debriment	with	without	with	without	
Ø4.5 Fixture	4.4	10.2	-	12.9	
Ø5.0 Fixture	11.6	19.9	14.1	24.5	

118 SURGICAL KIT

INNO SUB. SMART SURGICAL KIT [KSA002]



- > For the INNO Submerged Implant System (Sub. Ø3.5, 4.0, 4.5 & 5.0).
- > A simpler kit for implant surgery, applied to fixtures with 8~14mm in length using the Drills and Stoppers.



















Stopper













Extension & Driver















Length Marking & Stopper Actual Length of Drill: Fixture length + 1mm

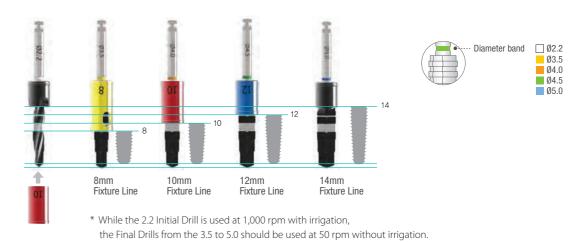

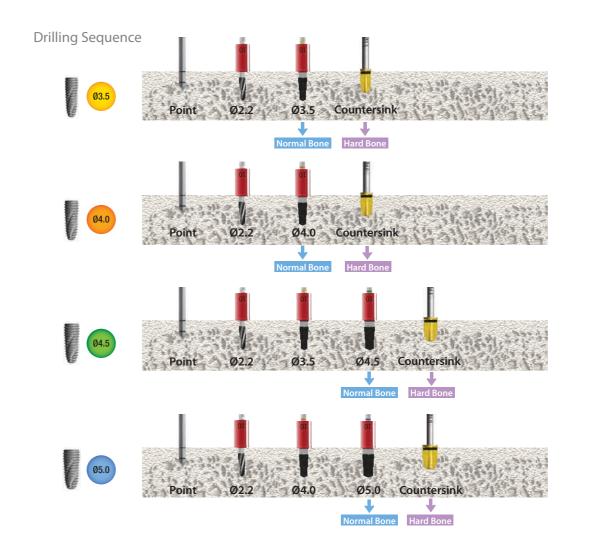

Ø4.5 Ø5.0

Ø6.0

INNO SUB-SHORT SURGICAL KIT [KS1001]



> For the INNO Submerged Short Implant System (Sub.).





















Stopper

4mm Drill Stopper

SIDS04

5.5mm Drill Stopper SIDS05

KSSD2004













Extension & Driver










Length Marking & Stopper Actual length of drill: Fixture + 0.5mm. Diameter band 02.0 04.0 Fixture Line Fixture Line Fixture Line

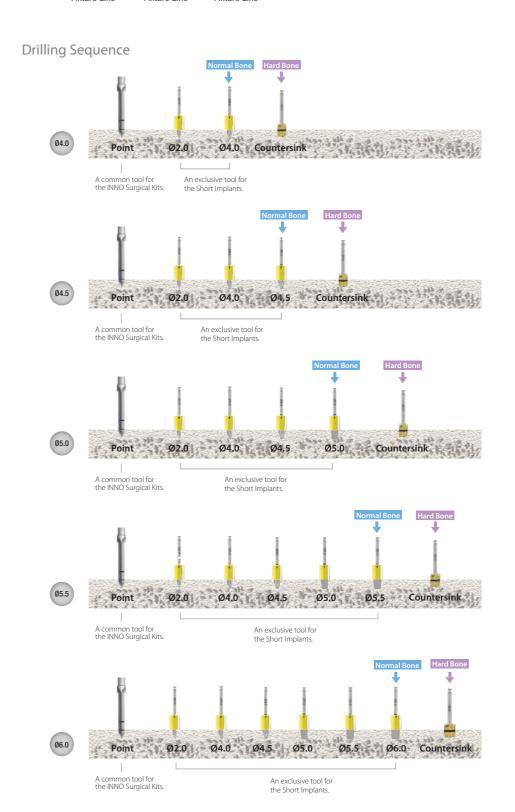

INNO SUB-NARROW SURGICAL KIT [KNA001]



> For the INNO Submerged Narrow Implant System (Sub-N).















Stopper



10 Drill Stopper KNDS10







Extension & Driver












Fixture Driver



Туре	Ratchet	Machine
	KHDS01XN	KMMS01XN

Mount Driver

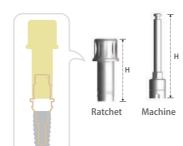

Туре	Ratchet	Machine
	KRMD19L	*KMMD12X
Llood to inct	all facture	*Extra produc

Parallel Pin



Code	KPP 003
	After Ø2.2 Drilling. After Ø3.1 / Ø3.3 Drilling.

Cuff Height



COWELL® PROSTHETIC KIT [KPA003]



- > For the INNO Submerged Implant System.
- > Try-in Kit for determining abutment specifications.




Abutment Gauge > Measuring of the diameter, cuff length and angulation of abutment.







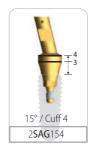





Straight Type

Angulated Type

Abutment Driver > Used in the Shoulder, Solid, Absolute, and Straight Abutment placement.












Trephine soft tissue over the Cover Screw of fixture.

Mini Plus® Implant system


Mini Plus® Implant
Cement Type
Ball Type
Surgical kit

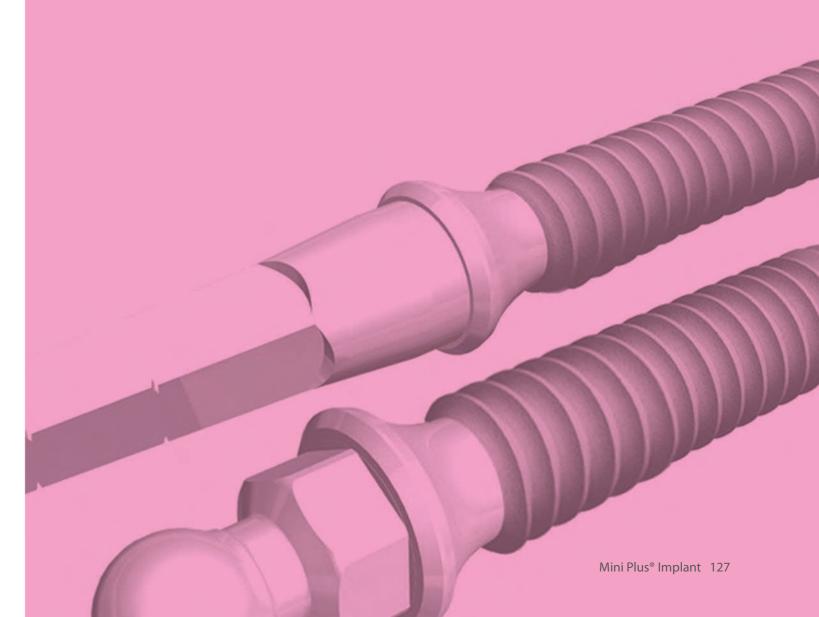

DESIGN OF MINI PLUS® FIXTURE (1P-C.)

Cement Type


- > For mandible anterior spaces and edentulous arch.
- > For semi-permanent or temporary solution.

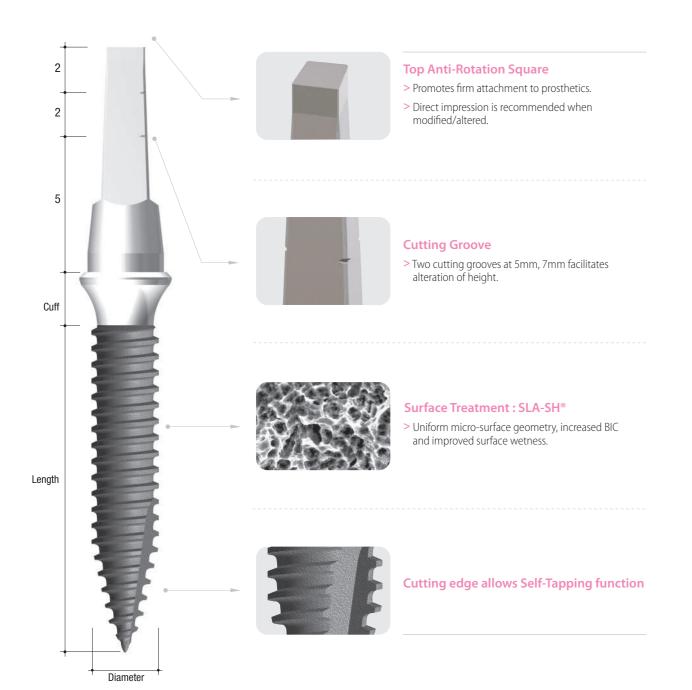

System Flow



Implant

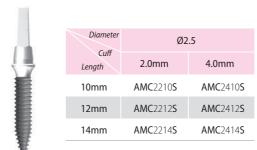

- > Packing unit : 1 Fixture.
- > Abutment level impression.

Diameter	Ø3.0	
Cuff Length	2.0mm	4.0mm
10mm	AMC 3210 S	AMC 3410 S
12mm	AMC3212S	AMC3412S
14mm	AMC3214S	AMC3414S

- > Packing unit: 1 Fixture.
- > Abutment level impression.

Impression Coping / Lab Analog




Impression Coping

- > Packing unit: 1 Impression Coping.
- > Fastened on the shoulder of the cement post and connected with the Lab Analog for working cast.
- > Direct impression is recommended when modified/altered.

Lab Analog

- > Packing unit: 1 Lab Analog.
- > The same adjustment must be made for the Lab Analog when the abutment portion of the fixture is modified/altered.
- > Replacement of the cement post shape in working mode.

Protection Cap

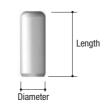

> Provides temporary protection of mucogingiva and tongue after implantation.

Diameter Length	Ø4.0
7mm	AMCC 001
9mm	AMCC 002
11mm	AMCC003


> Packing unit: 1 Protection Cap.

128 Mini Plus® Implant

DESIGN OF MINI PLUS® FIXTURE (1P-B.)

Ball Type

> For semi-permanent or temporary solution for overdenture prosthesis.

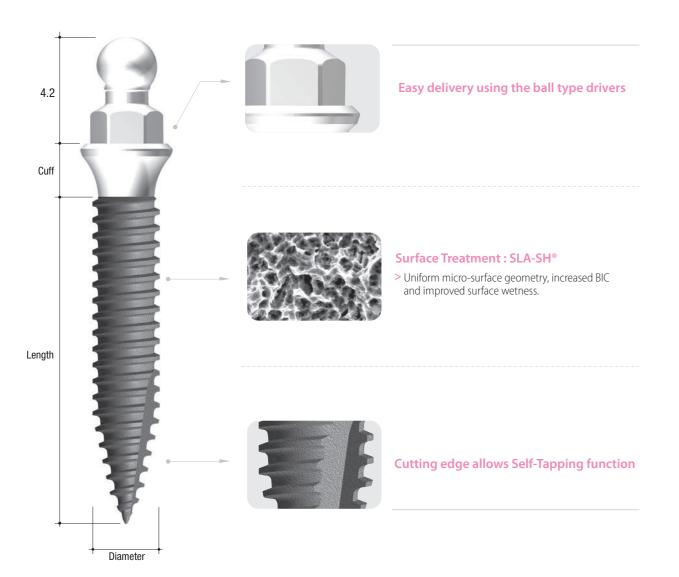

System Flow



Implant



Diameter	Ø2.5	
Cuff Length	2.0mm	4.0mm
10mm	AMB2210S	AMB2410S
12mm	AMB2212 S	AMB2412S
14mm	AMB2214S	AMB2414S

> Packing unit: 1 Fixture.

Diameter	Ø	3.0
Cuff Length	2.0mm	4.0mm
10mm	AMB 3210 S	AMB 3410 S
12mm	AMB3212S	AMB3412S
14mm	AMB 3214 S	AMB 3414 S

> Packing unit: 1 Fixture.

Ball Outer Cap




Diameter Length	Ø3.4
2.4	BATC003C

BATC003I

> Packing unit : 2 Outer Caps.

Ball Inner Cap

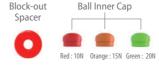

- > Packing unit: 2 Block-out Spacers + 6 Inner Caps (2 per each color).
- > Retention force : Red 10N, Orange 15N & Green 20N.

Impression Coping / Lab Analog



Impression Coping

- > Packing unit: 1 Impression Coping.
- > Used for impression taking of the ball post.


Lab Analog

- > Packing unit: 1 Lab Analog.
- > Replacement of the ball post shape in working model.

130 Mini Plus® Implant




Point Drill

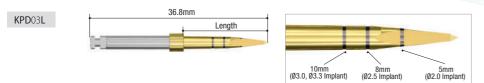


Ø1.3 Twist Drill



Ø1.8 Twist Drill

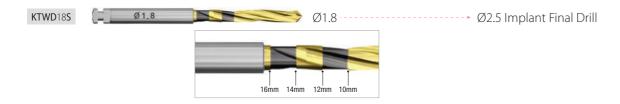


Ø2.3 / Ø2.6 Twist Drill

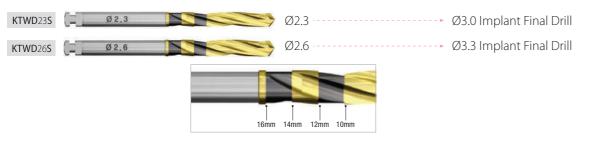


Driver




Tissue Punch *Extra product

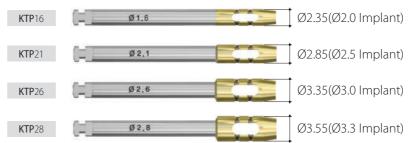

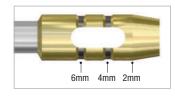


- > Easy removal of soft tissue for flapless surgery.
- > 0.3mm wider than implant diameter allows more predictable results.




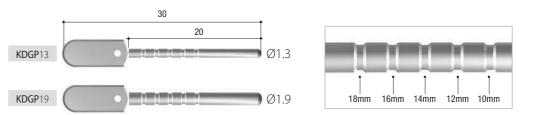


Drilling Sequence



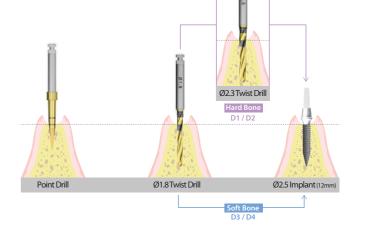

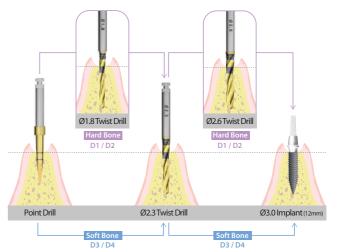

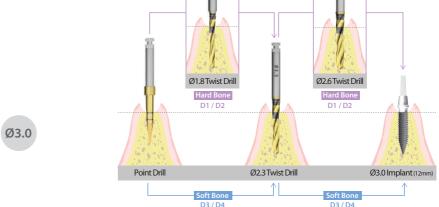

* In case of D4 bone quality, make proper adjustment in drilling. Due to narrow diameter, the Mini Plus® Implants have self-tapping characteristic.

132 Mini Plus® Implant

COWELL® DIGITAL PRODUCTS

Drive yourself to COWELLMEDI's Digital Transformation

Digital Guided Surgery Kit

InnoFit® Lodestar Plus Kit
InnoFit® Lodestar Kit

Digital Prosthesis

SFIT Ti-Base System

- · Sub. SFIT Ti-Base System
- · Sub. SFIT Absolute Ti-Base System
- · Sub. & Sub-N. Multi SFIT Ti-Base System
- · Int. SFIT Ti-Base System

InnoFit® Hybrid Ti-Base System

- · Sub. Hybrid Ti-Base System
- · Sub. & Sub-N. Multi Hybrid Ti-Base System
- · Sub. Lock Hybrid Ti-Base System
- · Sub-N. Hybrid Ti-Base System
- · Int. Hybrid Ti-Base System

InnoFit® Lodestar Plus Kit



Exclusive for the INNO Submerged and Submerged Narrow Implant System.

InnoFit® Lodestar Kit



Universal to any implant system.

COWELL® DIGITAL PRODUCTS

SFIT Ti-Base System



Cementless (Spread Fit™)




Angulated screw channels in both anterior and posterior teeth



Low prosthetic height



InnoFit® Lodestar Plus Kit [KLSP001]

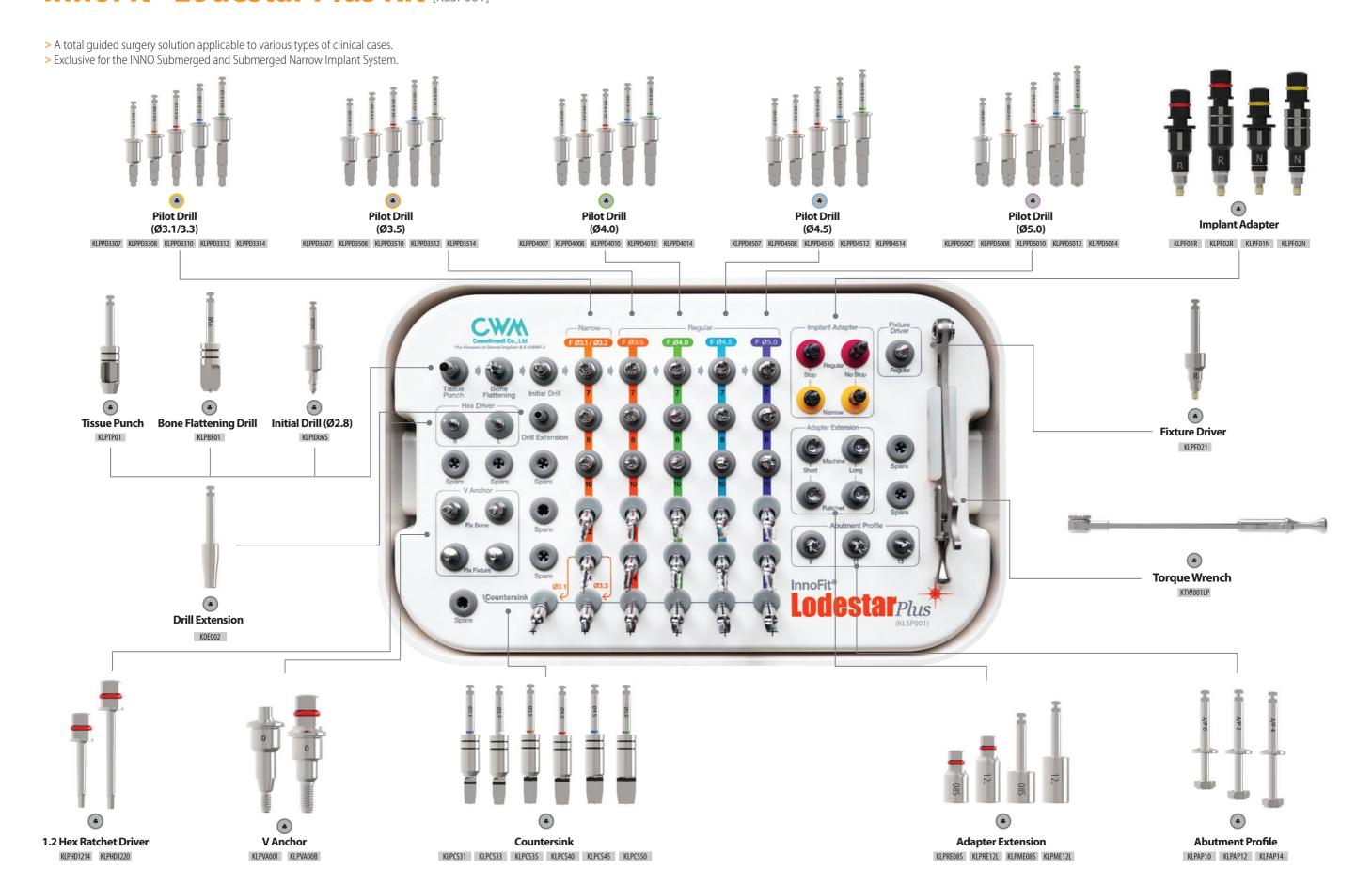

Workflow



Preparation before Operation



Disinfection of surgical guide template

Disinfection must be done before the operation. Immerse the surgical guide template into the alcohol and chlorhexidine solution in a ratio of 9:1 or disinfection fluids such as CidexOPA, betadine, etc. for more than 20 minutes. Then rinse with the saline solution and install in patient's oral cavity.



Installation of surgical guide template

- Check if inward of surgical guide and outward of teeth are accurately contacted through the windows of mounted surgical guide template. In case of insufficient scan data, delete and adjust the inner side of the surgical guide template to contact precisely.
- Install the surgical guide template while scanning CT to check implantation path and precision before the operation (Implantation path may also be checked in post operation by scanning CT with installation of the surgical guide template).



Verification of dental implant

Check if marked dental implant is in the surgical report.





Confirmation of protocol

Confirm the surgical report and surgical protocol for the last time.

Comprehension and Usage of Offset


- > The basic length from the fixture platform to the top of the Sleeve is 9mm.
- > In case that gingiva is thick or fixture needs to be placed deeper due to low bone density, use the Sleeve 2 or 4mm upright to the top.
- > The higher offset value, the less accurate it will be, so use 9mm if possible.

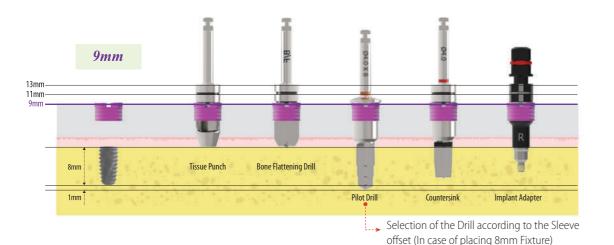

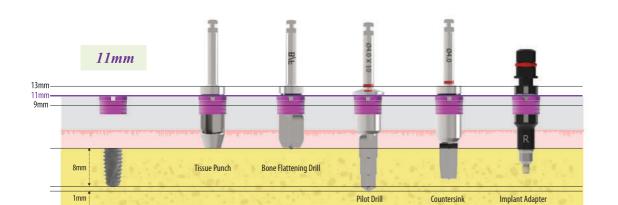

Selection of the Drill according to the Sleeve offset (In case of placing 8mm Fixture) : In case of offset 11mm(2mm) - Select 10mm Drill.

: In case of offset 9mm(0mm) - Select 8mm Drill.

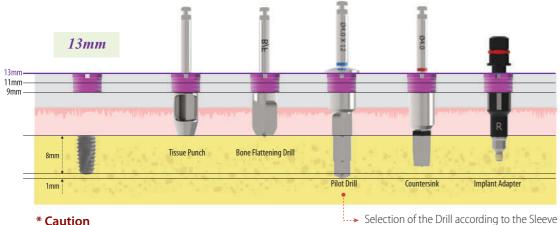

* Caution

Please be noted that actual depth of drilling is 1mm deeper than marks of the Drill. Ex) 4.0 X 8mm Drill - Drilling depth: 9mm.

offset (In case of placing 8mm Fixture) : In case of offset 13mm(4mm) - Select 12mm Drill.

Tissue Punch



- > Used for soft tissue elimination (the gingiva in the position where the implant is to be placed can be incised in a circular shape).
- > Hemostatic effect, small scar, fast wound healing affect is occurred after the operation due to small diameter of tissue punch.
- > Able to apply offset (9mm, 11mm, 13mm).
- > 50 rpm without irrigation.

Double blade

The internal cutting edge of the Tissue Punch cuts the gingiva into small pieces so that those can be removed by suction without extra work.



* Caution The Tissue Punch must be kept clean. Otherwise, it may cause rust or blade damage due to residual gingival pieces or others in the Tissue Punch after the operation (remove the residual gingiva piece by explorer, steam and etc.).

Bone Flattening Drill




- > Flattens the bone level of operation site.
- > Inclined bone level may glide the Drill and can not drill as planned.
- > Eliminate the soft tissue after using the Tissue Punch.
- > The point in the middle of the Drill guides the position of the Drill and helps to drill in an accurate site.
- > Able to apply offset (9mm, 11mm, 13mm).
- > 400 rpm without irrigation / 800 rpm with irrigation.



The point in the middle of the Drill guides the position of the Drill and helps to drill in an accurate site.

Initial Drill

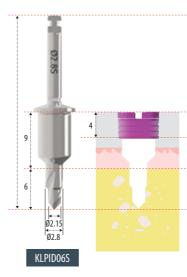


> High speed, 1,000 rpm with irrigation.





Creates the hole on the bone surface so that the axis of the next step Drill is not moved and it guides the Drill position by preventing slip even at the inclined bone level.

140 InnoFit® Lodestar Plus Kit InnoFit® Lodestar Plus Kit 141


Ø3.1/Ø3.3 Fixture

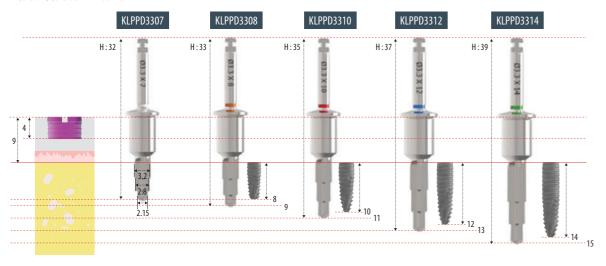

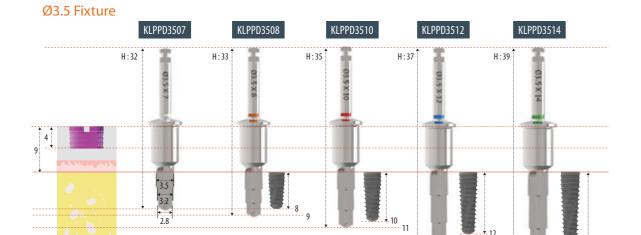


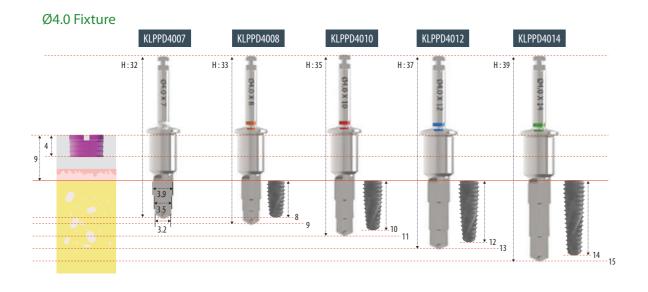

Ø4.5 Fixture

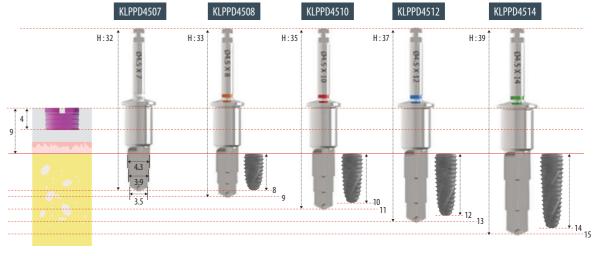

Ø5.0 Fixture

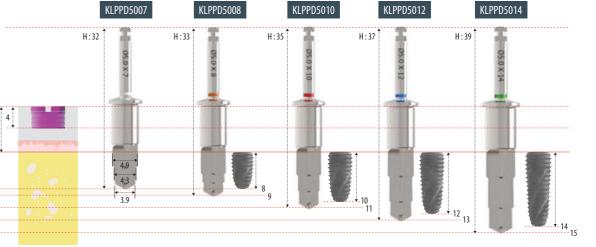

Countersink

- > Expand the cortical bone in D1/D2 bone to prevent excessive implantation of the fixture.
- > Able to apply offset (9mm, 11mm, 13mm).
- > 50 rpm without irrigation.



Adapter Extension

> In case the Implant Adapter is too short to use, connect the Ratchet or Machine Adapter Extension to place the fixture.

Groove for Removal

In case of cold welding,



Implant Adapter


- > Move fixture to the Sleeve to implant safely.
- > Match the depth of laser marks of the Sleeve offset and Implant Adapter.
- > When implanting the fixture, the direction of the Implant Adapter and directional identification groove of the Sleeve are matched, it lines with the hex direction of temporary abutment.



Fixture Driver - Molar




- > Use in case the Implant Adapter can not be used due to small size of opened mouth or narrow gap between antagonist tooth.
- > After implanting 4~5mm, change to the Implant Adapter to complete placement.

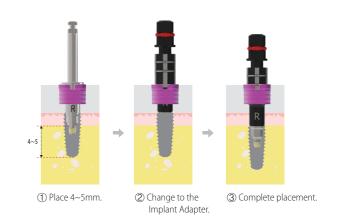

Abutment Profile

- > Used for elimination of the alveolar bone that interferes with the accurate connection of abutment. Remove residual bone by rotating and drilling 360°.
- > In case of thick cortical bone, higher the drilling rpm and use with irrigation (within 100 rpm).

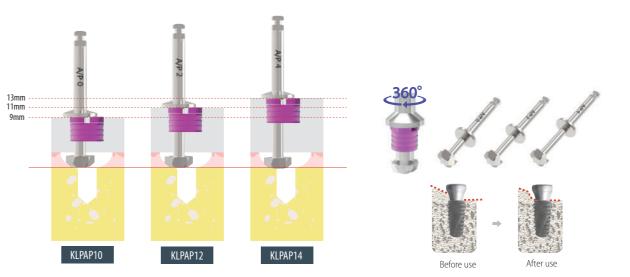

144 InnoFit® Lodestar Plus Kit

InnoFit® Lodestar Plus Kit 145

V Anchor - Fix Fixture

> Connect the 1.2 Hex Driver to implanted fixture to prevent the movement of the surgical guide template in cases as edentulous.



- Connect by aligning to the Sleeve offset of connected fixture.
- Basic composition of the Sleeve offset 9mm (11, 13mm extra).

V Anchor - Fix Bone

> Fix the V Anchor using the Torque Wrench in the hole made after initial drilling to prevent the movement of the surgical guide template in cases as edentulous.




- Connect by aligning to the Sleeve offset of connected fixture.
- Basic composition of the Sleeve offset 9mm (11, 13mm extra).

1.2 Hex Ratchet Driver



Torque Wrench(Square)


- > Use to implant the fixture (Connect to the Implant Adapter).
- > Use after connecting to the 1.2 Hex Driver.
- > Use after connecting to the V Anchor (Fix Bone).

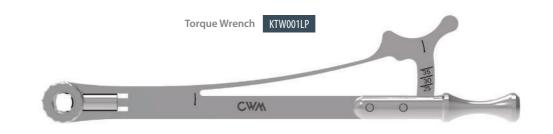




* Packing Unit: 5 Sleeves

Extra

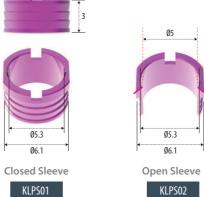

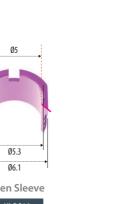

* Packing Unit: 5 Sleeves



> Use in case of connecting the Cover Screw or Healing Abutment.



Drill Protocol


Drill Protocol (7~10mm)

INNO Sub Fixture Ø5 x 10mm




Drill Protocol(12~14mm)

INNO Sub Fixture Ø5 x 14mm

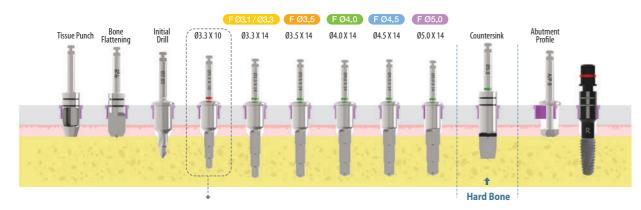

8~10mm drilling should be done in advance for the sleeve contact.


- * **Drilling method** Make sure with drilling in a desired direction without a change in path through the primary drill contact (A) with the hole created by the previous drilling and the secondary contact (B) with the sleeve.
 - Create the hole using the initial drill and insert the next drill into the hole made during the previous step and drill after achieving the drill and sleeve contact (A&B).
 - If drilling only with the sleeve contact (B) without the drill contact (A), the path may not be correct.

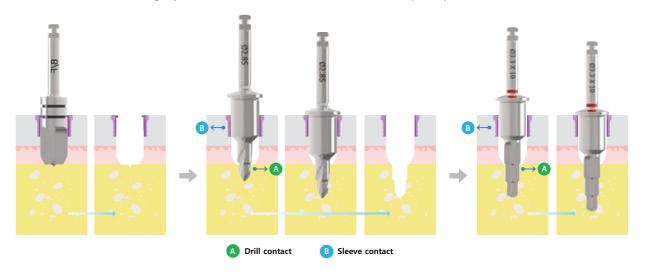

* Precaution when irrigating

- Irrigate enough to the end of the drill hole.




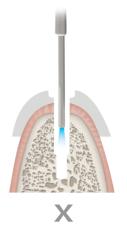


InnoFit® Lodestar Plus Kit 149

InnoFit® Lodestar Kit [KLS001]


- > A cost-effective guided surgery solution applicable to various types of clinical cases.
- > Universal to any implant system.

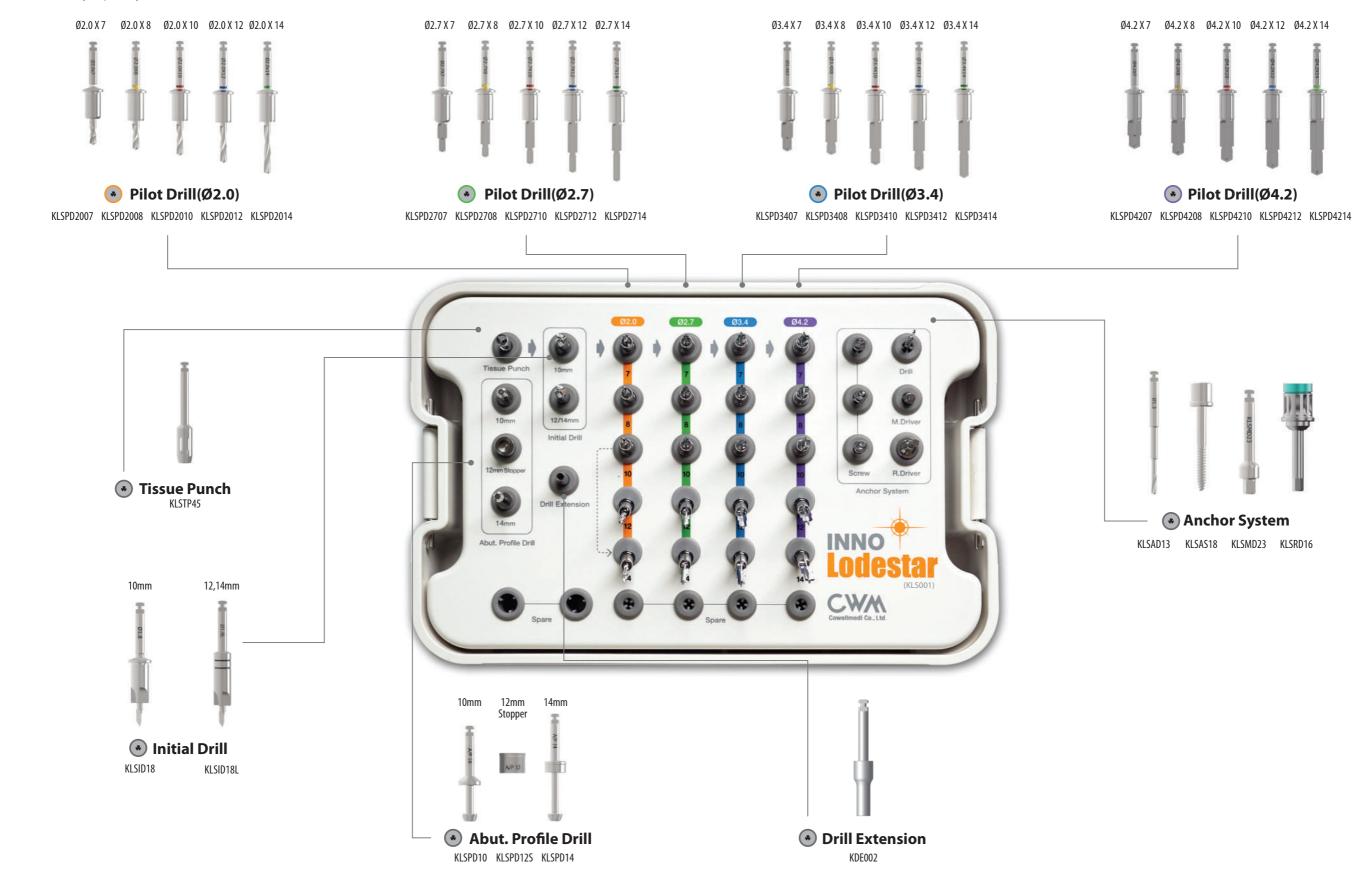


Tissue Punch

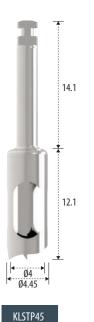

> The gingiva in the position where the implant is to be placed can be incised in a circular shape, and it can also be used in inclined bones (50 rpm without irrigation).

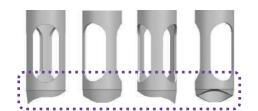

The gingiva can be incised in a circular shape although bone level is inclined or not parallel.

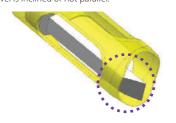



The internal cutting edge of the Tissue Punch cuts the gingiva into small pieces so that those can be removed by suction without extra work.



Initial Drill

> The Drill combined with Bone Flattening Drill and Point Drill which no separate Bone Flattening Drill is required provides simper procedure and shorter chair time (1,000 rpm with irrigation).

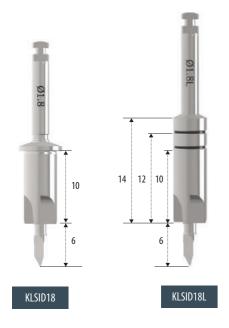


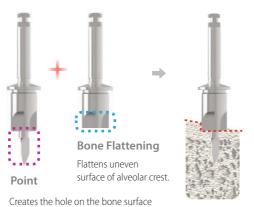


Creates the hole on the bone surface so that the axis of the next step Drill is not moved and it guides the Drill position by preventing slip even at the inclined bone level.

Pilot Drill

> Ø2.0 / Ø2.7 / Ø3.4 / Ø4.2.

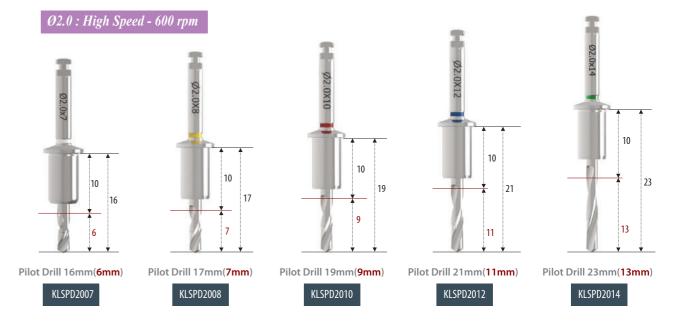




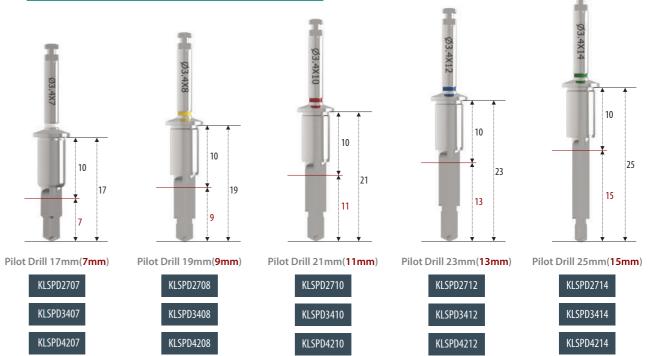

152 InnoFit® Lodestar Kit


Abutment Profile

> Used for elimination of the alveolar bone that interferes with the accurate connection of abutment.

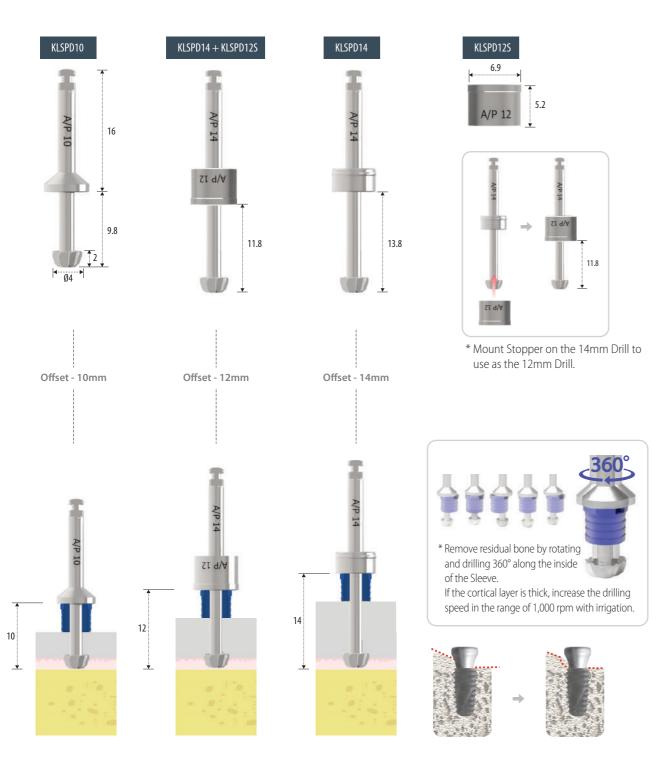

* Offset length setting

- The basic length from the fixture platform to the top of the Sleeve is 10mm.
- In case that gingival is thick or fixture needs to be placed deeper due to low bone density, use the Sleeve 2 or 4mm upright to the top.
- The higher offset value, the less accurate it will be, so use 10mm if possible.

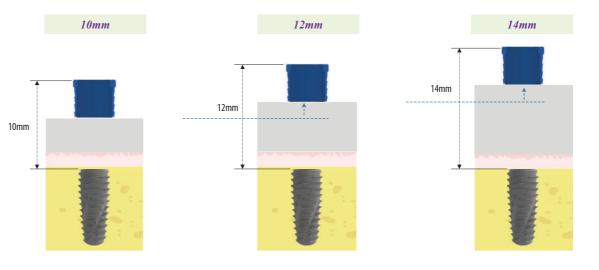

Anchor System



154 InnoFit® Lodestar Kit

InnoFit® Lodestar Kit 155


Optional


> These products are optional as extra ones which are not included in the kit.


Guide Reamer Extra

Used for precise contact of Drill and Sleeve (Sleeve / Non Sleeve). Use the 4.5mm Guide Reamer for Non Sleeve, and the 5.3 Guide Reamer for Sleeve (800 rpm without irrigation).


Guide Reamer (Non Sleeve) KLSRD45

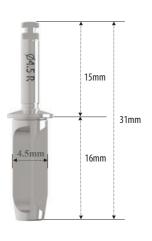

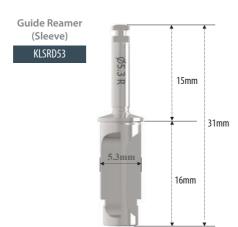

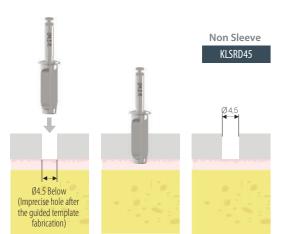


Revises imprecisely formed hole after the guided template fabrication using the 4.5 Guide Reamer to create the hole to be in exact contact with the Drill.

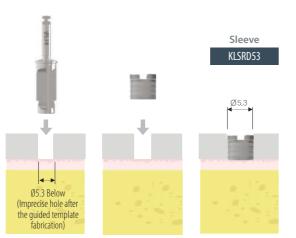


Revises imprecisely formed hole after the guided template fabrication using the 5.3mm Guide Reamer to precisely insert the Sleeve.


Sleeve Closed Sleeve



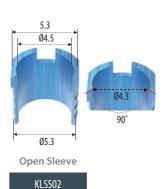



* Packing Unit: 5 Sleeves

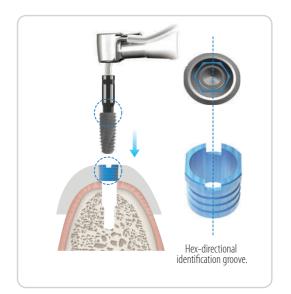


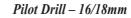




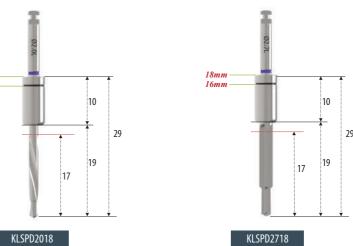

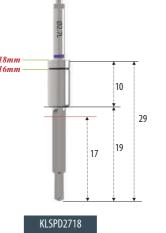


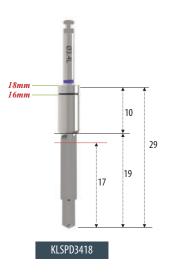

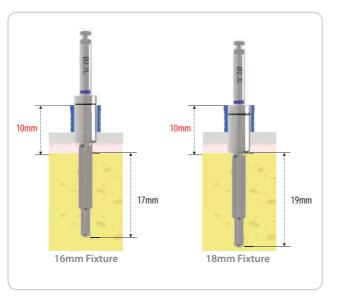


Drilling Sequence

Drill Protocol (Non Sleeve)

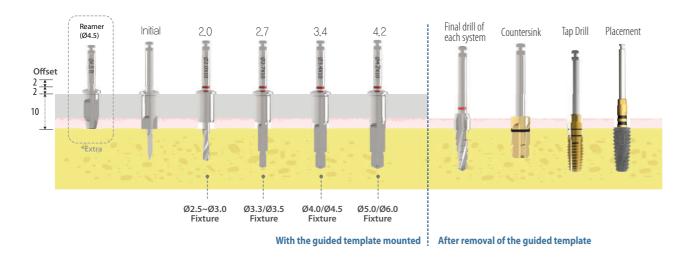


Drill Protocol (Sleeve)

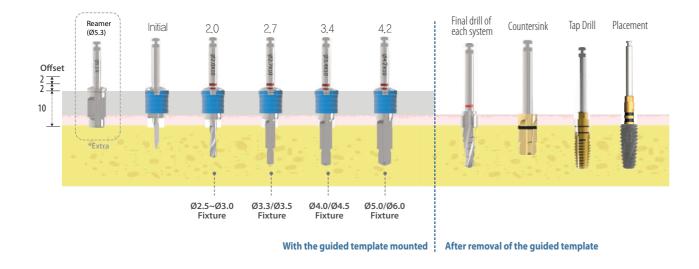

* The 14mm Drill must be used after using the 10mm Drill to enable Sleeve contact


For the use of the 14mm Drill with exact contact to the Sleeve, use the Ø2.0x10mm Drill first before using the 14mm Drill.

e.g.) 3.4 X 14mm Drilling Sequence

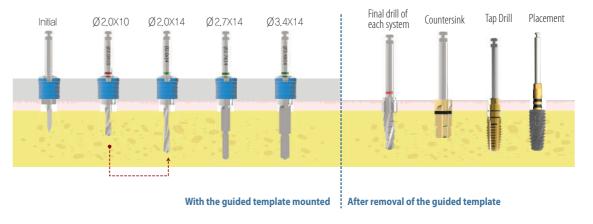


* Drilling method


- Make sure with drilling in a desired direction without a change in path through the primary Drill contact (A) with the hole created by the previous drilling and the secondary contact (B) with the Sleeve.
- Create the hole using the Initial Drill and insert the next drill into the hole made during the previous step and Drill after achieving the Drill and Sleeve contact (A&B).
- If drilling only with the Sleeve contact (B) without the Drill contact (A), the path may not be correct.

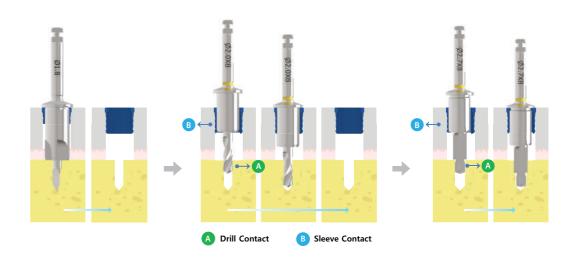

* Precaution when irrigating

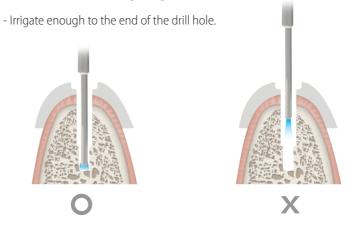


Intra-oral scanning procedure

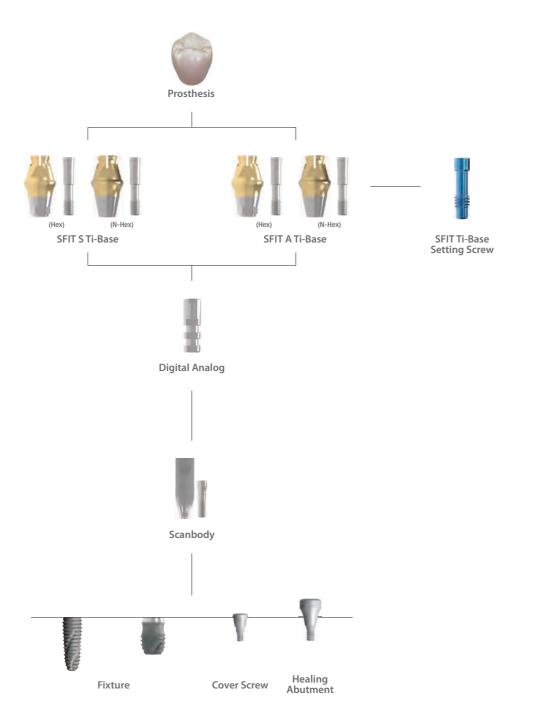

Component selection guide for the Sub. SFIT Ti-Base System

 Model-scanning Handwork procedure

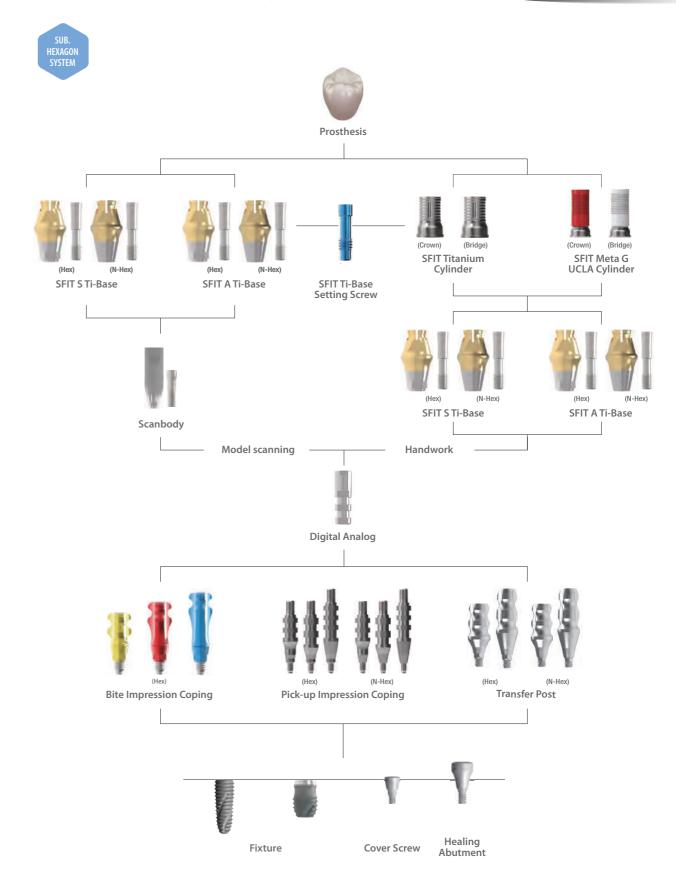


Hex



Туре	Hex			N-Hex		
Diameter	Ø3.8	Ø4.5	Ø5.5	Ø3.8	Ø4.5	Ø5.5
Length Cuff	3.6	3.6	3.6	3.6	3.6	3.6
1	2 SSFH 38135	2 SSFH 45135	2 SSFH 55135	2 SSFN 38135	2 SSFN 45135	2 SSFN 55135
2	2 SSFH 38235	2 SSFH 45235	2 SSFH 55235	2 SSFN 38235	2 SSFN 45235	2 SSFN 55235
3	2 SSFH 38335	2 SSFH 45335	2 SSFH 55335	2 SSFN 38335	2 SSFN 45335	2 SSFN 55335
4	2 SSFH 38435	2 SSFH 45435	2 SSFH 55435	2 SSFN 38435	2 SSFN 45435	2 SSFN 55435

- > Packing unit: 1 SFIT S Ti-Base + 1 SFIT S Ti-Base Screw.
- > For Screw Retained Prosthesis with straight screw channel.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Connected with the SFIT STi-Base Screw.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the SFIT Scanbody for digital workflow.
- > Fixture level impression.

SFIT A	A Ti-Base	5	1.8 Stargrip • _ [•···
D	L C		2.3
Ø3.8	Ø4.5	Ø5.5	
	Hex		

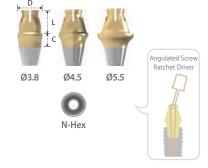

Туре	Hex			N-Hex		
Diameter	Ø3.8	Ø4.5	Ø5.5	Ø3.8	Ø4.5	Ø5.5
Length Cuff	3.6	3.6	3.6	3.6	3.6	3.6
1	2 SSFH 38135 A	2 SSFH 45135 A	2 SSFH 55135 A	2 SSFN 38135 A	2 SSFN 45135 A	2 SSFN 55135 A
2	2 SSFH 38235 A	2 SSFH 45235 A	2 SSFH 55235 A	2 SSFN 38235 A	2 SSFN 45235 A	2 SSFN 55235 A
3	2 SSFH 38335 A	2 SSFH 45335 A	2 SSFH 55335 A	2 SSFN 38335 A	2 SSFN 45335 A	2 SSFN 55335 A
4	2 SSFH 38435 A	2 SSFH 45435 A	2 SSFH 55435 A	2 SSFN 38435 A	2 SSFN 45435 A	2 SSFN 55435 A

- > Packing unit: 1 SFIT A Ti-Base + 1 SFIT A Ti-Base Screw.
- > For Screw Retained Prosthesis with angulated screw channel.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Connected with the SFIT A Ti-Base Screw.
- > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the SFIT Scanbody for digital workflow.
- > Fixture level impression.
- * Angulated Screw Ratchet Driver



Height Type	Ratchet
24(Short)	KRBUD15
29(Long)	KRBUD20

- > Stable to internal slip or fracture due to wide contact area of the Angulated Driver and
- the dedicated Stargrip Abutment Screw.

 > Tightening torque force: 30 N.cm (50 N.cm Max.).

SFIT S Ti-Base Screw



SFIT Ti-Base Cuff	1	2	3	4
Height Diameter	10.6	11.6	12.6	13.6
Ø2.3	2 SSFS 10 S 35	2 SSFS 20 S 35	2 SSFS 30 S 35	2 SSFS 40 S 35

- > Packing unit: 1 SFIT S Ti-Base Screw.
- > Used to connect the SFIT S Ti-Base.
- 2SSFS10S35: SFIT S Ti-Base with Cuff 1, 2SSFS20S35: SFIT S Ti-Base with Cuff 2, 2SSFS30S35: SFIT S Ti-Base with Cuff 3, 2SSFS40S35: SFIT S Ti-Base with Cuff 4.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.


SFIT A Ti-Base Screw



SFIT Ti-Base Cuff	1	2	3	4
Height Diameter	10.6	11.6	12.6	13.6
Ø2.3	2 SSFS 10 A 35	2 SSFS 20 A 35	2 SSFS 30 A 35	2 SSFS 40 A 35

- > Packing unit: 1 SFIT A Ti-Base Screw.
- > Used to connect the SFIT A Ti-Base.
- 2SSFS10A35: SFIT A Ti-Base with Cuff 1, 2SSFS20A35: SFIT A Ti-Base with Cuff 2, 2SSFS30A35: SFIT A Ti-Base with Cuff 3, 2SSFS40A35: SFIT A Ti-Base with Cuff 4.
- > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.

Scanbody

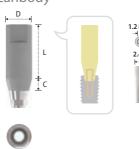

Туре	Hex
Diameter	Ø4.3
Length Cuff	8
2	2 SSB 4329

- > Packing unit: 1 Scanbody + 1 Abutment Screw (2SSHR100).
- > For both intra-oral scanning and model-scanning.
- > Made of 100% titanium alloy with a special coating applied.
- > No need to spray.
- > Connected with the Abutment Screw (2SSHR100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Bite Impression Coping




Туре	Hex(Short)	Hex(Long)	Hex(X-Long)
Diameter	Ø4.5	Ø4.5	Ø4.5
Length Cuff	2	4	6
4.0	2 SBIC 45 S	2 SBIC 45 L	2 SBIC 45 X

- > Packing unit: 1 Bite Impression Coping + 1 Guide Guide Pin (inbuilt).
- > Designed to simultaneously take bite and impression.
- > For closed tray impression (bite impression).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

162 SFIT Ti-Base System: Sub. SFIT Ti-Base

Pick-up Impression Coping

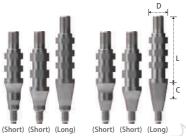


(Short)	(Short)	(Long)	(Short)	(Short)	(Long)	

)	(Short)	(Long)	(Short)	(Short)	(Long)	_
	0			0		2 9
	Hex			N-Hex		9

Туре	Hex			N-Hex		
Diameter Length / Cuff	Ø4.5	Ø5.5	Ø6.5	Ø4.5	Ø5.5	Ø6.5
14 (Short) / 4	2 SIH 454 S	2 SIH 554 S	2 SIH 654 S	2 SIN 454 S	2 SIN 554 S	2 SIN 654 S
14 (Short) / 2	2 SIH 45 S	2 SIH 55 S	2 SIH 65 S	2 SIN 45 S	2 SIN 55 S	2 SIN 65 S
16 (Long) / 4	2 SIH 45 L	2 SIH 55 L	2 SIH 65 L	2SIN45L	2 SIN 55 L	2SIN65L

- > Packing unit: 1 Pick-up Impression Coping + 1 Guide Pin (2SISR001SS / 2SISR001SL).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Tra	nsfe	ar P	nst

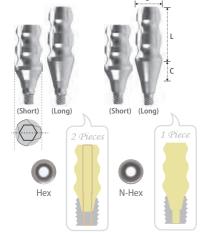

Туре		Hex			N-Hex	
Diameter Length / Cuff	Ø4.5	Ø5.5	Ø6.5	Ø4.5	Ø5.5	Ø6.5
9 (Short) / 2	2 STH 45 S	2 STH 55 S	2 STH 65 S	2 STN 45 S	2 STN 55 S	2 STN 65 S
11 (Long) / 4	2 STH 45 L	2 STH 55 L	2 STH 65 L	2STN45L	2STN55L	2STN65L

> Packing unit: Hex - 1 Transfer Post + 1 Guide Pin (Short: 2STH001SS / Long: 2STH001SL), N-HeX – 1 Transfer Post (solid screw).

- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Digital Analog



Diameter Height	Ø3.9
12	2 SDR 001

- > Packing unit: 1 Digital Analog.
- > Analog of fixture for working cast.
- > Used for both 3D printed model (RP) and stone model.


SFIT Ti-Base Setting Screw



Diameter Height	Ø1.95
8.2	2 SSFSS 100

- > Packing unit: 5 SFIT Ti-Base Setting Screws.
- > Used to screw the SFIT S or A Ti-Base into the Digital Analog or the Fixture when revising such prosthetic adaptions as shape, contact surface, occlusion and margin without giving a force to the head of the SFIT Ti-Base S or A Ti-Base to spread and fit in the inner space of the prosthesis.
- > Tightened with the 1.2 Hex Ratchet Driver.


SFIT Titanium Cylinder

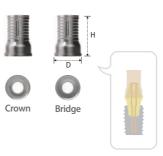

Туре		Crown			Bridge	
SFIT Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5	Ø3.8	Ø4.5	Ø5.5
— Diameter Height —	Ø4.1	Ø4.8	Ø5.8	Ø4.1	Ø4.8	Ø5.8
7	2 SSFTC 3835 H	2 SSFTC 4535 H	2 SSFTC 5535 H	2 SSFTC 3835 N	2 SSFTC 4535 N	2 SSFTC 5535 N

- > Packing unit: 1 SFIT Titanium Cylinder.
- > Used over both SFIT S or A Ti-Base and SFIT Absolute S or A Ti-Base.
- > For provisional and permanent restorations.
- > Connected with the SFIT S or A Ti-Base Screw and SFIT Absolute S or A Ti-Base Screw.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.

SFIT Meta G UCLA Cylinder

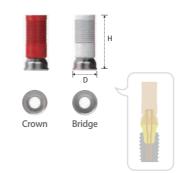

Туре		Crown			Bridge	
SFIT Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5	Ø3.8	Ø4.5	Ø5.5
Diameter Height	Ø4.3	Ø5.0	Ø5.7	Ø4.3	Ø5.0	Ø5.7
11.4	2 SSFC 3835 C	2 SSFC 4535 C	2 SSFC 5535 C	2 SSFC 3835 B	2 SSFC 4535 B	2 SSFC 5535 B

- > Packing unit: 1 SFIT Meta G UCLA Cylinder.
- > Used over both SFIT S or A Ti-Base and SFIT Absolute S or A Ti-Base.
- > Modifiable to various types of abutments.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the SFIT S or A Ti-Base Screw and SFIT Absolute S or A Ti-Base Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.

SFIT Gono Gauge



4	Н	
D G	4.5/5.5	Nh
_	3244.322	

SFIT Ti-Base Diameter	Ø3.8	Ø4.5 & Ø5.5
Diameter Height	Ø5.5	Ø5.5
32	2SSF	GNG

- > Packing unit: 2 SFIT Gono Gauge (1 3.8 SFIT Gono Gauge, 1 4.5 & 5.5 SFIT Gono Gauge).
- > Used to check whether the inner milling is done correctly when fabricating the crown.

When fastening in the G(Go) direction, the contact point with the crown must fit.





When tightening in the N(No) direction, the contact point with the crown must misfit.

* Refer to the SFIT Ti-Base System Catalog for details.


164 SFIT Ti-Base System: Sub. SFIT Ti-Base

SFIT Ti-Base System: Sub. SFIT Ti-Base 165

Component selection guide for the Sub. SFIT Absolute Ti-Base System

Intra-oral scanning procedure




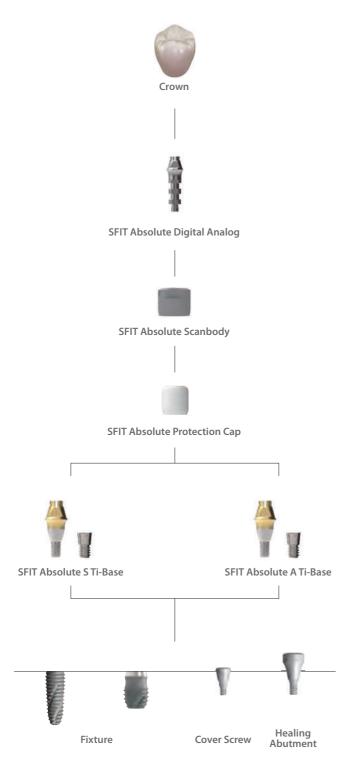

Component selection guide for the Sub. SFIT Absolute Ti-Base System

Model-scanningHandwork procedure



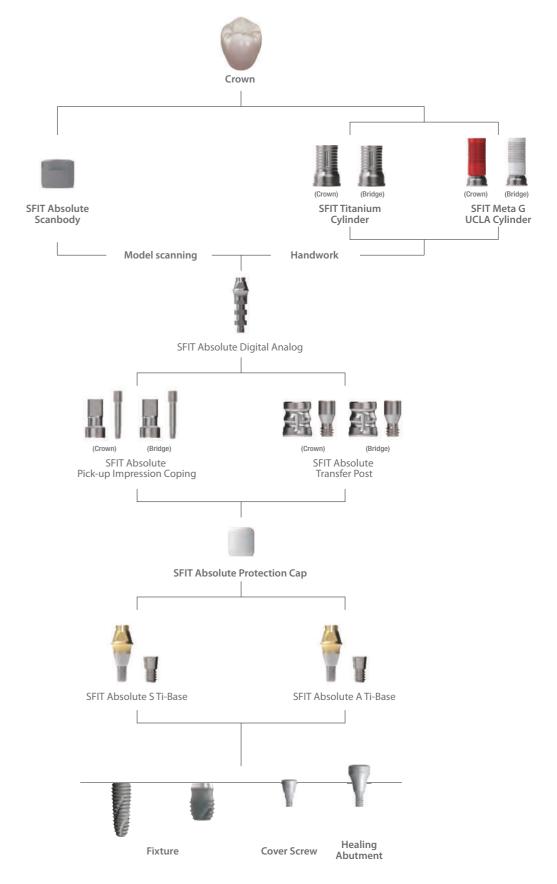

SFIT Absolute S Ti-Base









Diameter	Ø3.8	Ø4.5	Ø5.5
Length Cuff	3.6	3.6	3.6
1	2 SSF 38135	2 SSF 45135	2 SSF 55135
2	2 SSF 38235	2 SSF 45235	2 SSF 55235
3	2 SSF 38335	2 SSF 45335	2 SSF 55335
4	2 SSF 38435	2 SSF 45435	2 SSF 55435

- > Packing unit: 1 SFIT Absolute S Ti-Base + 1 SFIT Absolute S Ti-Base Screw.
- > For Screw Retained Prosthesis with straight screw channel.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Integrated with the Screw and Ti-Base (solid screw).
- > Tightened with the 1.2 Hex Ratchet Driver or Absolute Ratchet Driver and Torque Wrench.
- > Prosthesis is connected to the SFIT Absolute S Ti-Base by the SFIT Absolute S Ti-Base Screw using the 1.2 Hex Ratchet Driver.
- > Tightening torque force: 30 N.cm.
- > Use the SFIT Absolute Scanbody for digital flow.
- > Abutment level impression.

SFIT Absolute A Ti-Base







Diameter	Ø3.8	Ø4.5	Ø5.5
Length Cuff	3.6	3.6	3.6
1	2 SSF 38135 A	2 SSF 45135 A	2 SSF 55135 A
2	2 SSF 38235 A	2 SSF 45235 A	2 SSF 55235 A
3	2 SSF 38335 A	2 SSF 45335 A	2 SSF 55335 A
4	2 SSF 38435 A	2 SSF 45435 A	2 SSF 55435 A

- > Packing unit: 1 SFIT Absolute A Ti-Base + 1 SFIT Absolute A Ti-Base Screw.
- > For Screw Retained Prosthesis with angulated screw channel.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Integrated with the Screw and Ti-Base (solid screw).
- > Tightened with the 1.2 Hex Ratchet Driver or Absolute Ratchet Driver and Torque Wrench.
- > Prosthesis is connected to the SFIT Absolute A Ti-Base by the SFIT Absolute A Ti-Base Screw using the Angulated Screw Ratchet Driver.
- > Tightening torque force: 30 N.cm.
- > Use the SFIT Absolute Scanbody for digital flow.
- > Abutment level impression.
- * Angulated Screw Ratchet Driver



Height Type	Ratchet
24(Short)	KRBUD15
29(Long)	KRBUD20
29(Long)	KRBUD20

- > Stable to internal slip or fracture due to wide contact area of the Angulated Driver and the dedicated Stargrip Abutment Screw.
- > Tightening torque force : 30 N.cm (50 N.cm Max.).

SFIT Absolute S Ti-Base Screw



Diameter Height	Ø2.3
4.1	2SSFS100S

- > Packing unit: 1 SFIT Absolute S Ti-Base Screw.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.


SFIT Absolute A Ti-Base Screw



Diameter Height	Ø2.3	
4.1	2 SSFS 100 A	

- > Packing unit: 1 SFIT Absolute A Ti-Base Screw.
- > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.

SFIT Absolute Ratchet Driver

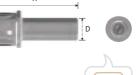




Crown					
Ø3.8	Ø4.5 & Ø5.5				
Ø4.2	Ø4.8				
KSFR 3812 S	KSFR 4512 S				
KSFR3819L	KSFR4519L				
	Ø3.8 Ø4.2 KSFR3812S				

- > Packing unit: 1 SFIT Absolute Ratchet Driver.
- > Used to install and uninstall the SFIT Absolute S & A Ti-Base in case of internal slip.
- > KSFR3812S & KSFR3819L for the SFIT Absolute S & A Ti-Base with Ø3.8 and KSFR4512S & KSFR4519L for the SFIT Absolute S & A Ti-Base with Ø4.5 and Ø5.5.

SFIT Absolute Protection Cap

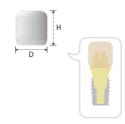

SFIT Absolute Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5
Diameter Height	Ø4.6	Ø5.5	Ø6.5
5	2 SSFPC 38	2 SSFPC 45	2 SSFPC 55

- > Packing unit: 1 SFIT Absolute Protection Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Gingival retraction for prosthodontic margin of abutment.
- > Alternative usage for sub-structure of temporary prosthesis.


SFIT Absolute Scanbody



Туре	Crown					
SFIT Absolute Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5			
Diameter Height	Ø4.25	Ø4.95	Ø5.85			
4.1	2 SSFSB 3835 H	2 SSFSB 4535 H	2 SSFSB 5535 H			

- > Packing unit: 1 SFIT Absolute Scanbody.
- > For both intra-oral scanning and model-scanning.
- > Made of 100% titanium alloy with a special coating applied.
- > No need to spray
- > For the connection to the SFIT Absolute S Ti-Base, use the SFIT Absolute S Ti-Base Screw with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > For the connection to the SFIT Absolute A Ti-Base, use the SFIT Absolute A Ti-Base Screw with the Angulated Screw Ratchet Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.


SFIT Absolute Pick-up Impression Coping

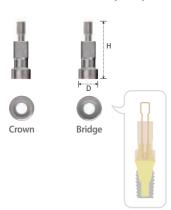

Туре	Crown			Bridge		
SFIT Absolute Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5	Ø3.8	Ø4.5	Ø5.5
Diameter Height	Ø3.95	Ø4.65	Ø5.65	Ø3.95	Ø4.65	Ø5.65
14.6	2 SSFI 3835 C	2 SSFI 4535 C	2 SSFI 5535 C	2 SSFI 3835 B	2 SSFI 4535 B	2 SSFI 5535 B

- > Packing unit: 1 SFIT Absolute Pick-up Impression Coping + 1 Guide Pin.
- > For open tray impression.
- > Connected with the Guide Pin (2SSFIGP14).
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

SFIT Absolute Transfer Post

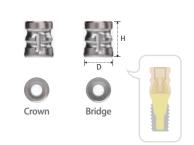

Туре	Crown			Bridge		
SFIT Absolute Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5	Ø3.8	Ø4.5	Ø5.5
Diameter Height	Ø4.3	Ø4.95	Ø5.9	Ø4.3	Ø4.95	Ø5.9
6	2 SSFT 3835 LH	2 SSFT 4535 LH	2 SSFT 5535 LH	2 SSFT 3835 LN	2 SSFT 4535 LN	2 SSFT 5535 LN

- > Packing unit: 1 SFIT Absolute Transfer Post + 1 Guide Pin.
- > For closed tray impression.
- > Also used as a scanbody for intra-oral scanning and model-scanning.
- > Connected with the Guide Pin (2SSFT01SL).
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

SFIT Absolute Digital Analog



SFIT Absolute Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5
	Ø3.8	Ø4.5	Ø5.5
3.6	2 SSFLA 3835	2 SSFLA 4535	2 SSFLA 5535

- > Packing unit: 1 SFIT Absolute Digital Analog.
- > Replacement of the SFIT Absolute S or A Ti-Base shape in working cast.
- > Used for both 3D printed model (RP) and stone model.
- > Select according to dimension of the SFIT Absolute S or A Ti-Base.

SFIT Titanium Cylinder


Crown



Туре	Crown			Bridge		
SFIT Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5	Ø3.8	Ø4.5	Ø5.5
Diameter Height	Ø4.1	Ø4.8	Ø5.8	Ø4.1	Ø4.8	Ø5.8
7	2 SSFTC 3835 H	2 SSFTC 4535 H	2 SSFTC 5535 H	2 SSFTC 3835 N	2 SSFTC 4535 N	2 SSFTC 5535 N

- > Packing unit: 1 SFIT Titanium Cylinder.
- > Used over both SFIT S or A Ti-Base and SFIT Absolute S or A Ti-Base.
- > For provisional and permanent restorations.
- > Connected with the SFIT S or A Ti-Base Screw and SFIT Absolute S or A Ti-Base Screw.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.

SFIT Meta G UCLA Cylinder

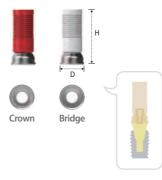

Туре	Crown			Bridge		
SFIT Ti-Base Diameter	Ø3.8	Ø4.5	Ø5.5	Ø3.8	Ø4.5	Ø5.5
— Diameter Height —	Ø4.3	Ø5.0	Ø5.7	Ø4.3	Ø5.0	Ø5.7
11.4	2 SSFC 3835 C	2 SSFC 4535 C	2 SSFC 5535 C	2 SSFC 3835 B	2 SSFC 4535 B	2 SSFC 5535 B

- > Packing unit: 1 1 SFIT Meta G UCLA Cylinder.
- > Used over both SFIT S or A Ti-Base and SFIT Absolute S or A Ti-Base.
- > Modifiable to various types of abutments.
- > CCM alloy core for precise connection.
- > Cast with non-precious metal or gold alloy.
- > Connected with the SFIT S or A Ti-Base Screw and SFIT Absolute S or A Ti-Base Screw.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.


Component selection guide for the Sub. & Sub-N. Multi SFIT Ti-Base System

Intra-oral scanning procedure

Abutment





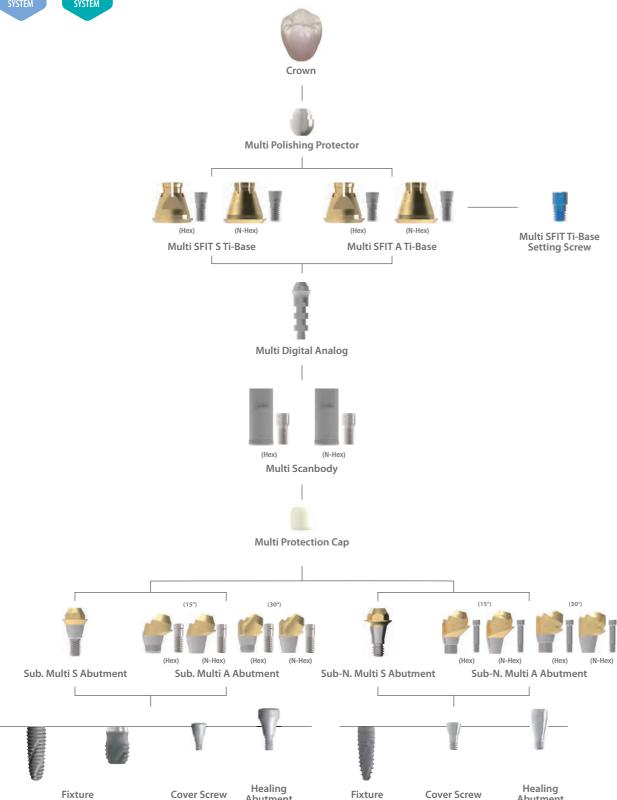


Component selection guide for the Sub. & Sub-N. Multi SFIT Ti-Base System

Model-scanning procedure






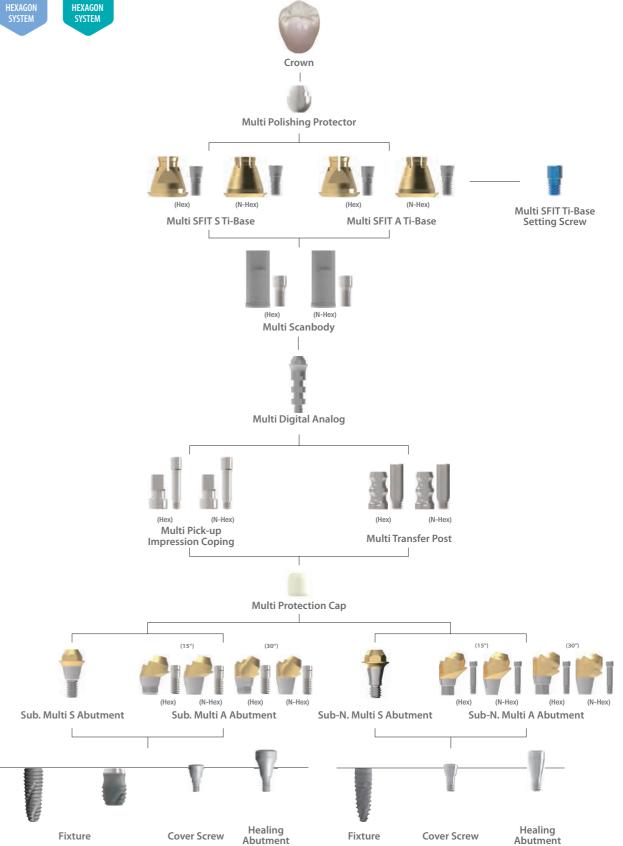

Multi S Abutment




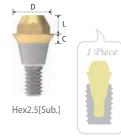


Fixture Connection Hex2.1[Sub-N.]		Hex2.5[Sub.]			
Platform[Fixture Dia.]	Ø4.5 [Ø3.1 / Ø3.3]	Ø4.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0]			
Diameter	Ø4.5	Ø4.5	Ø5.5		
Cuff Length	2	2	2		
1	SMS 451 N	2 SMS 451	2 SMS 551		
2	SMS452N	2 SMS 452	2 SMS 552		
3	SMS453N	2 SMS 453	2 SMS 553		
4	SMS454N	2 SMS 454	2 SMS 554		
5		2 SMS 455	2 SMS 555		

- > Packing unit: 1 Multi S Abutment.
- > For Screw-Retained Prosthesis.
- > Titanium base for the Multi SFIT S or A Ti-Base.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Integrated with the screw and abutment (solid screw).
- > Use the S Holder for more stable position.
- > Tightened with the S Machine or S Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the Multi Scanbody for digital flow.
- > Abutment level impression.

Multi A Abutment

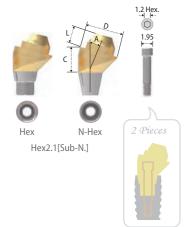

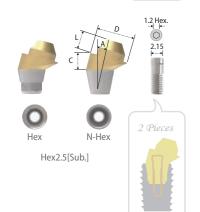


Туре		Hex					
Fixture Connection	Hex2.1	[Sub-N.]	Hex2.5[Sub.]				
Platform[Fixture Dia.]	Ø4.5 [Ø3.1 / Ø3.3]		Ø4.5 [Ø3.5 / Ø4.0 / Ø4	.5 / Ø5.0 / Ø5.5 / Ø6.0]	Ø5.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0]		
Diameter(Angle)	Ø4.5(15°)	Ø4.5(30°)	Ø4.5(15°)	Ø4.5(30°)	Ø5.5(15°)	Ø5.5(30°)	
Cuff Length	2	2	2	2	2	2	
2	★ SMAH 45152 BN		• 2 SMAH 45152				
3	• SMAH45153BN	★SMAH 45303 BN	★ 2 SMAH 45153	• 2 SMAH 45303	★ 2 SMAH 55153	★ 2 SMAH 55303	
4	• SMAH45154BN	● SMAH45304BN	★ 2 SMAH 45154	★ 2 SMAH 45304	★ 2 SMAH 55154	★ 2 SMAH 55304	
5					★ 2 SMAH 55155	★ 2 SMAH 55305	

TEAL (Jub 11.)	Туре	N-Hex						
Diameter(Angle) Ø4.5(15°) Ø4.5(30°) Ø4.5(15°) Ø4.5(30°) Ø5.5(15°) Ø5.5(30°) Cuff Length 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2 2	Fixture Connection	Hex2.1[Sub-N.]			Hex2.5[Sub.]			
Cuff 2 2 2 2 2 2 2 * SMAN45152NB * 2SMAN45152	Platform[Fixture Dia.]	Ø4.5 [Ø3.1 / Ø3.3]		Ø4.5 [Ø3.5 / Ø4.0 / Ø4	Ø4.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0] Ø		Ø5.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0]	
2 *SMAN45152NB	Diameter(Angle)	Ø4.5(15°)	Ø4.5(30°)	Ø4.5(15°)	Ø4.5(30°)	Ø5.5(15°)	Ø5.5(30°)	
	Cuff Length	2	2	2	2	2	2	
3 • SMAN45153NB *SMAN45303NB *2SMAN45153 • 2SMAN45303 *2SMAN55153 *2SMAN55303	2	★ SMAN 45152 NB		• 2 SMAN 45152				
	3	• SMAN45153NB	★ SMAN 45303 NB	★ 2 SMAN 45153	• 2 SMAN 45303	★ 2 SMAN 55153	★ 2 SMAN 55303	
4 • SMAN45154NB • SMAN45304NB * 2SMAN45154 * 2SMAN45304 * 2SMAN55154 * 2SMAN55304	4	• SMAN45154NB	• SMAN45304NB	★ 2 SMAN 45154	★ 2 SMAN 45304	★ 2 SMAN 55154	★ 2 SMAN 55304	
5 * 2SMAN55155 * 2SMAN55305	5					★ 2 SMAN 55155	★ 2SMAN55305	

- > Packing unit: 1 Multi A Abutment + 1 Abutment Screw.
- > For Screw-Retained Prosthesis.
- > Titanium base for the Multi SFIT S or A Ti-Base.
- > Gold color for more translucent restoration. > Library available for EXOCAD®, 3Shape® & others.
- > Use the A Holder for more stable position. > Connected with the Abutment Screw (SSHR200N: ★ SSHR300N: ● / 2SSHR300 : ★ 2SSHR400 : ●).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the Multi Scanbody for digital flow.
- > Abutment level impression.


Abutment Screw

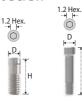

Height Diameter	8.7	9.3	7.5	6.5
1.95	★ SSHR200N	• SSHR300N		
2.15			★ 2 SSHR 300	• 2 SSHR 400

- > Packing unit: 1 Abutment Screw.
- > To connect the Multi A Abutment.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.


Multi Protection Cap

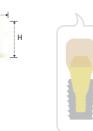

Multi S & A butment Diameter	Ø4.5	Ø5.5
— Diameter Height —	Ø5.2	Ø6.2
5	2 SMPC 45	2 SMPC 55

- > Packing unit: 1 Multi Protection Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Gingival retraction for prosthodontic margin of abutment.
- > Alternative usage for sub-structure of temporary prosthesis.


Multi Scanbody

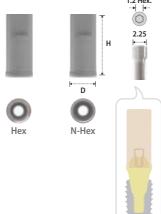

Туре	Hex	N-Hex
Multi S & A Abutment Diameter	Ø4.5 & Ø5.5	Ø4.5 & Ø5.5
Diameter Height	Ø4.5	Ø4.5
9	2 SMB 001 H	2 SMB 001 N

- > Packing unit: 1 Multi Scanbody + 1 Cylinder Screw. > For both intra-oral scanning and model-scanning.
- > Made of 100% titanium alloy with a special coating applied.
- > No need to spray.
- > Connected with the Cylinder Screw (2SMC100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.


Multi Pick-up Impression Coping

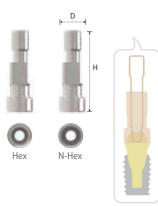

Туре	Hex		N-Hex	
Multi S & A Abutment Diameter	Ø4.5 Ø5.5		Ø4.5	Ø5.5
Diameter Height	Ø4.65	Ø5.65	Ø4.65	Ø5.65
9	2 SMIH 45	2 SMIH 55	2 SMIN 45	2 SMIN 55

- > Packing unit: 1 Multi Pick-up Impression Coping + 1 Guide Pin (2SMGP012).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Multi Transfer Post

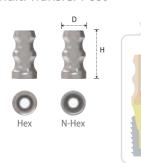

Туре	Hex		N-Hex	
Multi S & A Abutment Diameter	Ø4.5 Ø5.5		Ø4.5	Ø5.5
— Diameter Height —	Ø4.5	Ø5.5	Ø4.5	Ø5.5
7.5	2 SMTH 45	2 SMTH 55	2SMTN45	2 SMTN 55

- > Packing unit: 1 Multi Transfer Post + 1 Guide Pin (2SMTHS100).
- > For closed tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Multi Digital Analog



Multi S & A Abutment Diameter	Ø4.5	Ø5.5
Diameter Length	Ø4.5	Ø5.5
2	2 SMLA 45	2 SMLA 55

- > Packing unit: 1 Multi Digital Analog.
- > Replacement of the Multi S or A Abutment shape in working cast.
- > Used for both 3D printed model (RP) and stone model.
- > Select according to dimension of the Multi S or A Abutment.

Multi SFIT S Ti-Base





N-Hex

- N-Hex Hex Туре Multi S & A utment Diame Ø4.5 Ø4.5 Ø5.5 Ø4.5 Ø4.5 Ø5.5 Diameter Ø5.0 Ø5.3 Ø5.7 Ø5.0 Ø5.3 Ø5.7 3.5 3.5 3.5 3.5 0.5 2SMSFT45H 2SMSFT55H 2SMSFT45N 2SMSFT55N 1.5 2SMSFT40H 2SMSFT40N
- > Packing unit: 1 Multi SFIT S Ti-Base + 1 Multi SFIT S Ti-Base Screw.
- > Used over both Multi S or A Abutment.
- > For Screw Retained prosthesis.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Connected with the Multi SFIT S Ti-Base Screw.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Abutment level impression.

Multi SFIT A Ti-Base





)	Туре		Hex			N-Hex	
	Multi S & A Abutment Diameter	Ø4.5	Ø4.5	Ø5.5	Ø4.5	Ø4.5	Ø5.5
	Diameter	Ø5.0	Ø5.3	Ø5.7	Ø5.0	Ø5.3	Ø5.7
	Length Cuff	3.5	3.5	3.5	3.5	3.5	3.5
	0.5		2SMSFT45HA	2 SMSFT 55 HA		2SMSFT45NA	2 SMSFT 55 NA
	1.5	2SMSFT40HA			2SMSFT40NA		

- > Packing unit: 1 Multi SFIT A Ti-Base + 1 Multi SFIT A Ti-Base Screw.
- > Used over both Multi S or A Abutment.
- > For Screw Retained prosthesis.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Connected with the Multi SFIT A Ti-Base Screw.
- > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Abutment level impression.
- * Angulated Screw Ratchet Driver



Height Type	Ratchet
24(Short)	KRBUD15
29(Long)	KRBUD20

- > Stable to internal slip or fracture due to wide contact area of the Angulated Driver and the dedicated Stargrip Abutment Screw.
- > Tightening torque force : 30 N.cm (50 N.cm Max.)

Multi SFIT S Ti-Base Screw





Diameter Height	Ø2.3
5.5	2 SMCS 100 SF

- > Packing unit: 1 Multi SFIT S Ti-Base Screw.
- > To connect the Multi SFIT S Ti-Base.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.

Multi SFIT A Ti-Base Screw





Diameter Height	Ø2.3
5.5	2 SMCS 100 AF

- > Packing unit: 1 Multi SFIT A Ti-Base Screw.
- > To connect the Multi SFIT A Ti-Base.
- > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.

Multi SFIT Ti-Base Setting Screw



4.3	2 SMCSS 100
Diameter Height	Ø2.0


- > Packing unit: 5 Multi SFIT Ti-Base Setting Screws.
- > Used to screw the Multi SFIT S or A Ti-Base into the Multi Digital Analog or the Multi S & A Abutment when revising such prosthetic adaptions as shape, contact surface, occlusion and margin without giving a force to the head of the Multi SFIT S or A Ti-Base to spread and fit in the inner space of prosthesis.
- > Tightened with the 1.2 Hex Ratchet Driver.

Multi Polishing Protector




Туре	Hex	
Multi S & A Abutment Diameter	Ø4.5	Ø5.5
Diameter Length	Ø4.5	Ø5.5
2	2 SMPP 45	2 SMPP 55


- > Packing unit: 1 Multi Polishing Protector.
- > To polish work during lab procedure.

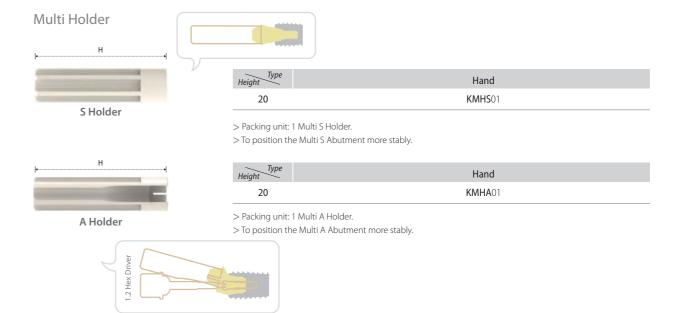



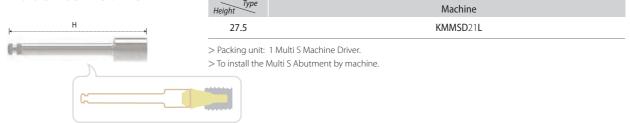

Multi S Ratchet Driver

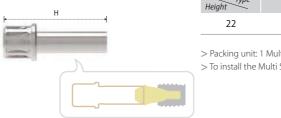

Type Height	Ratchet
22	KRMSD15L

- > Packing unit: 1 Multi S Ratchet Driver.
- > To install the Multi S Abutment by hand.

Multi SFIT Gono Gauge



Diameter	32		34
Ø5.5		2 SSFN	MGNG

- > Packing unit: 2 Multi SFIT Gono Gauge (1 40 Multi SFIT Gono Gauge, 1 45 & 55 Multi SFIT Gono Gauge).
- > Used to check whether the inner milling is done correctly when fabricating the crown.
- > 40 Multi SFIT Gono Gauge : Multi SFIT S Ti-Base with Cuff 1.5 (2SMSFT40H, 2SMSFT40N).
- > 45 & 55 Multi SFIT Gono Gauge: Multi SFIT S Ti-Base with Cuff 0.5 (2SMSFT45H, 2SMSFT55H, 2SMSFT45N, 2SMSFT55N).

When fastening in the G(Go) direction, the contact point with the crown must fit.




When tightening in the N(No) direction, the contact point with the crown must misfit.


* Refer to the SFIT Ti-Base System Catalog for details.

Component selection guide for the Int. SFIT Ti-Base System

 Intra-oral scanning Model-scanning



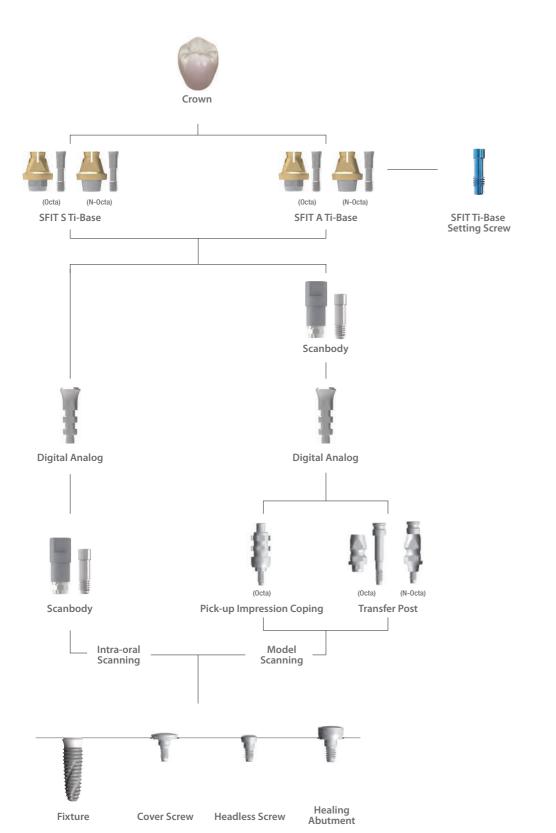

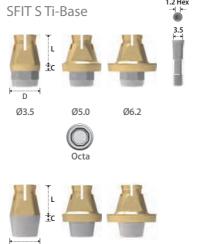

Ø5.0

N-Octa

Ø3.5

Platform [Fixture Dia.] Ø4.8 [Ø3.5/Ø4.0/Ø4.5] Ø4.8 [Ø3.5/Ø4.0/Ø4.5] Ø5.9 [Ø5.0/Ø6.0] Ø4.8 [Ø3.5/Ø4.0/Ø4.5] Ø4.8 [Ø3.5/Ø4.0/Ø4.5] Ø4.8 [Ø3.5/Ø4.0/Ø4.5] Ø5.9 [Ø5.0/Ø6.0] Diameter Ø3.5 Ø5.0 Ø6.2 Ø3.5 Ø5.0 Ø6.2 Cuff 3.6 3.6 3.6 3.6 3.6 3.6 1 ISFO35035 ISFN35035 ISFN50135 ISFN60135 ISFN60135 2 ISFO50235 ISFO60235 ISFN50235 ISFN60235 3 ISFO50335 ISFN60335 ISFN50335 ISFN60335	Туре	Octa			N-Octa		
Cuff 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 3.6 <td></td> <td></td> <td></td> <td>,</td> <td>,</td> <td></td> <td></td>				,	,		
0 ISFO35035 ISFO60135 ISFN60135 ISFN60135 2 ISFO50235 ISFO60235 ISFN60235	Diameter	Ø3.5	Ø5.0	Ø6.2	Ø3.5	Ø5.0	Ø6.2
1 ISFO50135 ISFO60135 ISFN50135 ISFN60135 2 ISFO50235 ISFO60235 ISFN50235 ISFN60235		3.6	3.6	3.6	3.6	3.6	3.6
2 ISFO50235 ISFO60235 ISFN50235 ISFN60235	0	ISFO 35035			ISFN35035		
	1		ISFO 50135	ISFO 60135		ISFN 50135	ISFN 60135
3 ISFO50335 ISFO60335 ISFN50335 ISFN60335	2		ISFO50235	ISFO60235		ISFN50235	ISFN60235
	3		ISFO50335	ISFO60335		ISFN50335	ISFN60335

- > Packing unit: 1 SFIT S Ti-Base + 1 SFIT S Ti-Base Screw. > For Screw Retained Prosthesis with straight screw channel.
 - > Gold color for more translucent restoration.
 - > Library available for EXOCAD®, 3Shape® & others.
 - > Connected with the SFIT S Ti-Base Screw.
 - > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
 - > Tightening torque force: 30 N.cm.
 - > Use the SFIT Scanbody for digital workflow.
 - > Fixture level impression.

SFIT A Ti-	Base		1.8 Stargrip
±C	4	A	3.5 ⊷
Ø3.5	Ø5.0	Ø6.2	
	Octa		
±c	4	A	
Ø3.5	Ø5.0	Ø6.2	<i>N</i>
	N-Octa		ulated Screw

Туре	Octa			N-Octa		
Platform [Fixture Dia.]	Ø4.8 [Ø3.5/Ø4.0/Ø4.5]	Ø4.8 [Ø3.5/Ø4.0/Ø4.5]	Ø5.9 [Ø5.0/Ø6.0]	Ø4.8 [Ø3.5/Ø4.0/Ø4.5]	Ø4.8 [Ø3.5/Ø4.0/Ø4.5]	Ø5.9 [Ø5.0/Ø6.0]
Diameter	Ø3.5	Ø5.0	Ø6.2	Ø3.5	Ø5.0	Ø6.2
Length Cuff	3.6	3.6	3.6	3.6	3.6	3.6
0	ISFO35035A			ISFN35035A		
1		ISFO50135A	ISFO60135A		ISFN50135A	ISFN60135A
2		ISFO50235A	ISFO60235A		ISFN50235A	ISFN60235A
3		ISFO50335A	ISFO60335A		ISFN50335A	ISFN60335A

- > Packing unit: 1 SFIT A Ti-Base + 1 SFIT A Ti-Base Screw.
- > For Screw Retained Prosthesis with angulated screw channel.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others. > Connected with the SFIT A Ti-Base Screw.
- > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the SFIT Scanbody for digital workflow.
- > Fixture level impression.
- * Angulated Screw Ratchet Driver



Height Type	Ratchet
24(Short)	KRBUD15
29(Long)	KRBUD20

- > Stable to internal slip or fracture due to wide contact area of the Angulated Driver and the dedicated Stargrip Abutment Screw.
- > Tightening torque force : 30 N.cm (50 N.cm Max.)

180 SFIT Ti-Base System: Int. SFIT Ti-Base SFIT Ti-Base System: Int. SFIT Ti-Base 181

SFIT S Ti-Base Screw



SFIT'S Ti-Base Cuff	0	1	2	3
Height Diameter	9	10.6	11.6	12.6
Ø2.3	ISFS00S	ISFS10S	ISFS20S	ISFS30S

- > Packing unit: 1 SFIT S Ti-Base Screw.
- > Used to connect the SFIT S Ti-Base.
- ISFS00S: SFIT S Ti-Base with Cuff 0, ISFS10S: SFIT S Ti-Base with Cuff 1, ISFS20S: SFIT S Ti-Base with Cuff 2, ISFS30S: SFIT S Ti-Base with Cuff 3.
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.


SFIT A Ti-Base Screw



SFIT'S Ti-Base Cuff	0	1	2	3
Height Diameter	9	10.6	11.6	12.6
Ø2.3	ISFS00A	ISFS10A	ISFS20A	ISFS30A

- > Packing unit: 1 SFIT A Ti-Base Screw.
- > Used to connect the SFIT A Ti-Base.
- ISFS00A: SFIT A Ti-Base with Cuff 0, ISFS10A: SFIT A Ti-Base with Cuff 1, ISFS20A: SFIT A Ti-Base with Cuff 2, ISFS30A: SFIT A Ti-Base with Cuff 3.
- > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.


Scanbody

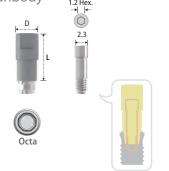

Tuno	Octa
Туре	OCIa
Platform [Fixture Dia.]	Ø4.8 & Ø5.9 [Ø3.5 / Ø4.0 / Ø4.5, Ø5.0 / Ø6.0]
Diameter Length	Ø4.5
6	ISB406
10	ISB 410

- > Packing unit: 1 Scanbody + 1 Abutment Screw (ISHR110).
- > For both intra-oral scanning and model-scanning.
- > Made of 100% titanium alloy with a special coating applied.
- > No need to spray.
- > Connected with the Abutment Screw (ISHR110).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.


Pick-up Impression Coping

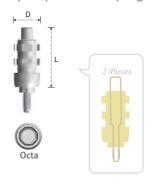

Туре	Octa		
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	
— Diameter Length —	Ø5.5	Ø6.6	
13.7	IIOR001	IIOW 001	

- > Packing unit: 1 Pick-up Impression Coping + 1 Guide Pin (IIOR001S).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Transfer Post

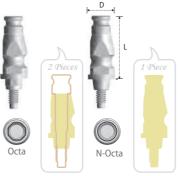

Туре	Octa		N-O	cta
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Length	Ø4.8	Ø5.9	Ø4.8	Ø5.9
11.6	ITOR400	ITOW500	ITNR400	ITNW500

- > Packing unit: Octa 1 Transfer Post + 1 Guide Pin (Regular: ITOR400S / Wide: ITOW500S) N-Octa - 1 Transfer Post (solid screw).
- > For closed tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Digital Analog



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
— Diameter Height —	Ø4.8	Ø5.9
13.5	IDR001R	IDR001W

- > Packing unit: 1 Digital Analog.
- > Analog of fixture for working cast.
- > Used for both 3D printed model (RP) and stone model.
- > Select according to fixture platform.

SFIT Ti-Base Setting Screw



	Ø1.95	
7.5	ISFSS100	


- > Packing unit: 5 SFIT Ti-Base Setting Screws.
- > Used to screw the SFIT S or A Ti-Base into the Digital Analog or the Fixture when revising such prosthetic adaptions as shape, contact surface, occlusion and margin without giving a force to the head of the SFIT Ti-Base S or A Ti-Base to spread and fit in the inner space of the prosthesis.
- > Tightened with the 1.2 Hex Ratchet Driver.

182 SFIT Ti-Base System: Int. SFIT Ti-Base


Component selection guide for the Sub. Hybrid Ti-Base System

- Intra-oral scanning
- Model-scanning



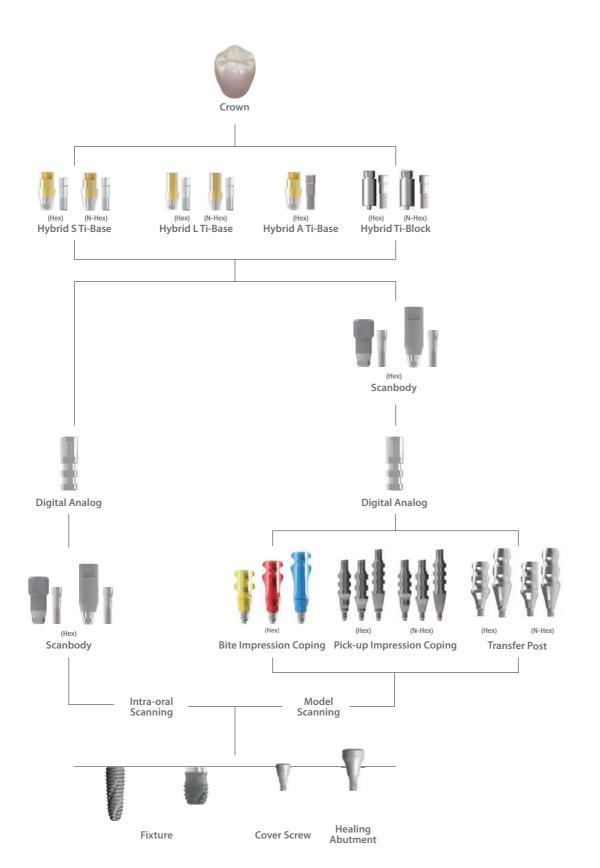

Hybrid S Ti-Base

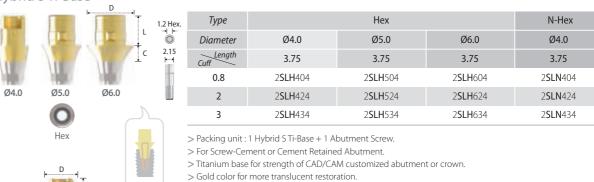

> Lingual surface hole for more esthetic restoration (Ø4.0).

> Tightened with the 1.2 Hex Driver and Torque Wrench.

> Library available for EXOCAD®, 3Shape® & Others. > Connected with the Abutment Screw (2SSHR200)

> Tightening torque force: 30 N.cm.

> Use the Scanbody for 3D Work.


> Fixture level impression

> Right angled (Ø4.0) and humped design (Ø5.0, Ø6.0) for anti-rotation of prosthesis.

Hybrid L Ti-Base

Ø4.0

0

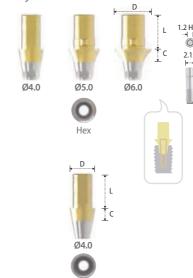

Diameter	040			
	Ø4.0	Ø5.0	Ø6.0	Ø4.0
Length Cuff	5.5	5.5	5.5	5.5
1	2 SLH 415	2 SLH 515	2 SLH 615	2 SLN 415
2	2 SLH 425	2 SLH 525	2 SLH 625	2 SLN 425
3	2 SLH 435	2 SLH 535	2 SLH 635	2 SLN 435

- > Packing unit: 1 Hybrid L Ti-Base + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Abutment.
- > Titanium base for strength of CAD/CAM customized abutment or crown.
- > Gold color for more translucent restoration. > Cutting surface (Ø4.0) and humped design (Ø5.0, Ø6.0) for anti-rotation of prosthesis.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Connected with the Abutment Screw (2SSHR200).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Fixture level impression.


Ø6.5

2**SIN**654**S**

2**SIN**65**S**

2**SIN**65L

Hybrid A Ti-Base

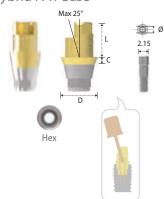

Туре	Hex
Diameter	Ø4.0
Length Cuff	3.75
0.8	2 SLH 404 A
2	2 SLH 424 A
3	2 SLH 434 A

- > Packing unit: 1 Hybrid A Ti-Base + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Abutment.
- > Titanium base for strength of CAD/CAM customized abutment or crown.
- > For Fabrication of Angulated Screw Channel up to 25°.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Connected with the Stargrip Abutment Screw (2SLAH100, 2SLAH200 & 2SLAH300).
- > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Fixture level impression.
- * Angulated Screw Ratchet Driver



Height Type	Ratchet
24(Short)	KRBUD15
29(Long)	KRBUD20

- > Stable to internal slip or fracture due to wide contact area of the Angulated Driver and the dedicated Stargrip Abutment Screw.
- > Tightening torque force : 30 N.cm (50 N.cm Max.).


Hybrid Ti-Block



Туре	Hex			N-Hex		
<u>Diameter</u> <u>Length</u>	10	12	14	10	12	14
20	CSHH10S	CSHH12S	CSHH14S	CSHN10S	CSHN12S	CSHN14S

- > Packing unit: 1 Hybrid Ti-Block + 2 Abutment Screws.
- > For Screw-Cement or Cement Retained Abutment.
- > Block abutment for CAD/CAM customized abutment.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Connected with the Abutment Screw (2SSHR100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Fixture level impression.

Scanbody

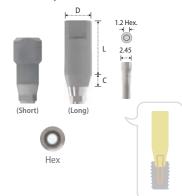

Туре	Hex(Short)	Hex(Long)
Diameter	Ø4.3	Ø4.3
Length Cuff	4	8
2		2 SSB 4329
3	2 SSB 4325	

- > Packing unit: 1 Scanbody + 1 Abutment Screw.
- > For both, model scanner and intra oral scanner.
- > For Hybrid S & L Ti-Base and Hybrid A Ti-Block.
- > Made of 100% titanium alloy with a special coating applied.
- > Connected with the Abutment Screw (2SSHR100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 15~20 N.cm.


Bite Impression Coping




Туре	Hex(Short)	Hex(Long)	Hex(X-Long)
Diameter	Ø4.5	Ø4.5	Ø4.5
Cuff Length	2	4	6
4.0	2 SBIC 45 S	2 SBIC 45L	2 SBIC 45 X

- > Packing unit: 1 Bite Impression Coping (Inbuilt Guide Pin).
- > Designed to simultaneously take bite and impression.
- > For Closed Tray Impression (Bite Impression).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Pick-up Impression Coping

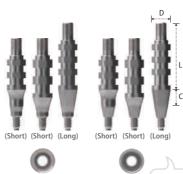


N-Hex

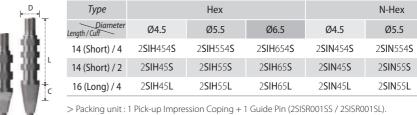

- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Transfer Post

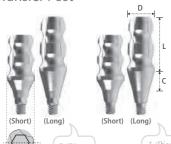

Туре	Hex			N-Hex		
Diameter Length/Cuff	Ø4.5	Ø5.5	Ø6.5	Ø4.5	Ø5.5	Ø6.5
9 (Short) / 2	2 STH 45 S	2 STH 55 S	2 STH 65 S	2 STN 45 S	2 STN 55 S	2 STN 65 S
11 (Long) / 4	2 STH 45 L	2 STH 55 L	2STH65L	2 STN 45 L	2 STN 55 L	2STN65L

- > Packing unit: 1 Transfer Post + 1 Guide Pin (Hex) / 1 Transfer Post (N-Hex, Solid Type).
- > For closed tray impression.
- > Connected with the Guide Pin (2STH001SS / 2STH001SL).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Digital Analog



Diameter Height	Ø3.9
12	2 SDR 001

- > Packing unit: 1 Digital Analog.
- > Analog of fixture for working cast.
- > Used for both 3D printed model (RP) and stone model.

Abutment Screw



Diameter Height	Ø2.45	Ø2.15
8.5	2 SSHR 100	2 SSHR 200

- > Packing unit : 1 Abutment Screw.
- > 2SSHR100 : Hybrid Block / Scanbody.
- > 2SSHR200 : Hybrid S & L Ti-Base.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

A Abutment Screw



Height Diameter	2	3.2	4.2
Ø2.15	2 SLAH 100	2 SLAH 200	2 SLAH 300

- > Packing unit: 1 A Abutment Screw.
- > Exclusive for Hybrid A Ti-Base (2SLAH100 for 2SLH404A, 2SLAH200 for 2SLH424A & 2SLAH300 for 2SLH434A).
- > Tightened with the Angulated Driver and Torque Wrench.

Component selection guide for the Sub. & Sub-N. Multi Hybrid Ti-Base System

- Intra-oral scanning
- Model-scanning

Healing







Multi Polishing Protector














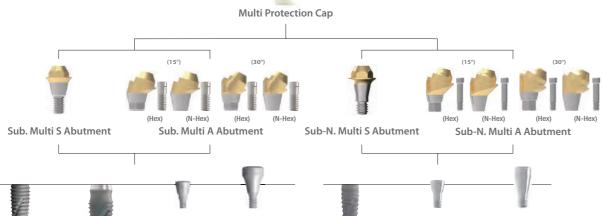

Healing

Multi S Abutment







Fixture Connection	Hex2.1[Sub-N.]	Hex2.5[Sub.]		
Platform[Fixture Dia.]	Ø4.5 [Ø3.1 / Ø3.3]	Ø4.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0]		
Diameter	Ø4.5	Ø4.5	Ø5.5	
Cuff Length	2	2	2	
1	SMS 451 N	2 SMS 451	2 SMS 551	
2	SMS452N	2 SMS 452	2 SMS 552	
3	SMS453N	2 SMS 453	2 SMS 553	
4	SMS454N	2 SMS 454	2 SMS 554	
5		2 SMS 455	2 SMS 555	

- > Packing unit: 1 Multi S Abutment.
- > For Screw-Retained Prosthesis.
- > Titanium base for the Multi Hybrid Ti-Base.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Integrated with the screw and abutment (solid screw).
- > Use the S Holder for more stable position.
- > Tightened with the S Machine or S Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the Multi Scanbody for digital flow.
- > Abutment level impression.

Multi A Abutment

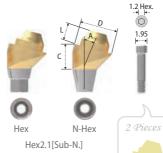



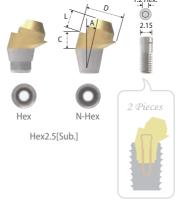

Туре	Hex					
Fixture Connection	Hex2.1	[Sub-N.]	Hex2.5[Sub.]			
Platform[Fixture Dia.]	Ø4.5 [Ø3.1 / Ø3.3]		Ø4.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0]		Ø5.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0]	
Diameter(Angle)	Ø4.5(15°)	Ø4.5(30°)	Ø4.5(15°)	Ø4.5(30°)	Ø5.5(15°)	Ø5.5(30°)
Cuff Length	2	2	2	2	2	2
2	★ SMAH 45152 BN		• 2 SMAH 45152			
3	• SMAH45153BN	★SMAH 45303 BN	★ 2 SMAH 45153	• 2 SMAH 45303	★ 2 SMAH 55153	★ 2 SMAH 55303
4	• SMAH45154BN	● SMAH45304BN	★ 2 SMAH 45154	★ 2 SMAH 45304	★ 2 SMAH 55154	★ 2 SMAH 55304
5					★ 2 SMAH 55155	★ 2 SMAH 55305
T						
Туре	N-Hex					

Type			IN-I ICX			
Fixture Connection	Hex2.1[Sub-N.]		Hex2.5[Sub.]			
Platform[Fixture Dia.]	Ø4.5 [Ø3	3.1 / Ø3.3]	Ø4.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0]		Ø5.5 [Ø3.5 / Ø4.0 / Ø4.5 / Ø5.0 / Ø5.5 / Ø6.0]	
Diameter(Angle)	Ø4.5(15°)	Ø4.5(30°)	Ø4.5(15°)	Ø4.5(30°)	Ø5.5(15°)	Ø5.5(30°)
Cuff Length	2	2	2	2	2	2
2	★ SMAN45152NB		• 2 SMAN 45152			
3	• SMAN45153NB	★SMAN45303NB	★ 2 SMAN 45153	• 2 SMAN 45303	★ 2 SMAN 55153	★ 2 SMAN 55303
4	• SMAN45154NB	• SMAN45304NB	★ 2 SMAN 45154	★ 2 SMAN 45304	★ 2 SMAN 55154	★ 2 SMAN 55304
5					★ 2 SMAN 55155	★ 2 SMAN 55305

- > Packing unit: 1 Multi A Abutment + 1 Abutment Screw.
- > For Screw-Retained Prosthesis.
- > Titanium base for the Multi Hybrid Ti-Base.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others. > Use the A Holder for more stable position.
- > Connected with the Abutment Screw (SSHR200N: ★ SSHR300N: / 2SSHR300 : ★ 2SSHR400 : ●).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the Multi Scanbody for digital flow.
- > Abutment level impression.



Height	8.7	9.3	7.5	6.5
1.95	★ SSHR200N	• SSHR300N		
2.15			★ 2 SSHR 300	• 2 SSHR 400

- > Packing unit: 1 Abutment Screw.
- > To connect the Multi A Abutment.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Multi Protection Cap




	Abu
	Н
7	> [
k	> [
E	> (

Multi S & A Abutment Diameter	Ø4.5	Ø5.5
Diameter Height	Ø5.2	Ø6.2
5	2 SMPC 45	2 SMPC 55

- Packing unit: 1 Multi Protection Cap.
- Protection from cheek and tongue for gingival healing period.
- Gingival retraction for prosthodontic margin of abutment.
- > Alternative usage for sub-structure of temporary prosthesis.

Multi Scanbody

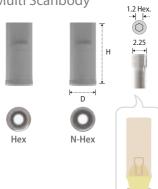

Туре	Hex	N-Hex
Multi S & A Abutment Diameter	Ø4.5 & Ø5.5	Ø4.5 & Ø5.5
Diameter Height	Ø4.5	Ø4.5
9	2 SMB 001 H	2 SMB 001 N

- > Packing unit: 1 Scanbody + 1 Cylinder Screw.
- > For both, model scanner and intra oral scanner.
- > For the Multi Hybrid Ti-Base.
- > Made of 100% titanium alloy with a special coating applied.
- > No need to spray.
- > Connected with the Cylinder Screw (2SMCS100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 15~20 N.cm.

Multi Pick-up Impression Coping



Туре	He	ex	N-Hex		
Multi S & A Abutment Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5	
Diameter Height	Ø4.65	Ø5.65	Ø4.65	Ø5.65	
9	2 SMIH 45	2 SMIH 55	2 SMIN 45	2 SMIN 55	

- > Packing unit: 1 Multi Pick-up Impression Coping + 1 Guide Pin (2SMGP012).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.


Multi Transfer Post









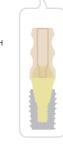

		2	
	1		
	-		

Туре	He	ex ex	N-H	ex
Multi S & A Abutment Diameter	Ø4.5	Ø5.5	Ø4.5	Ø5.5
Diameter Height	Ø4.5	Ø5.5	Ø4.5	Ø5.5
7.5	2 SMTH 45	2 SMTH 55	2 SMTN 45	2 SMTN 55

- > Packing unit: 1 Multi Transfer Post + 1 Guide Pin (2SMTHS100).
- > For closed tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Multi Digital Analog



Multi S & A Abutment Diameter	Ø4.5	Ø5.5
	Ø4.5	Ø5.5
2	2 SMLA 45	2 SMLA 55

- > Packing unit: 1 Multi Digital Analog.
- > Replacement of the Multi S or A Abutment shape in working cast.
- > Used for both 3D printed model (RP) and stone model.
- > Select according to dimension of the Multi S or A Abutment.

Multi Hybrid Ti-Base







Туре		Hex			N-Hex	
Multi S & A Abutment Diameter	Ø4.5	Ø4.5	Ø5.5	Ø4.5	Ø4.5	Ø5.5
Diameter	Ø4.5	Ø4.5	Ø5.5	Ø4.5	Ø4.5	Ø5.5
Length	4.5	4.5	4.5	4.5	4.5	4.5
0.5		2 SMHT 45 H	2 SMHT 55 H		2 SMHT 45 N	2 SMHT 55 N
1.5	2 SMHT 40 H			2SMHT40N		

- > Packing unit: 1 Multi Hybrid Ti-Base + 1 Cylinder Screw.
- > For Screw-Cement or Cement Retained Abutment.
- > Titanium base for strength of CAD/CAM customized abutment or crown.
- > Gold color for more translucent restoration.
- > Cutting surface for anti-rotation of prosthesis. > Library available for EXOCAD®, 3Shape® & Others.
- > Connected with the Cylinder Screw (2SMCS100).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Abutment level impression.

Multi Cylinder Screw




Diameter Height	Ø2.25
5	2 SMCS 100

- > Packing unit : 1 Multi Cylinder Screw.
- > Connected with the Multi Scanbody, Multi Hybrid Ti-Base.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

Component selection guide for the Sub. Lock Hybrid Ti-Base System

- Intra-oral scanning
- Model-scanning



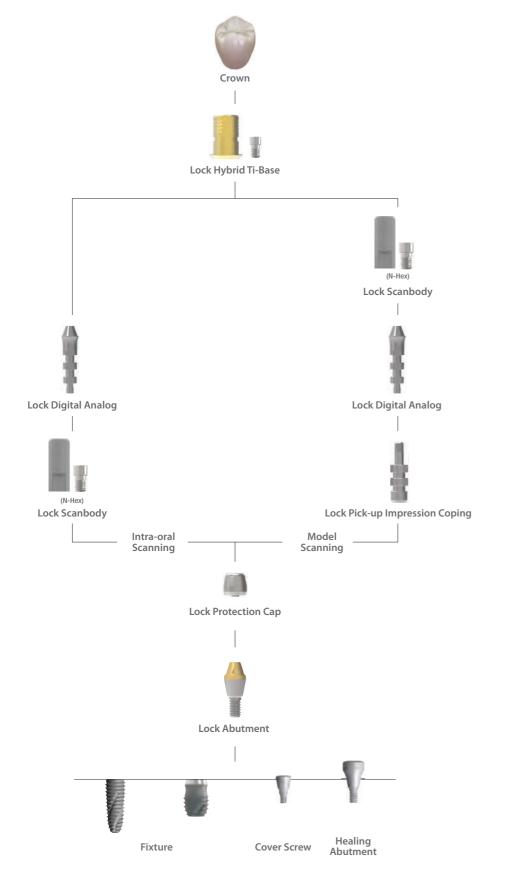

Lock Abutment



Diameter	Ø4.5
Cuff Length	2.15
0.5	2 SLA 400
1	2 SLA 410
2	2 SLA 420
3	2 SLA 430
4	2 SLA 440

- > Packing unit: 1 Lock Abutment.
- > For Screw-Retained Prosthesis.
- > Titanium base for Lock Hybrid Ti-Base.
- > Gold color for more translucent restoration.
- > Integrated with screw and abutment.
- > Tightened with the Lock Ratchet Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Abutment level impression.


Lock Protection Cap



Diameter Height	Ø4.5
4.3	2 SLP 45

- > Packing unit: 1 Lock Protection Cap.
- > Protection from cheek and tongue for gingival healing period.
- > Gingival retraction for prosthodontic margin of abutment.


Lock Scanbody

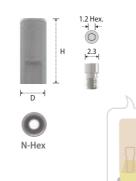

Diameter Height	Ø4.5
9	2 SLB 001 H

- > Packing unit: 1 Scanbody + 1 Cylinder Screw.
- > For both, model scanner and intra oral scanner.
- > For the Lock Hybrid Ti-Base.
- > Made of 100% titanium alloy with a special coating applied.
- > No need to spray.
- > Connected with the Cylinder Screw (2SLCS200).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 15~20 N.cm.

Lock Pick-up Impression Coping

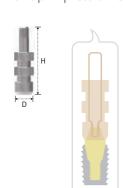

Diameter Height	Ø4.5	
16	2 SLIH 45	

- > Packing unit : 1 Lock Pick-up Impression Coping + 1 Guide Pin (2SLIH45S).
- > For open tray impression.

Lock Digital Analog




Diameter Length	Ø3.5	
2.2	2 SLLA 35	

- > Packing unit: 1 Lock Digital Analog.
- > Used for both 3D printed model (RP) and stone model.

Lock Hybrid Ti-Base



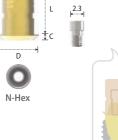

Diameter	Ø4.5
Length Cuff	4.5
0.5	2SLHT40N


- > Packing unit: 1 Lock Hybrid Ti-Base + 1 Cylinder Screw.
- > For Screw-Cement or Cement Retained Abutment.
- > Titanium base for strength of CAD/CAM customized abutment or crown.
- > Gold color for more translucent restoration.
- > Cutting surface for anti-rotation of prosthesis.
- > Library available for EXOCAD®, 3Shape® & Others.
- > Connected with the Cylinder Screw (2SLCS200). > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Abutment level impression.

Lock Cylinder Screw



Diameter Height	Ø2.3
4.8	2 SLCS 200


- > Packing unit: 1 Cylinder Screw.
- > Connected with the Lock Scanbody, Lock Hybrid Ti-Base.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

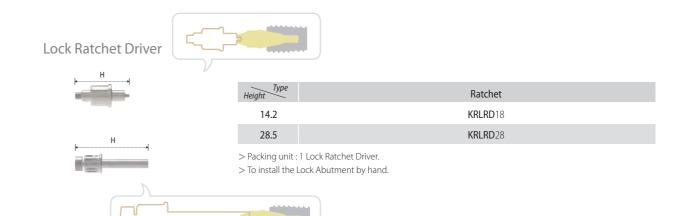


Component selection guide for the Sub-N. Hybrid Ti-Base System

 Intra-oral scanning Model-scanning



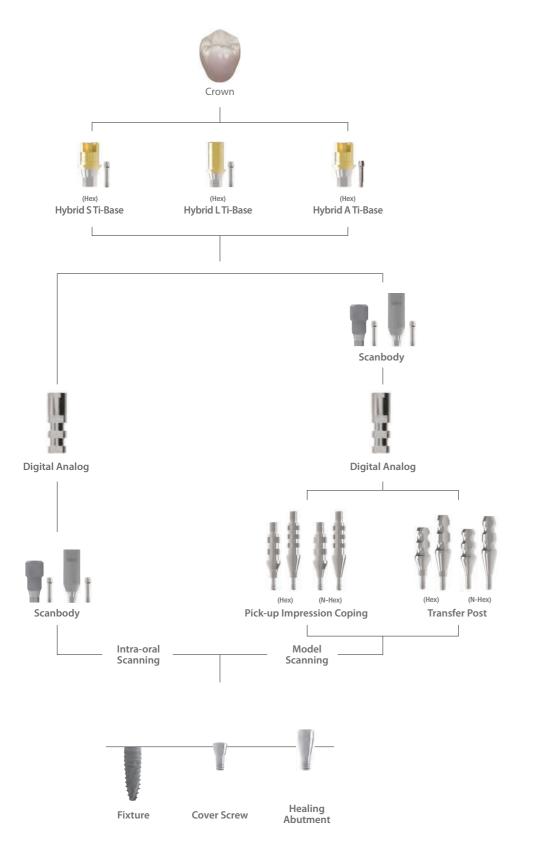


Hybrid S Ti-Base

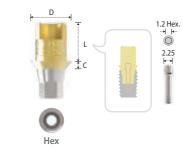

Туре	Hex
Diameter	Ø4.0
Length Cuff	3.75
0.8	SLH404N
2	SLH424N
3	SLH434N

- > Packing unit: 1 Hybrid STi-Base + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Abutment.
- > Titanium base for strength of CAD/CAM customized abutment or crown.
- > Gold color for more translucent restoration.
- > Lingual surface hole for more esthetic restoration. > Right angled for anti-rotation of prosthesis.
- > Library available for EXOCAD®, 3Shape® & Others. > Connected with the Abutment Screw (SSHR100N).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Fixture level impression.


Hybrid L Ti-Base

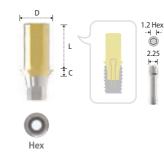

Туре	Hex
Diameter	Ø4.0
Length Cuff	5.5
1	SLH415N
2	SLH425N
3	SLH435N

- > Packing unit: 1 Hybrid L Ti-Base + 1 Abutment Screw. > Library available for EXOCAD®, 3Shape® & Others.
- > For Screw-Cement or Cement Retained Abutment.
- > Titanium base for strength of CAD/CAM customized
- > Gold color for more translucent restoration.
- > Cutting surface for anti-rotation of prosthesis.
- > Connected with the Abutment Screw (SSHR100N).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.
- > Use the Scanbody for 3D Work.
- > Fixture level impression.

Hybrid A Ti-Base

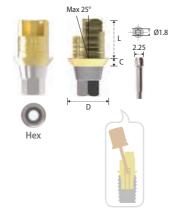

Туре	Hex
Diameter	Ø4.0
Length Cuff	3.75
0.8	SLH404AN
2	SLH424AN
3	SLH434AN

- > For Screw-Cement or Cement Retained Abutment.
- > Titanium base for strength of CAD/CAM customized abutment or crown.
- > For Fabrication of Angulated Screw Channel up to 25°.
- > Packing unit: 1 Hybrid A Ti-Base + 1 Abutment Screw. > Library available for EXOCAD®, 3Shape® & Others. > Connected with the Stargrip Abutment Screw
 - (SLAH100N, SLAH200N & SLAH300N).
 - > Tightened with the Angulated Screw Ratchet Driver and Torque Wrench.
 - > Tightening torque force : 30 N.cm.
 - > Use the Scanbody for 3D Work.
 - > Fixture level impression.
- * Angulated Screw Ratchet Driver



Height Type	Ratchet
24(Short)	KRBUD15
29(Long)	KRBUD20

- > Stable to internal slip or fracture due to wide contact area of the Angulated Driver and the dedicated Stargrip Abutment Screw.
- > Tightening torque force : 30 N.cm (50 N.cm Max.).


Scanbody

0

Туре Hex(Short) Hex(Long) Diameter Ø4.3 Ø4.3 2 SSB4329N SSB4325N

- > Packing unit: 1 Scanbody + 1 Abutment Screw.
- > For both, model scanner and intra oral scanner.
- > For the Hybrid S and L Ti-Base.
- > Made of 100% titanium alloy with a special coating applied.
- > No need to spray.
- > Connected with the Abutment Screw (SSHR100N).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 15~20 N.cm.


Pick-up Impression Coping

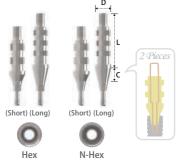

Туре	Hex	N-Hex
Diameter Length/Cuff	Ø4.5	Ø4.5
14 (Short) / 2	SIH45SN	SIN45SN
16 (Long) / 4	SIH45LN	SIN45LN

- > Packing unit : 1 Pick-up Impression Coping + 1 Guide Pin.
- > For open tray impression.
- > Connected with the Guide Pin (SIS001SN / SIS001LN).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Transfer Post

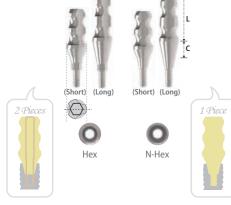

Туре	Hex	N-Hex
Diameter Length/Cuff	Ø4.5	Ø4.5
9 (Short) / 2	STH45SN	STN45SN
11 (Long) / 4	STH45LN	STN45LN

- > Packing unit: 1 Transfer Post + 1 Guide Pin (Hex) / 1 Transfer Post (N-Hex, Solid).
- > For closed tray impression.
- > Connected with the Guide Pin (STS001SN / STS001LN).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.

Digital Analog



Diameter Height	Ø3.9
12	SDR001N

- > Packing unit: 1 Digital Analog.
- > Analog of fixture for working cast.
- > Used for both 3D printed model (RP) and stone model.

Abutment Screw



Diameter Height	Ø2.25	
10.2	SSHR100N	

- > Packing unit: 1 Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

A Abutment Screw




Diameter	10.2	11.4	12.4
Ø2.25	SLAH100N	SLAH200N	SLAH300N

- > Packing unit: 1 A Abutment Screw.
- > Exclusive for Hybrid A Ti-Base (SLAH100N for SLH404AN, SLAH200N for SLH424AN & SLAH300N for SLH434AN).
- > Tightened with the Angulated Driver and Torque Wrench.


Component selection guide for the Int. Hybrid Ti-Base System


 Intra-oral scanning Model-scanning



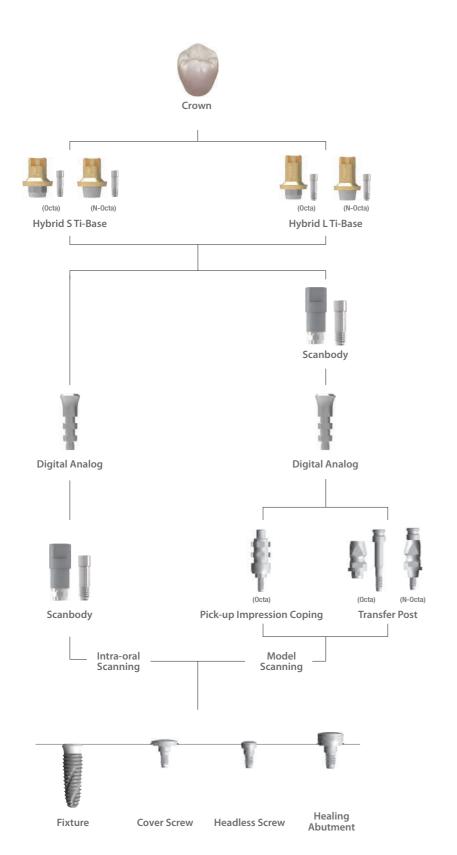

Hybrid S Ti-Base

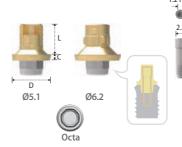


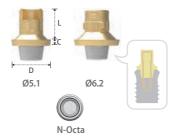


ex					
)	Туре	Octa		N-Octa	
3	Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
	Diameter	Ø5.1	Ø6.2	Ø5.1	Ø6.2
	Length Cuff	4	4	4	4
	0.8	ILO 4814	ILO 5914	ILN 4814	ILN 5914
	2	ILO4824	ILO 5924	ILN4824	ILN5924
	3	ILO4834	ILO 5934	ILN4834	ILN5934

- > Packing unit: 1 Hybrid S Ti-Base + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Abutment.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Connected with the Abutment Screw (ILHS100).
- > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench. > Tightening torque force: 30 N.cm.
- > Use the Scanbody for digital workflow.
- > Fixture level impression.

Hybrid LTi-Base

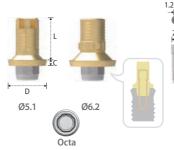

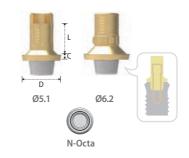


Туре	Type Octa		N-Octa	
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter	Ø5.1	Ø6.2	Ø5.1	Ø6.2
Length Cuff	5.5	5.5	5.5	5.5
0.8	ILO 4815	ILO 5915	ILN4815	ILN 5915
2	ILO4825	ILO 5925	ILN4825	ILN5925
3	ILO4835	ILO5935	ILN4835	ILN 5935

- > Packing unit: 1 Hybrid L Ti-Base + 1 Abutment Screw.
- > For Screw-Cement or Cement Retained Abutment.
- > Gold color for more translucent restoration.
- > Library available for EXOCAD®, 3Shape® & others.
- > Connected with the Abutment Screw (ILHS100). > Tightened with the 1.2 Hex Ratchet Driver and Torque Wrench.
- > Tightening torque force: 30 N.cm.
- > Use the Scanbody for digital workflow.
- > Fixture level impression.

Туре	Octa
Platform [Fixture Dia.]	Ø4.8 & Ø5.9 [Ø3.5 / Ø4.0 / Ø4.5, Ø5.0 / Ø6.0]
Diameter Length	Ø4.5
6	ISB406
10	ISB 410

- > Packing unit: 1 Scanbody + 1 Abutment Screw (ISHR110).
- > For both intra-oral scanning and model-scanning.
- > Made of 100% titanium alloy with a special coating applied.
- > No need to spray.
- > Connected with the Abutment Screw (ISHR110).
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.


Pick-up Impression Coping

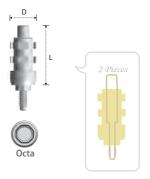

Туре	Octa		
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	
Diameter Length	Ø5.5	Ø6.6	
13.7	IIOR001	IIOW001	

- > Packing unit: 1 Pick-up Impression Coping + 1 Guide Pin (IIOR001S).
- > For open tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Transfer Post

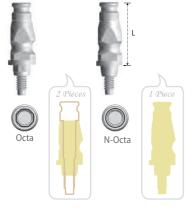

Туре	Octa		N-O	cta	
Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]	
Diameter Length	Ø4.8	Ø5.9	Ø4.8	Ø5.9	
11.6	ITOR400	ITOW500	ITNR400	ITNW500	

> Packing unit: Octa - 1 Transfer Post + 1 Guide Pin (Regular: ITOR400S / Wide: ITOW500S) N-Octa - 1 Transfer Post (solid screw).

- > For closed tray impression.
- > Connected with the Guide Pin.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force: 12~15 N.cm.

Digital Analog



Platform [Fixture Dia.]	Ø4.8 [Ø3.5 / Ø4.0 / Ø4.5]	Ø5.9 [Ø5.0 / Ø6.0]
Diameter Height	Ø4.8	Ø5.9
13.5	IDR001R	IDR001W

- > Packing unit: 1 Digital Analog.
- > Analog of fixture for working cast.
- > Used for both 3D printed model (RP) and stone model.
- > Select according to fixture platform.

Abutment Screw



Diameter Height	Ø2.3	
8.6	ILHS100	

- > Packing unit: 1 Abutment Screw.
- > Tightened with the 1.2 Hex Driver and Torque Wrench.
- > Tightening torque force : 30 N.cm.

COWELL® EXPERT INSTRUMENTS

An Expert knows what makes the results



Designed to perform maxillary sinus lifting. Aqua Membrane Lifter, Drill designs and Stopper System prevent perforation of sinus membrane. The kit includes all the instruments required for both crestal and lateral approach.

Sinus Lift Kit

This revolutionary kit contains US Patented Modified Tap Drills and Spreaders in order to allow any dentists to easily lift, split or condense surrounding bone with simple drilling. Dentists can expect more predictable results, and patients can enjoy less traumatic surgeries with shorter chair time.

MFR Kit (Multi-Functional Removal™ Kit)

An ideal solution for fixture, abutment and screw removal without trauma and bone loss.

InnoGenic™ GBR Kit

An all-in-one solution for various types of GBR procedures.

InnoGenic™ Autobone Harvester

Devised to harvest autogenous bone not only from the general site but also from the site where the implant will be placed. More than 1cc of bone chips can be harvested within 10 seconds.

COWELL® BMP Trephine Kit

An easy-to-use kit with drills and instruments for block-type bone collection, failed fixture removal, crestal & window approach for sinus lift and bone chip extraction.











Atraumatic Extraction Kit

Used for the immediate and effortless extraction of the root of the tooth with simple procedures.

Direct Surgical Guide Kit

Used for a flapless surgery. It can also serve as a guide for positioning the drill while measuring the thickness of the buccal bone, thereby preventing bone resorption and reducing the burden on the patient.

AO4 Surgical Stent

An excellent guide template to place implant precisely, especially for AO4 or AO6 technique.

Volume-up™ Guide System


Devised for preventing food penetration and forming natural cervical area by restoring contracted buccal alveolar bone & gingiva to the original shape and width.




Multi-Functional Sinus™ Kit

MFS KIT [KSA004]

> A comprehensive kit to approach direct & indirect maxillary sinus lift simply.

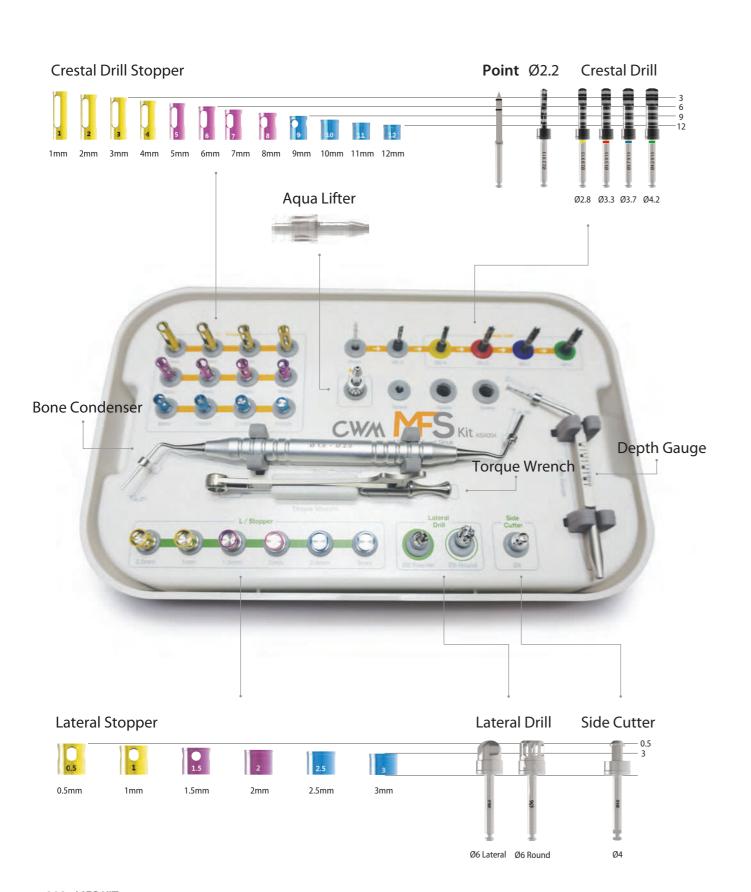

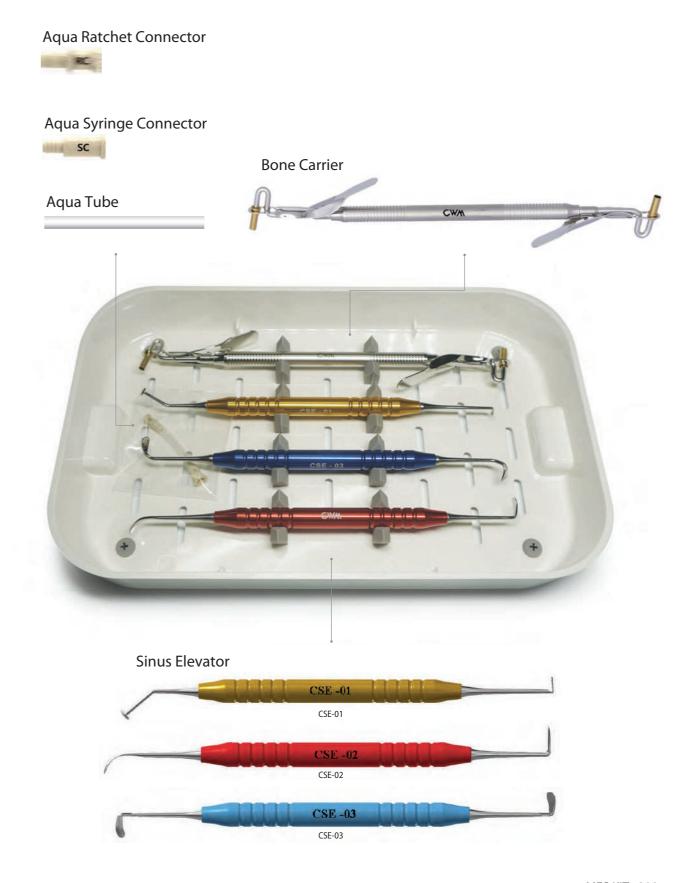


Crestal Approach - Components

1. Point Drill 800~1,000 rpm

- > Use to mark the point of perforation on cortical bone.
- > In the case of the remaining bone height is as low as 3.5mm, pay more attention when drilling.



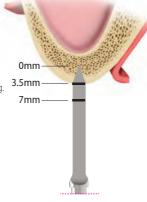


2. 2.2 Twist Drill 800~1.000 rpm

- > Use for making guide hole before using the Crestal Drill.
- > Connect the Crestal Drill Stopper according to the height of the remaining bone.






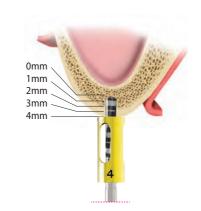


3. Crestal Drill 400~800 rpm

- > Use the Crestal Drill sequentially according to the diameter of the fixture to be placed.
- > Can also be used if sinus floor is flat, incline, septum.
- > The Crestal Drill can be used about 50 times (depending on bone quality).



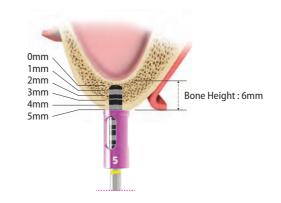

Fixture Dia.	Ø3.3	Ø3.5	Ø4.0	Ø4.5 / Ø5.0
Diameter	Ø2.8	Ø3.3	Ø3.7	Ø4.2
	KSCD28	KSCD33	KSCD37	KSCD42

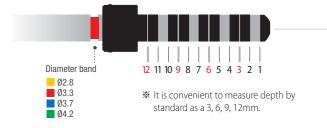



※ Flat floor edge minimize damage to membrane.


4. Crestal Drill Stopper

- > Connected with a stopper to be drilled to the same length of the cartilage height of maxillary sinus which is measured by CT.
- > If not equipped with CT, fasten the stopper one step lower than expected and gradually increase the length.



5. Depth Gauge

- > Measure thickness of the residual bone after checking the perforation of the cartilage of the maxillary sinus (do not open completely, only the entrance side should be opened).
- > The stopper is attached to the base of the residual bone to separate the cartilage and membrane from the maxillary sinus.

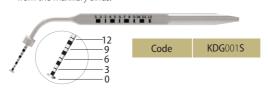



6. Aqua Membrane Lifter System

- > After confirming that elevation of the cartilage of maxillary sinus, elevate membrane with the Aqua Membrane Lifter System.
- 1) Fasten the Agua Lifer Drill to the drilled hole. ② Connect the Aqua Tube to syringe using the Aqua Syringe Connector (SC).
- 3 Inject saline solution equal to the amount of bone graft material
- 4 Tube connection to the Aqua Lifter Drill using the Aqua Ratchet Connector (RC).
- (5) Inject saline solution.



KSALT030


KSAL01SC

Agua Tube

Membrane is elevated by injecting saline solution

(0.1cc inject -> elevation

7. Torque Wrench

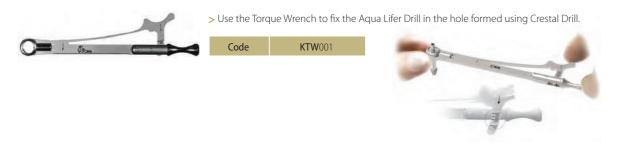

8. Bone Carrier





If you need to expand hole, drill 1mm deeper using the crestal drill.



> After connecting the stopper with the Bone Condenser, elevate bone graft materials to inside of maxillary sinus.







If you need to expand hole, drill 1mm deeper using the crestal drill.

10. Implant Drill (Final)

> Drill 1~2mm more deeply than steps of the Crestal Drill.



11. Implant Placement

> If the residual bone is less than 3mm, do not implant the fixture, but bone graft only.



Crestal Approach - Drilling Sequence

> Placing implant over Ø 4.0 is highly recommended.

1. Ø3.3 Narrow Fixture



2. Ø3.5 Fixture



3. Ø4.0 Fixture



4. Ø4.5 Fixture



- ₩ Ø5.0 Fixture Normal Bone : Drilling with the Final Drill before placing implants are required.
- ₩ Use a Drill that is one step shorter than the implant (E.g. 10mm implant, 8~9mm Drill).

Lateral Approach - Components

1. Ø6 Lateral Reamer 800~1,000 rpm

- > Drill after fastening the stopper according to the height of the bone.
- > Round shape to prevent membrane perforation.







2. Ø6 Lateral Round Drill 800~1,000 rpm

- > Drill after fastening the stopper according to the height of the bone.
- > Round shaped edge.
- > The residual bone should be replaced in the original position after drilling, sinus lifting & augmentation.







3. Lateral Stopper



Drilling Depth	0.5mm	1mm	1.5mm	2mm	2.5mm	3mm
	KSDSL05	KSDSL10	KSDSL15	KSDSL20	KSDSL25	KSDSL30

4. Sinus Elevator


> CSE-01 : Initial elevation of sinus membrane.





> CSE-02 : as stepwise, after using CSE-01, used for elevation of sinus membrane.



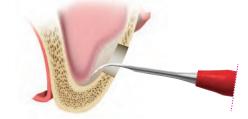

> CSE-03 : as stepwise, after using CSE-02, used for elevation of sinus membrane.








5. Ø4 Side Cutter 800~1,000 rpm

> When you expand window, must be connected with Stopper.






6. Sinus Bone Graft

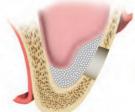

7. Implant Drill (Final)




8. Implant Placement

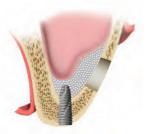

Sinus Lift KIT [KSA001]

> Sinus Lift Kit is the world's most innovating kit for performing maxillary sinus lift, ridge splits, and bone condensing cases. This revolutionary kit contains US Patented modified Tap Drills and Spreaders in order to allow any dentists to easily lift, split, or condense surrounding bone with simple drilling. Dentists can expect more predictable results, and patients can enjoy less traumatic surgeries with shorter chair time.

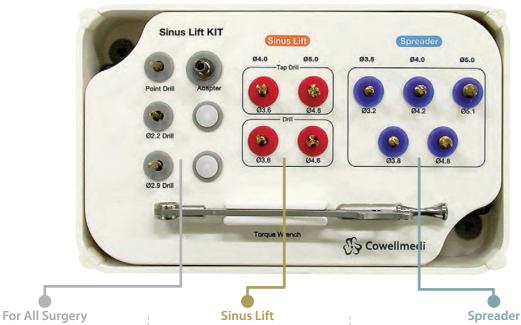

> Universally used Drills / used for both sinus lift or ridge split.

> Drilling must be accompanied with copious amounts of refrigerated sterile irrigation.

Drill Speed: 800-2,000 rpm





For manual operation using Torque Wrench, be connected with Machine Driver.





> Used in any maxillary sinus implantation.

Drill Speed: 20-30 rpm Torque: 45 N.cm






- > Used in bone condensing or ridge split implantation.
- > Also used in maxillary sinus lift & immediate

Drill Speed: 20-30 rpm Torque: 45 N.cm



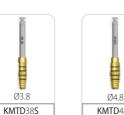






Sinus Lift

Tap Drill (Ø3.6, Ø4.6)

- > The Tap Drill uses low speed and high torque to grind through the maxillary bone, and safely elevates sinus without membrane perforation.
- > Must be used at 20 rpm / 40~45 N.cm.
- > No irrigation is required.



Diameter	Ø3.6	Ø4.6
	KMTD36S	

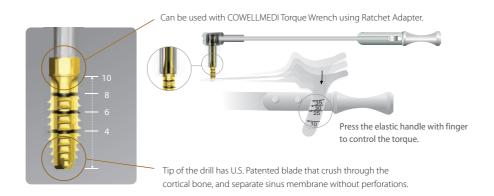

Twist Drill (Ø3.6, Ø4.6)

- > The Twist Drill is used after tapping as final drill for dense bone (Bone quality of 2 or greater), or to eliminate tapping thread in order to facilitate bone grafting.
- > Must be used at 20 rpm / 40~45 N.cm.
- > No irrigation is required.




Diameter	Ø3.6	Ø4.6
	KTWD36S	KTWD46S




Spreader

Tap Drill (Ø3.2, Ø3.8, Ø4.2, Ø4.8, Ø5.1)

- > The Spreader Drill is used to condense and/or spread the bone in either sinus lift or ridge split cases.
- > Must be used at 20 rpm / 40~45 N.cm.
- > No irrigation is required.

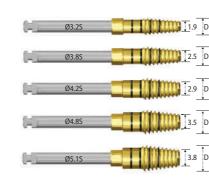

Diameter	Ø3.2	Ø3.8	Ø4.2	Ø4.8	Ø5.1
	KMTD32S	KMTD38S	KMTD42S	KMTD48S	KMTD51S

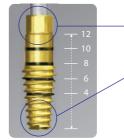


Can be used with COWELLMEDI Torque Wrench using Ratchet Adapter.

Tip of the drill has U.S. Patented blade that crush through the cortical bone, and separate sinus membrane without perforations.


Sequence -Sinus Lift


Only use of Sinus Lift Drill

1. Sinus Lift (Ø4.0 Fixture)

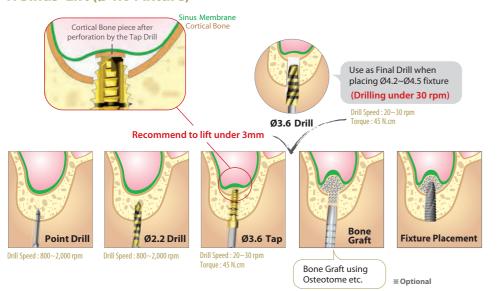


▶ Immediate Implantation and Sinus Lift Technique with Tap Drill (**Ø4.0 Fixture**)

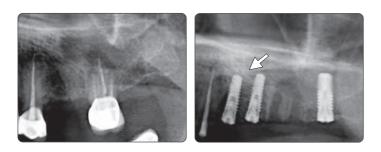

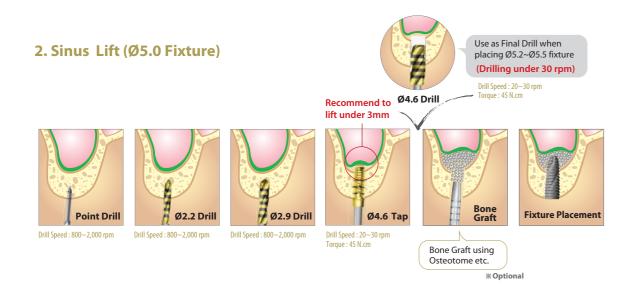


→ Immediate Implantation and Sinus Lift Technique with Tap Drill (Ø5.0 Fixture)

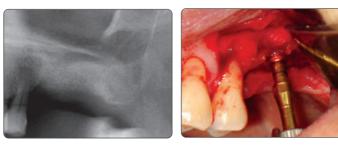






Recommend to use Sinus Lift Drill and Spreader Drill together

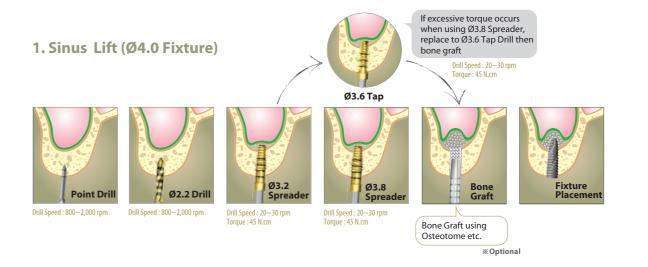

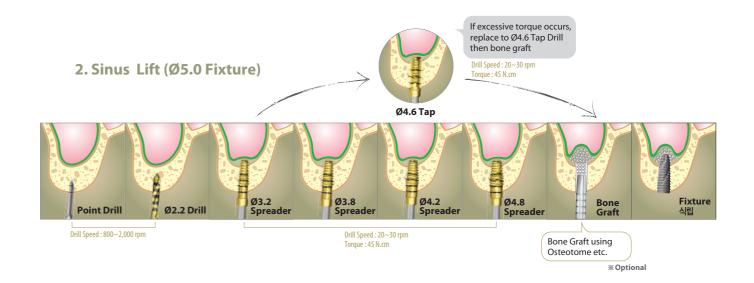


Note

- > Recommend to use Sinus Lift Drill and Spreader Drill together during the Sinus Lift operation.
- > Easily operate by using Ø3.2 Spreader rather than Point Drill.
- > Avoid to over press surrounding alveolar bone using Final Drill before fixture placement in D2.


ex) Ø4.0 Fixture placement


Point Drill Ø2.2 Drill Ø3.6 Tap Bone Graft Final drill Insertion of Fixture

Sequence-Spreader

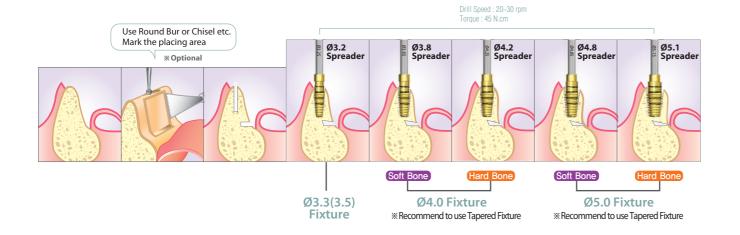


→ Ridge Split and Block Bone Augmentation Technique with Spreader Drill (**Ø4.0 Fixture**)

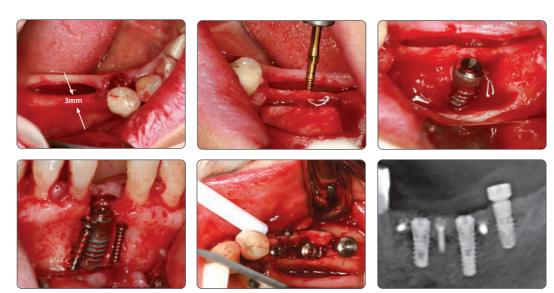

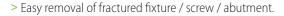



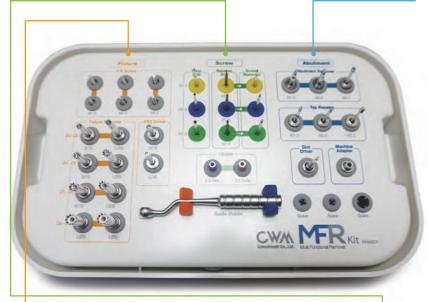









Screw Removal System



Fixture Removal System

Fixture Remover

Ø3~Ø4

Ø4~Ø5












MFR Kit - Components

1. Fixture Removal System

- 1 Connect the F/R Screw to the FRS Driver.
- 2 Connect the F/R Screw mounted FRS Driver to the fixture (clockwise 40~60 N.cm) and remove the FRS Driver.
- ③ Connect the Fixture Remover to the F/R Screw (counterclockwise).
- 4 Remove the fixture after connecting the Torque Wrench (counterclockwise, 100~400 N.cm).
- (5) To remove the fixture from the Fixture remover, use such device as vise to fix the Fixture Remover and connect to the Torque Wrench.
- 6 After connecting the FRS Driver to the F/R Screw, use the Torque Wrench to remove the F/R Screw (counterclockwise).

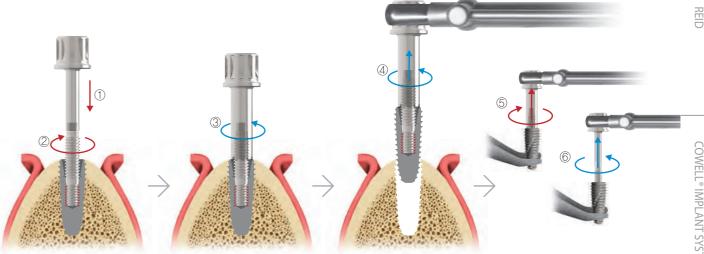


- X One-time use of the F/R Screw is recommended (bending or fracture may happen if more than 100 N.cm and using twice may be possible if less than 100 N.cm).
- * Sufficient irrigation is required when removing the fixture.
- ₩ When the maximum torque is exceeded, the fixture may be bent or fractured.
- ₩ If the fixture can not be removed even with maximum torque, remove the Fixture Remover & F/R Screw, remove bones around fixture using round bur and retry to remove.

2. Screw Removal System

Talon Drill

- 1) Check the broken screw size inside the fixture.
- 2 Connect the I Guide corresponding to the fixture to the Guide Holder and fasten to the fixture.
- (3) Insert the Talon Drill into the I Guide Hole.

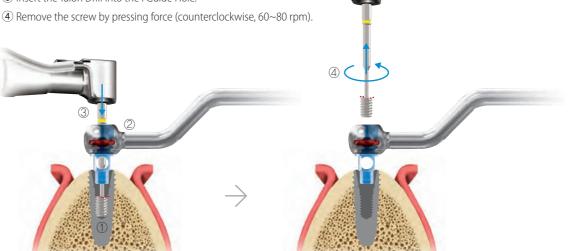


* If the I Guide and fixture could not be correctly connected, the path is not correct.

222 MFR KIT

Reverse Drill & Screw Remover

If the screw could not be removed by the Talon Drill

- ① Form the hole on the fractured screw (depth 1~2mm / counterclockwise / 1,200~1,400 rpm).
- ② Use the Screw Remover according to the created drill hole, remove the screw by pressing force (counterclockwise, 80 N.cm).

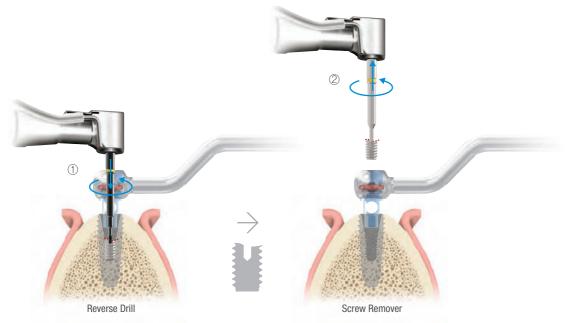


- 💥 If the path of the I Guide and fixture did not match, It would be difficult to remove the screw because the drill hole is away from the center of the screw.
- ※ Reverse drilling requires removal of chips by irrigation & suction.
- \divideontimes The fractured screw may be removed during reverse drill hole creation.
- ₩ If necessary, fasten to the Machine Adapter and use the hand or Torque Wrench.



3. Abutment Removal System


Abutment Remover

- ① Used when 2 piece type abutment is fractured.

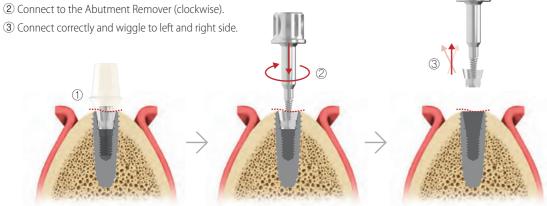

Tap Repair


- ① Used when the thread inside the fixture is occluded and damaged.
- ② Reproduce the thread using the Tap Repair.

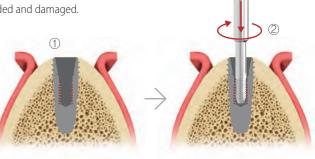

Slot Driver

- ① Used for the damaged solid abutment (1piece type), healing abutment and cover screw.
- 2 Form a slot on the surface of the damaged abutment using round bur. ③ Connect the Slot Driver to the formed slot and remove it (counterclockwise).

4. Torque Wrench

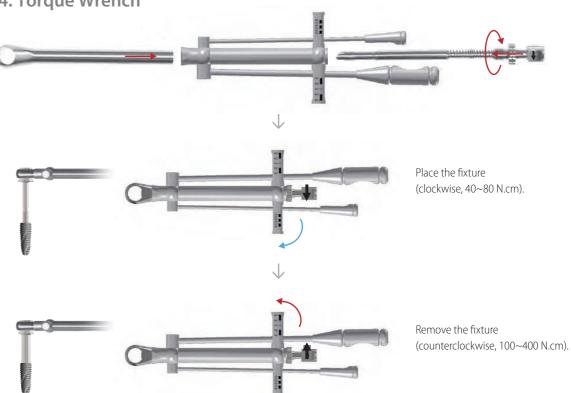

InnoGenic™ GBR Kit [KIGBRO01]

> An all-in-one solution for various types of GBR procedures. 3mm 7mm KIGFS03 KIGTS07 KIGFS03 KIGTS07 The length of the product can KIGFS03 KIGTS07 be measured with the scale marked on the middle tray of KIGFS03 KIGTS07 10mm KIGFS03 0 357 10 1315 KIGTS10 Fixing Screw Drill KIGFS05 KIGTS10 KIGFS05 KIGTS10 Tenting Screw Drill KIGFS05 KIGTS10 KIGFS05 KIGTS13 KIGFS05 3mm KIGDS03 KIGTS13 KIGFS07 5mm KIGDS05 KIGFS07 7mm KIGES07 KIGDS07 KIGTS15 KIGFS07 Machine KFSMD24 Fixing KIGTS15 KFSHD70 KIGTC32 Machine KTSMD24 -InnoGenic™ GBR Kit KIGBROO1 KIGTC32 Tenting Handle KIGTC32 Cover Cap C/Cap Healing Cap H/Cap Fix Connector Fixture Harvesting Drill Ø3.5 Drill Ø5.0 Drill 0.9 Hex Driver Ø4.5 Driver Handle Round Bur Bone Trimmer 1.0mm 1.0mm 1.5mm 1.5mm Ø4.5(3mm) Ø4.5(4mm) Ø4.5(4mm) KIGH KIGRB10 KIGBT50 KBH35 KBH50 KHD0921 KIGCC45 KIGHC453 KIGHC453 KIGHC454 Ø4.5 Ø5.5(3mm) Ø5.5(3mm) Ø5.5(4mm) Ø5.5(4mm) KIGCC45 KIGHC553 KIGHC553 KIGHC554 KIGHC554

Silicon Shield

1EA assembled with SEA packed in the lower tray.

Screw Kit KIGICS001

- Used without removing the Screw Kit from the inside of the kit tray (Remove to use if necessary only).
- Made of special material for autoclaving.
- * Rotate the upper lid to take out the selected product.





Composition

Classification	Product	Cod	Quantity	
		100	KIGFS03	5
	Fixing Screw (Fixing)	haumon-	KIGFS05	5
	(Fixing)	-	KIGFS07	5
Bone			KIGTS07	4
DOTIC	Tenting Screw		KIGTS10	4
	(Tenting)		KIGTS13	4
			KIGTS15	4
	Tenting Cap (T/Cap)		KIGTC32	3
Fixture		januar .	KIGFC4505	2
	Fix Connector (F/Connector)	io an	KIGFC4510	2
			KIGFC4515	2
			KIGFC4520	2
	Cover Cap (C/Cap)	·-	KIGCC45	2
		-	KIGHC453	2
	Healing Cap		KIGHC454	2
	(H/Cap)	- Pa-	KIGHC553	2
		-	KIGHC554	2

Empty Screw Kit KIGICS





Bone

Fixing Screw (Fixing)

- Used to fix the membrane to the bone.
- Place slowly using the Fixing Driver (Machine/Handle).
- 3, 5 and 7mm length can be selected according to the bone quality. In hard bone, use after forming a basic drill hole using the Fixing Screw Drill.
- The wedge-shaped design is advantageous for self-tapping, allowing it to be fixed without drilling in normal bone.
- •The double thread shortens the placement time.

D(Ø,mm)	C(mm)	L(mm)	H(mm)	Code
		3.0	3.6	KIGFS03
2.0	2.0 0.6	5.0	5.6	KIGFS05
		7.0	7.6	KIGFS07

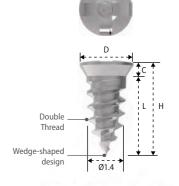

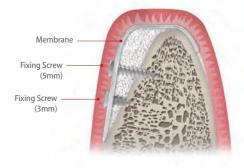



Bone

Tenting Screw (Tenting)

- Used when a large area of vertical / horizontal GBR is required. Leave space for bone grafts.
- Place slowly using the Tenting Screw Driver (Machine/Handle).
- Recommended placement depth : Hard bone-3mm, Normal bone-5mm, Soft bone-more than 5mm.
- Initial fixation of at least 15~25 N.cm is required. Tightening more than 35 N.cm may cause fracture of the Tenting Screw so it must be fixed below 35 N.cm.
- In normal bone, it is recommended to form a hole at least 3mm deep using the Tenting Screw Drill before placing the Tenting Screw.
- The wedge-shaped design is advantageous for self tapping, allowing it to be used without drilling in normal bone.
- •The double thread shortens the placement time.
- Use the Tenting Cap if necessary.

D(Ø,mm)	C(mm)	L(mm)	H(mm)	Code
	2.5	7.0	9.5	KIGTS07
3.2		10.0	12.5	KIGTS10
3.2	2.5	13.0	15.5	KIGTS13
		15.0	17.5	KIGTS15

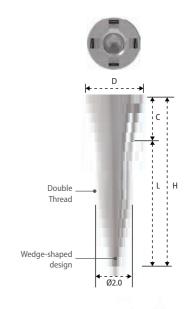

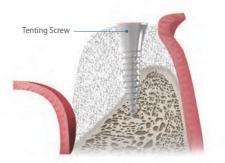


228 InnoGenic™ GBR Kit

Bone

Tenting Cap (T/Cap)

- Used to fix membrane on the Tenting Screw.
- Tightened with the 0.9 Hex Driver.
- Recommended tightening torque force : 5~8 N.cm.

D(Ø,mm)	C(mm)	L(mm)	H(mm)	Code
3.2	0.3	2.8	3.1	KIGTC32

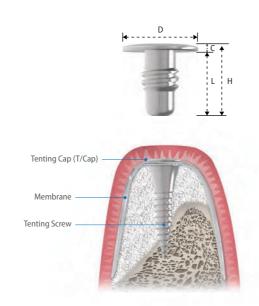


Fixture

Cover Cap (C/Cap)

- Used to fix membrane over the Fix Connector.
- For submerged surgery in case of sufficient soft tissue.
- Recommended tightened with the 0.9 Hex Driver.
- Recommended tightening torque force: 5~8 N.cm.

D(Ø,mm)	C(mm)	L(mm)	H(mm)	Code
4.5	0.3	3.4	3.7	KIGCC45

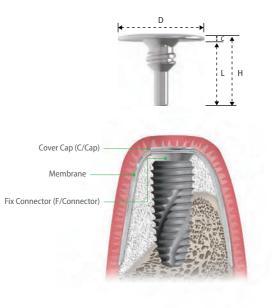


Fixture

Fix Connector (F/Connector)

- Used to fix the membrane along with the Cover Cap or Healing Cap after connecting to the fixture.
- Tightened with the 0.9 Hex Driver.
- Recommended tightening torque force: 12~15 N.cm.
- Available for the INNO Submerged, Submerged Short Fixtures and other fixtures compatible with them only.

D(Ø,mm)	C(mm)	L(mm)	H(mm)	Code
	0.5		6.2	KIGFC4505
4.5	1.0	5.7	6.7	KIGFC4510
4.5	1.5		7.2	KIGFC4515
	2.0		7.7	KIGFC4520



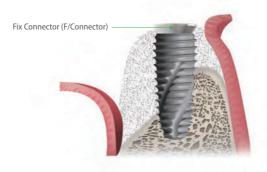


Fixture

Healing Cap (H/Cap)

- Used to fix membrane over the Fix Connector.
- For non-submerged surgery in case of insufficient soft tissue.
- Connect by using the 0.9 Hex Driver.
- Recommended tightening torque force: 5~8 N.cm.

mm)	L(mm)	H(mm)	6 1
		()	Code
3.0		6.4	KIGHC453
4.0	3.4	7.4	KIGHC454
3.0	5.1	6.4	KIGHC553
4.0		7.4	KIGHC554
	1.0	3.4	3.4 7.4 6.4



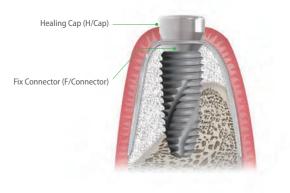

230 InnoGenic™ GBR Kit

Fixing Screw Drill & Tenting Screw Drill

- Used to place the Fixing Screw / Tenting Screw mainly in hard bone.
- Also used to perforate cortical bone when blood supply is required.
- For normal bone, drill only 3mm deep if necessary.
- Drill before placing the Fixing Screw / Tenting Screw.
- Laser-marked at 3, 5 and 7mm long from the tip of the drill and the length can be controllable using the Drill Stoppers.
- Color-banded for distinction (Red : Fixing Screw Drill, Blue : Tenting Screw Drill).
- Recommended drilling speed: 1,000~1,200 rpm.


Classification	D(Ø,mm)	L(mm)	H(mm)	Code
Fixing Screw Drill	1.0	10	31.5	KFSD10
Tenting Screw Drill	1.4	10		KTSD14





Drill Stopper

- Used by connecting to the Fixing Screw Drill / Tenting Screw Drill.
- 3mm : Yellow, 5mm : Red, 7mm : Blue

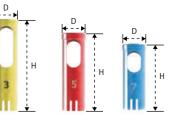


Classification	D(Ø,mm)	H(mm)	Code
3mm		13.5	KIGDS03
5mm	3.5	11.5	KIGDS05
7mm		9.5	KIGDS07



0.9 Hex Driver (Ratchet)

• Used to install the Tenting Cap, Fix Connector, Cover Cap and Healing Cap.

D(Ø,mm)	L(mm)	H(mm)	Code
	8	15	*KHD0915
1.2	14	21	KHD0921
	20	27	*KHD0927

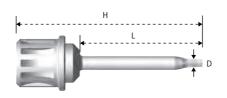



* Optional

Fixing Screw Driver & Tenting Screw Driver (Machine)

- Used to place the Fixing Screw / Tenting Screw using Contra-angle.
- Color-banded for distinction (Red : Fixing Screw Driver, Blue : Tenting Screw Driver).

Classification	D(Ø,mm)	L(mm)	H(mm)	Code
Fixing Screw Driver	1.6	6.0	24.0	KFSMD24
Tenting Screw Driver	2.2			KTSMD24

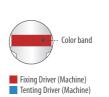

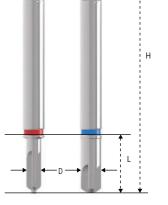

Fixing Screw Driver & Tenting Screw Driver (Handle)


- Used to place the Fixing Screw / Tenting Screw using the Driver Handle.
- Color-banded for distinction (Red : Fixing Screw Driver, Blue : Tenting Screw Driver).




Classification	D(Ø,mm)	L(mm)	H(mm)	Code
Fixing Screw Driver	1.6	6.0	70.0	KFSHD70
Tenting Screw Driver	2.2			KTSHD70

Driver Handle

• Used to place and remove the Fixing Screw / Tenting Screw by connecting the Driver Handle.

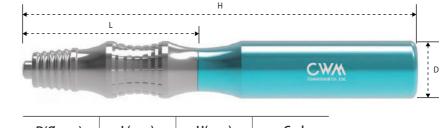


D(Ø,mm)	L(mm)	H(mm)	Code
19.8	75	135.0	KIGH

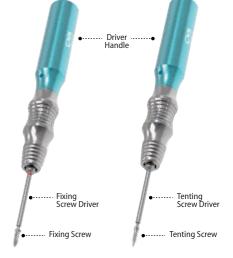


Round Bur

- Used to perforate cortical bone when blood supply is required.
- Recommended drilling speed: 1,200~1,500 rpm.

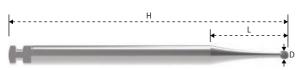


D(Ø,mm)	L(mm)	H(mm)	Code
1.0	9.5	34.0	KIGRB10

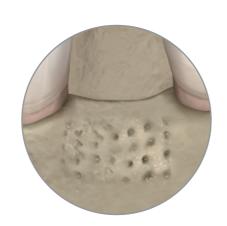


Bone Trimmer

- Used to perform osteoplasty on the outer wall of remaining bone all during GBR and to flat the bone surface for improving the fit of membrane.
- Used to remove remaining granulation tissue of bone defect part (use instead of surgical curette).
- Recommended drilling speed: 1,200~1,500 rpm.

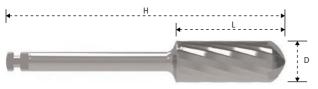

D(Ø,mm)	L(mm)	H(mm)	Code
5.0	13	34.0	KIGBT50

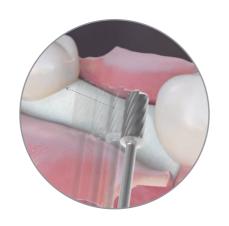


Harvesting Drill & Drill Stopper

- Drill for convenient harvesting of autogenous bone in the form of bone chip in a short period of time.
- The Silicon Shield of the Ø3.5 Harvesting Drill makes sure with no bone chip loss while drilling (Bone chip can be collected at implant site).
- 6 Silicon Shields are included in the Kit (1 is assembled with the Ø3.5 Harvesting Drill and 5 are packed in the lower tray).
- •The maximum drilling depth of the Ø3.5 Harvesting Drill is 12mm, so it needs to be drilled slowly.
- Remove while rotating the drill.
- Recommended drilling speed: 300~500 rpm.

D(Ø,mm)	L(mm)	H(mm)	Code
3.5	9.5	39.2	KBH35
5.0	6.5	36.5	KBH50

	D(Ø,mm)	H(mm)	Code
Drill Stopper	5.6	9	KBHD3540
	6	14.3	KBHD4550

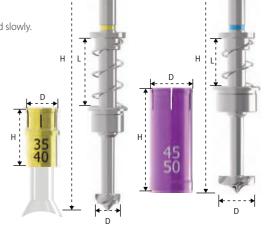

* For the details of InnoGenic™ Autobone Harvester, refer the pages 236~240.

Bone Carrier

- Narrow tip is beneficially handled in most of the bone graft techniques.
- Bone graft particles can be accurately and safely injected without contamination.
- rhBMP-2 can be easily coated to the implant due to circular groove of tip.
- \bullet Bone graft particles and rhBMP-2 solution can be well mixed on the circular groove.






* Bone Carrier length is 94mm and the total length after stretching is 114mm.

Bone Carrier	D(Ø,mm)	H(mm)	Code
bone Carrier	6	94	KBBC01

234 InnoGenic™ GBR Kit 235

InnoGenic[™] Autobone Harvester [KIAH001]

> Maximize Your Return On Minimal Investment, Guaranteed!

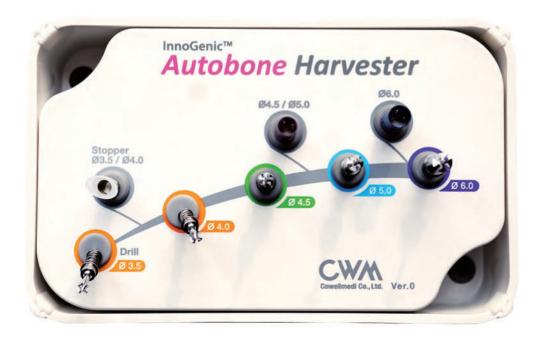

Harvesting Drill









Silicon Shield

KBHDSS01



Drill Stopper








* 1EA assembled with Drill. 5EA packed in the lower tray.

Key Concepts

Maximize your return on minimal investment

The key concept of the Autobone Harvester Plus to harvest a large amount of the autogenous bone chips from the implant site that can be wasted into the suction during implant drilling procedure.

Features: Drill

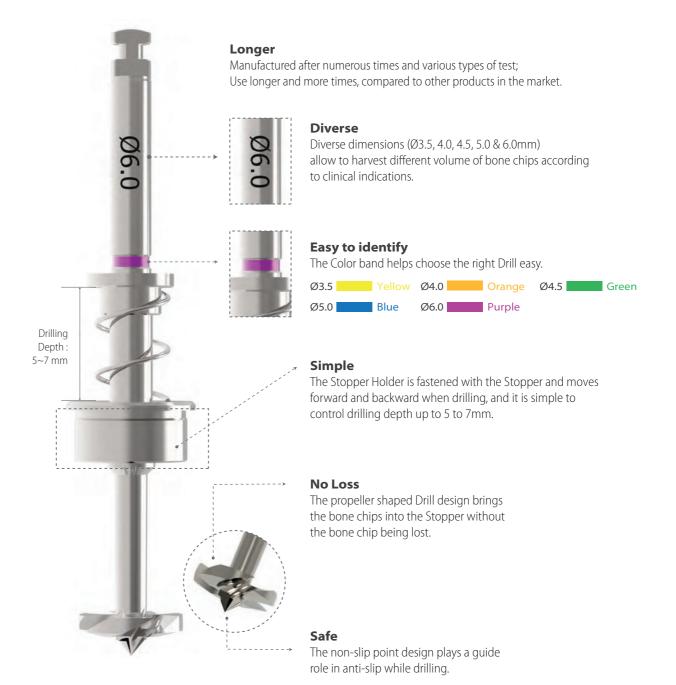

Features: Stopper & Silicon Shield

For Ø3.5 & 4.0 Drill



Stopper

Used by fastening to the Stopper Holder of Ø3.5 & 4.0 Drill.



Silicon Shield (*Exclusive for Ø3.5 & 4.0)

- Used by fastening to $\ensuremath{\text{\emptyset}3.5\ \&\ 4.0}$ stopper.
- Prevents deviation of bone chips.
- Allows bone chip harvesting from the implant site.
- Reusable transparent silicon material allows to check drilling position and bone chips being harvested.



Shield

The lip-shaped shield is brought into close contact with the bone and makes sure with no bone chip loss while drilling.

For Ø4.5 & 5.0 Drill



Used by fastening to the Stopper Holder of Ø4.5 & 5.0 Drill.





For 6.0 Drill



Stopper

Used by fastening to the Stopper Holder of Ø6.0 Drill.





Harvesting sequence:

Implant Site using Ø3.5/4.0 Harvesting Drill with the Silicon Shield



• Point drill to mark harvesting and implant site.



• Select Ø3.5/4.0 Drill and insert the Stopper into the selected Drill. And put the Shield on the Ø3.5&4.0 Stopper.



• Drill at 300 to 500 rpm with irrigation and harvest bone chips.



• Disassemble the Silicon Shield, the Stopper and collect the bone chips for bone grafting.



• Use Final Drill (equal to or over Ø3.5/4.0) according to the drilling protocol of the manufacturer and treatment planning.



• Place the implant.



• Apply the harvested bone chips on the site.



Harvesting sequence:

Buccal Bone Harvesting using Ø3.5/4.0/4.5/5.0/6.0 Harvesting Drill

Select the drill according to its diameter and clinical indications.









• Drill at 300 to 500 rpm with irrigation and harvest autogenous bone chips.

• Apply the harvested bone chips on the site.

A Clinical Case using Ø3.5/4.0 Harvesting Drill



Drilling at 300 rpm with irrigation was carried out after marking implant and harvesting position.



The Silicone Shield was brought into close contact with various types of bone level and prevented



The amount of bone taken was easily ascertained, through the transparent Silicone Shield.



The bone was transferred to bone dish after disassembling the Silicon Shield and Stopper. The amount of the bone was much more than expected.



After the implant placement, healing abutments were connected and carried out GBR in the defected area.

COWELL® BMP Trephine Kit [KBT001]

> An easy-to-use kit with drills and instruments for block-type bone collection, failed fixture removal, crestal & window approach for sinus lift and bone chip extraction.




Trephine Drill I: Block Bone Extraction





Implant Site Drill: Sinus Lift & Bone Chip Extraction Prior to Implant Placement

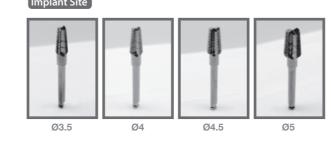

Trephine Drill III: Window Opening for Lateral Window Approach



J	
Ø7	


Product	Diameter	Code
	Ø 6.0 (Inner)	KBGT60
Block Bone Guide Drill	Ø 7.0 (Inner)	KBGT70
	Ø 8.0 (Inner)	KBGT80
	Ø 6.0 (Inner)	KBT60
Block Bone Trephine Drill	Ø 7.0 (Inner)	KBT70
	Ø 8.0 (Inner)	KBT80
	Ø 4.2 (Inner)	KFRT40
Fixture Removal Trephine Drill	Ø 4.7 (Inner)	KFRT45
	Ø 5.2 (Inner)	KFRT50
Window Trephine Drill	Ø 7.0 (Outer)	KWTT60
	Ø 3.5 (Fixture)	KTIS35
Local City D. III	Ø 4.0 (Fixture)	KTIS40
Implant Site Drill	Ø 4.5 (Fixture)	KTIS45
	Ø 5.0 (Fixture)	KTIS50


COWELL® BMP Trephine Kit 241 240 InnoGenic™ Autobone Harvester

^{* 2} Step Harvesting: Drilling to 7mm is recommended after transferring bone chips to bowl since the Stopper & Silicon Shield are fully filled with bone chips while 4mm drilling.

Trephine Drill I Block Bone Extraction

This Drill allows the collection of block-type autogenous bone with a required size in the case of regenerating a wide bone defect and severe bone resorption.

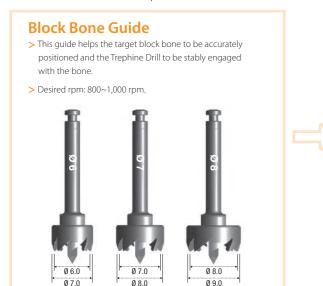

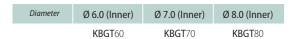


Block Bone Trephine Drill

> This drill is engaged with the bone groove with the help of the block bone guide to collect the block bone with a desired size.




Diameter	Ø 6.0 (Inner)	Ø 7.0 (Inner)	Ø 8.0 (Inner)
	KRT60	KRT 7∩	KRT80

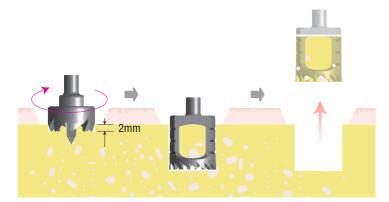




If the block bone is not removed using the Trephine Drill, use dental elevator or forceps to remove it.


Trephine Drill II Failed Fixture Removal

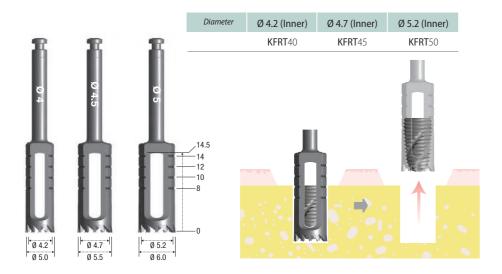

Trephine Drill III Window Opening for Lateral Window Approach

Diameter	Ø 7.0 (Outer)	
	KWTT60	

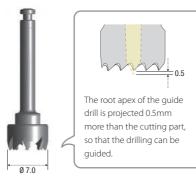

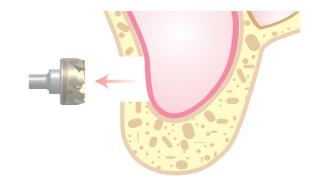


Implant Site Drill Sinus Lift & Bone Chip Extraction Prior to Implant Placement

Diameter	Ø 3.5	Ø 4.0	Ø 4.5	Ø 5.0	
	KTIS 35	KTIS40	KTIS45	KTIS50	



- > Used before the Final Drill is used (simplified drilling sequence).
- > Advantageous for securing autogenous bone.
- > Less rpm drilling leads to low bone heating.
- > Also used as a sinus lift tool (socket lift elevation).
- > Desired rpm: 20~30 rpm.

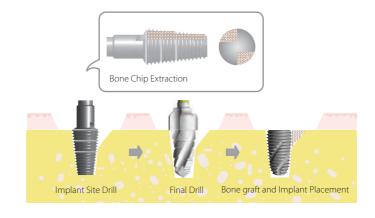



242 COWELL® BMP Trephine Kit COWELL® BMP Trephine Kit 243

Atraumatic Extraction Kit [KAE001]

> Used for the immediate and effortless extraction of the root of the tooth with simple procedures.



(1) Diversity

A root extraction can be done regardless of whether residual amount of root is large or small.

(2) Safety

A root extraction without the risk of damaging adjacent teeth is possible using the Rest Plate, Elevator etc.


(3) Convenience

A very simple and convenient root extraction is possible, compared to the existing extraction method.

(4) Reduced Procedure Time

The procedure time is reduced due to the simple procedure.

Composition

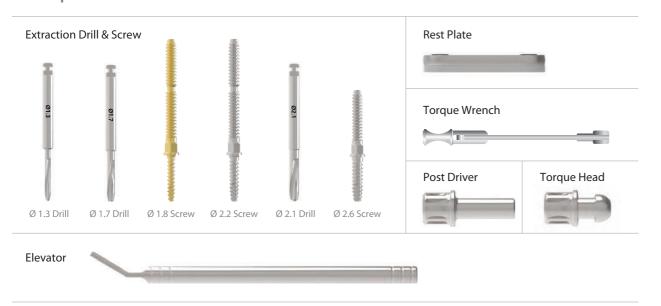


1. Extraction Drill

> The Extraction Drill is composed of three types of Drills (Ø1.3 / Ø1.7 / Ø2.1) that can be selected according to the case.





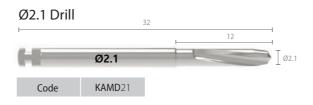


2. Extraction Screw

> The Extraction Screw is fastened into the hole that was created by the Extraction Drill via the Screw method, and it is stably fixed to the remaining root. It is composed of the \emptyset 1.8 / \emptyset 2.2 / \emptyset 2.6 Screws that can be selected according to the Extraction Drill.

> The Ø1.8 Screw is used for vital root of which canal is not treated, after using the Ø1.7 Drill.





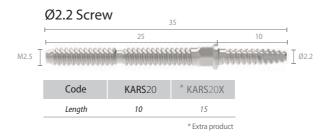








COWELL® DIGITAL PRODUCTS

3. Post Driver


> After connecting the Post Driver to the Extraction Screw, turn the Torque Wrench in a clockwise direction in order to fix it to the hole that was created by the Extraction Drill (recommended torque: 30 N.cm or more).

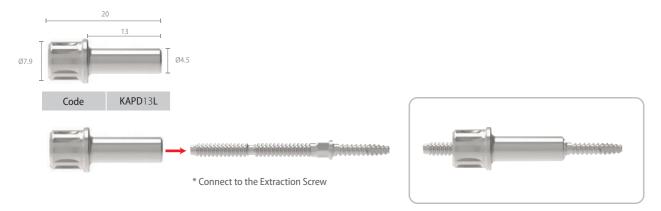

4. Rest Plate

> The Rest Plate is connected between the Extraction Screw and the Torque Head. It protects the part with silicon that comes into direct contact with the adjacent teeth in order to prevent teeth damage.

It also serves as a support for the Elevator and Torque Wrench.

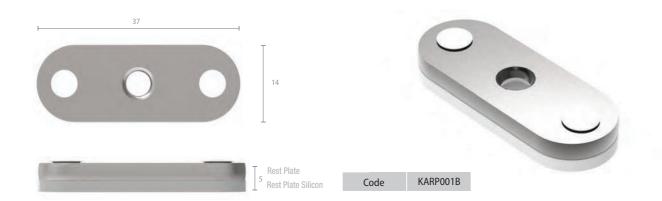


5. Torque Head


- > The Torque Head is connected to the Extraction Screw that is fixed in the tooth to be extracted. It fixes the gap of the Rest Plate and it can be used with the Elevator.
- > If the root to be extracted has both distal and mesial adjacent teeth, it will be extracted with the Torque Wrench (recommended torque: 100 N.cm or less).



6. Elevator

> The Elevator is used by connecting it with the Torque Head and extracting the root by applying force toward a distal or mesial direction.

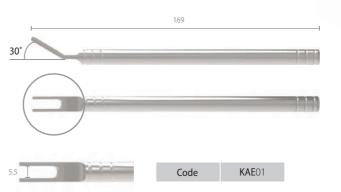

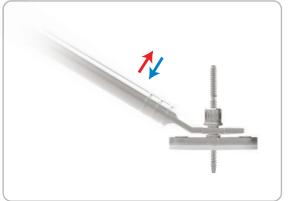

How to Use

1. Extraction Drill

Create a hole on the tooth to be extracted using the Extraction Drill.



Caution A

- The Extraction Drill must follow the neural root canal during drilling.
- Drill down to at least 10mm because extraction is possible even if the Drill and Screw penetrate the root.



2. Extraction Screw

Connect the Extraction Screw to the Post Driver and fix it to the hole created by rotating it clockwise (recommended torque: Min. 20 N.cm ~Max. 25 N.cm).




Connect Post Driver to the Extraction Screw.

246 Atraumatic Extraction Kit

* Drilling Sequence

Root Canal Preparation

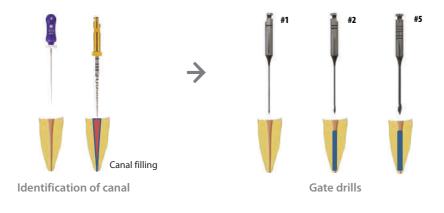

Atraumatic Extraction kit



Caution C

- Fix the screw with a torque of $20\sim25$ N.cm. If it is not applied, use a thicker Screw.
- The low torque force causes the Screw to fall out during the extraction, and the over torque force fractures tooth root.

3. Rest Plate

After removing the Post Driver, connect a Rest Plate to the Extraction Screw by taking into account the adjacent teeth.



Rest Plate

4. Torque Head

Connect the Torque Head to the Extraction Screw projected above the Rest Plate by rotating it clockwise.



Connect Torque Head to Screw

5. Torque Wrench

Extract the tooth by rotating the Torque Head clockwise using the Torque Wrench.




Extraction Root

Caution D

- Extraction using the Torque Wrench is possible in a root with mesiodistal root.

Caution E

- If there are adjacent teeth with 2 or higher swaying degrees, upward pulling or downward pressing should be applied using the Elevator so that the teeth will not receive force during extraction.

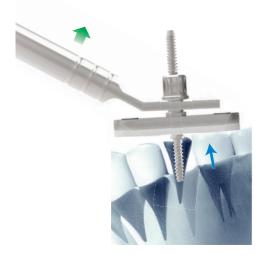




Caution F

- If there is an adjacent tooth projected to the mesiodistal root, it must be extracted using the Elevator.

Direct Surgical Guide Kit [KDSG002]

- > Used for a flapless surgery.
- > Served as a guide for positioning the drill while measuring the thickness of the buccal bone, thereby preventing bone resorption and reducing the burden on the patient.
- > Stable Drill positions can be guided to various cases by combining different surgical instruments (E.g., Position Guide, Linker and Width Guide).

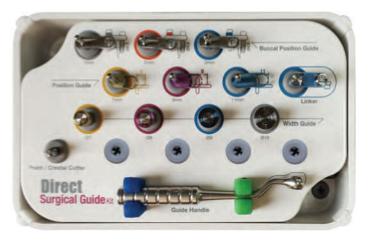

(1) Safety

Safe operation is possible by measuring the thickness of the buccal bone in order to prevent buccal bone resorption.

(2) Reduced Pain

It is possible to minimize the burden on the patient by reducing the pain and swelling from a flapless surgery without going through the incision step that is carried out during the general implant surgery.


(3) Accuracy and Diversity


An accurate operation is possible depending on the bone width and implant diameter by combining the operation instruments (E.g., Linker, Position Guide and Width Guide). It can be used for various cases, such as single, bridge etc.

1. Buccal Position Guide

- > Use the depth pin positioned at the bottom of the guide according to the bone width and fixture diameter.

 This will enable you to maintain the buccal bone thickness of the implant. The guide should be connected to the guide holder.
- > The length can be distinguished with the O-ring color of the post.
- > Be careful in using the depth pin because it is extremely sharp.

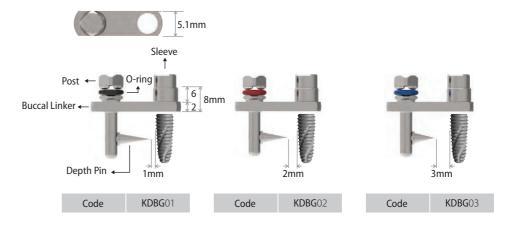

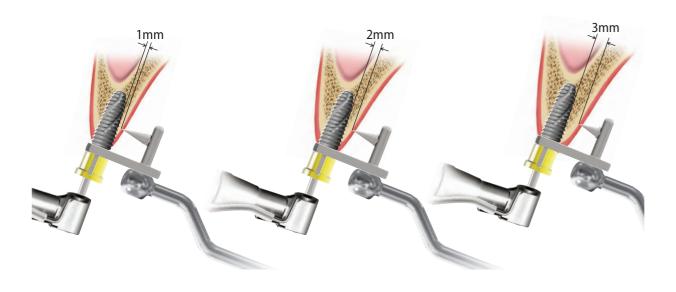

Caution C

Select the 1, 2 or 3mm Buccal Position Guide depending on the alveolar bone width and implant diameter to determine the buccal bone thickness.

Bucco-palatal bone width








Black O-Ring

Under 7mm bone width, under Ø4.0mm fixture

Caution A

Please fix the depth pin of the Buccal Position Guide to the buccal bone surface 5mm below the gingiva or the alveolar crest.

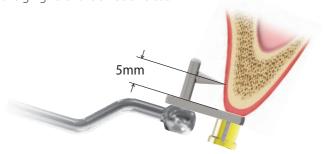

Bucco-palatal bone width




Fixture diameter



Red O-Ring

Bone Width 7~9mm, Ø4.0mm~Ø4.5mm fixture

In a defective buccal bone or a thin buccal bone due to tooth extraction, fix the depth pin of the Buccal Position Guide to the surface 5mm below the gingival or the alveolar crest, from the

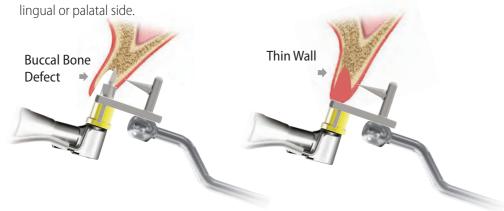

Bucco-palatal bone width



Fixture diameter




Blue O-Ring

Over 9mm Bone Width, Ø5mm fixture


252 Direct Surgical Guide Kit

2. Width Guide

> Select the 7, 8, 9, or 10mm-width guide according to the implant crown diameter, mount it on the Buccal Position Guide, and Drill the center of the implant (the size is distinguished by color).

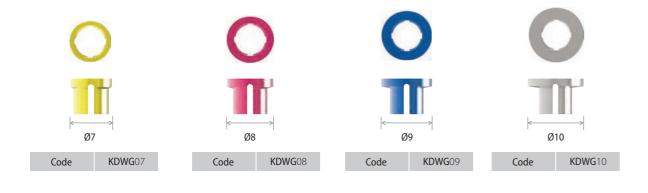

The width guide is used by connecting it with the sleeve of the Buccal Position Guide.

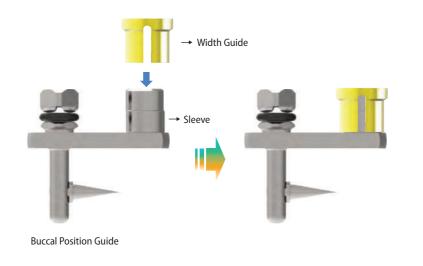








Mandible	Central incisor	Lateral incisor
Mesiodistal dimension	5.4	6.0


How to use according to the mesiodistal dimension-2

(Use of the combined Width Guides)

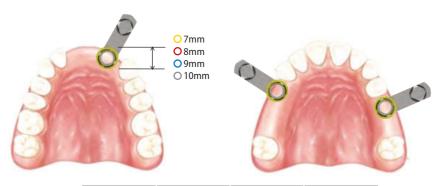



Maxilla	Lat. Incisor	1st Premolar	2 nd Premolar
Mesiodistal dimension	7.3	7.7	7.3



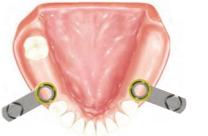

Mandible	Canine	1st Premolar	2 nd Premolar
Mesiodistal dimension	7.2	7.3	7.3

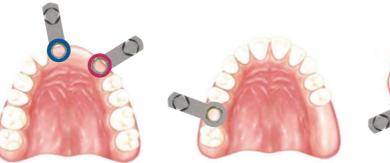

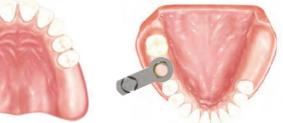



Maxilla	Central Incisor	Canine	1 st Molar	2 nd Molar
Mesiodistal dimension	9.2	8.3	10.5	9.5


Mandible	1 st Molar	2 nd Molar
Mesiodistal dimension	11	10

3. Position Guide

- > Guide the Drill position according to the distance between implant centerlines (7, 9, 11mm). The device consists of three gaps: 3mm, 5mm, and 7mm from the Ø4mm fixture interface.
- > In continuous implants, select the 7, 9, or 11mm Position Guide according to the crown size, and mount it on the Buccal Position Guide to drill.

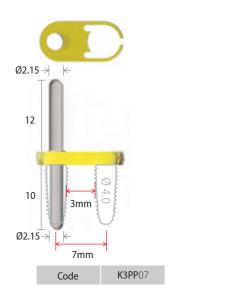

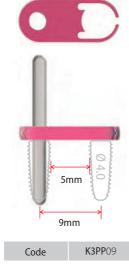


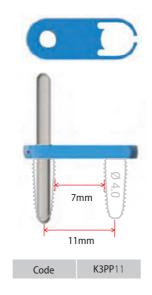

*The Position Guide must be used in combination with the Buccal Position Guide and linker. It cannot guide the Drill position accurately when used alone.

Application of the 7mm Position Guide

(when the mesiodistal distance between the two missing teeth is at least 14mm)

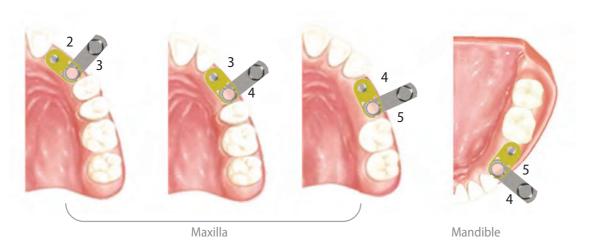

Application of the 9mm Position Guide (when the mesiodistal distance between the two missing teeth is at least 16mm)





Maxilla

Application of the 11mm Position Guide


(when the mesiodistal distance between





Mandible

4. Linker

> When you perform prosthesis with a bridge in a continuous implant, connect the 11mm Linker to the position guide and the Buccal Position Guide to drill.

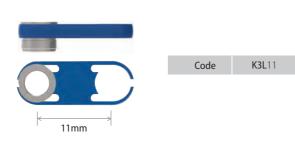

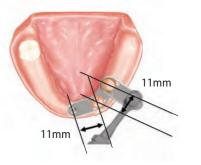


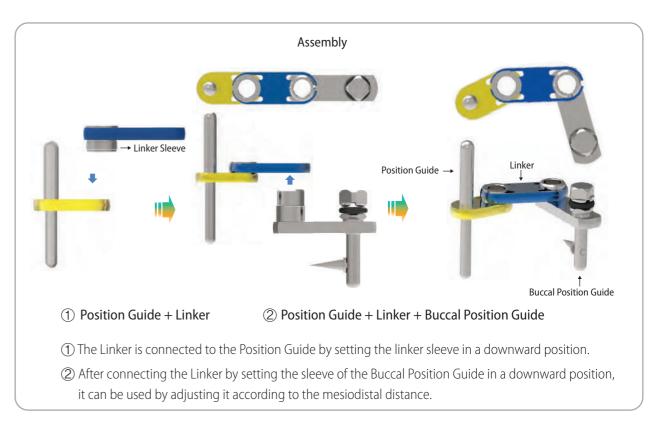

256 Direct Surgical Guide Kit 257

5. Point / Crestal Cutter Drill

- > A hole is created on the cortical bone in order to facilitate the Initial Drill.
- > The gingiva residue is removed while flattening the uneven alveolar bone surface.

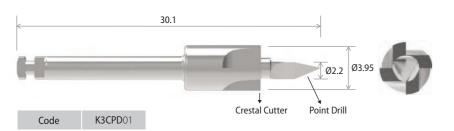






6. Guide Handle

> The Guide Handle guides the position and direction by connecting it with the Buccal Position Guide.



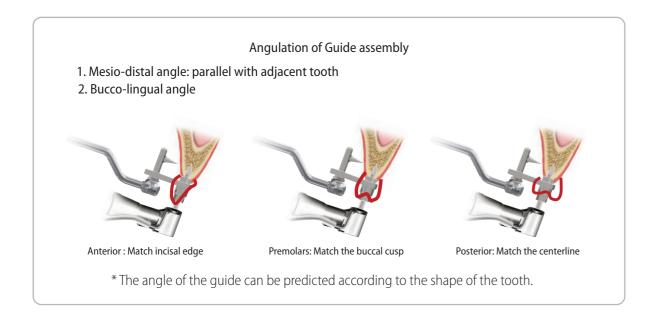

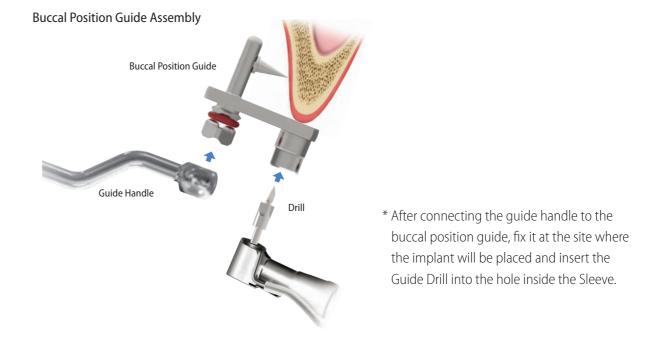

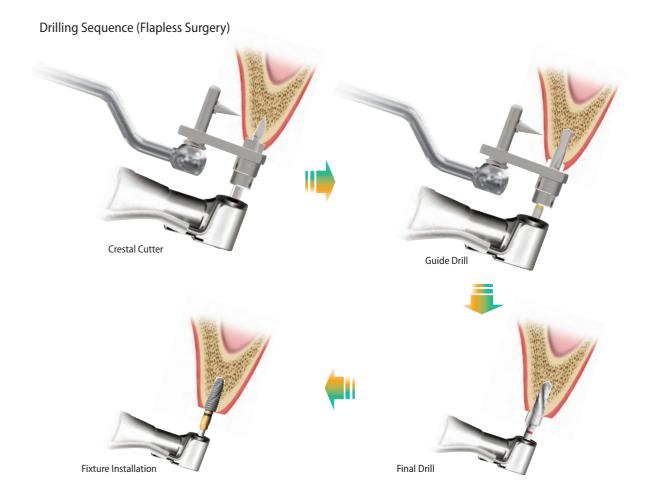


* Form basic holes with a crystal cutter. Then, select a drilling sequence according to the implant diameter.

258 Direct Surgical Guide Kit 259

AO4 Surgical Stent [KDSS001]

> Guide the position of Implant and Drill.

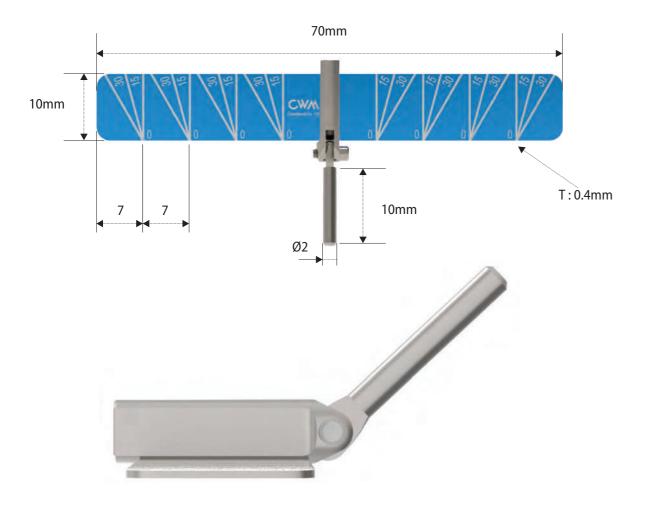


Characteristic

- > Guide the position of the implant and drill during implant placement.
- > It improves the stability and accuracy in surgery, and it can shorten the time.
- > By preventing the loss of healthy gums as much as possible, pre-fabricated prostheses can be placed immediately after surgery without the need for gum restoration.
- > Angled line allows more precise and predictable surgery.

Eligible for


- > A toothless patient.
- > Patient who do not want long-period of surgery.
- > Patients suffering from adult diseases such as hypertension and diabetes.
- > Patients who need precise implant treatment.

Instruction

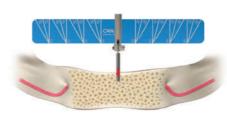

Place the AO4 Surgical Stent

- Make an incision for flap lift.
- Place the AO4 Surgical Stent using Ø2mm Twist Drill. * It is needed to check the position of mental foramen.

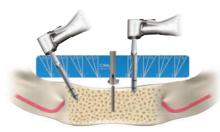

Place the INNO Fixture


• Drill with reference to the angled line and place the fixture.



Place the Multi S&A Abutment

- After placing the INNO fixture, connect the Multi S&A Abutment according to the site.
- * Posterior site : Place the Multi A abutment (30°) with 30 N.cm torque force.
- * Anterior site: Place the Multi A abutment (15°) or the Multi S abutment with 15 N.cm torque force (it is possible to allow emergence of the prosthetic screw).

or

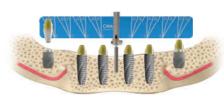

Placement Lock Abutment

- After placing the INNO Fixture, connect Lock Abutment according to the site.
 * If implant placement at an angle is not appropriate or not desired,
- * If implant placement at an angle is not appropriate or not desired using the INNO Sub. Short Implant is highly recommended.


260 AO4 Surgical Stent 261

Volume-up™ Guide System

> Devised for preventing food penetration and forming natural cervical area by restoring contracted buccal alveolar bone & gingiva to the original shape and width.

1. CONCEPT


• Peri-implant inflammations represent serious diseases after dental implant treatment, which affect both the surrounding hard and soft tissue.

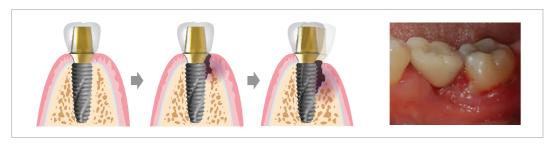


To achieve long term success of implant without complications like peri-implantitis, right position of fixture with min. 2mm of buccal bone width for buccal gingival regeneration and alveolar bone regeneration at min. 3mm lower position to maintain gingival height is extremely essential.

Min. 2mm of buccal bone regeneration to maintain having buccal gingiva. (Int J Periodontics Restorative Dent 2005)

Alveolar bone regeneration at minimum 3mm lower position to maintain gingival height. (Clin Oral Implants Res 2000;11: 1–11.)

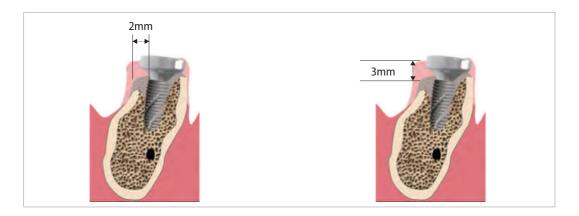


The Volume-up™ Guide System helps place implant in the right position according to 2 abovementioned clinical factors and helps select right dimension of the Healing Abutment to be used as a scaffold while gingival forming.

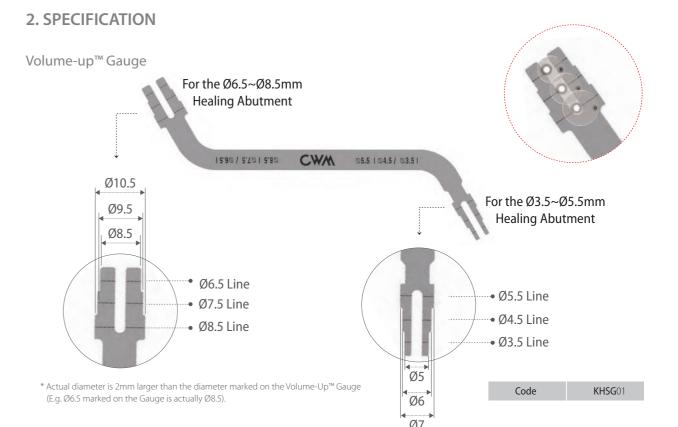


- > Used to guide the position of implant placement and to guide the election of the Healing Abutment dimensions in order to keep the cervical portion of the implant prosthesis at the natural tooth width.
- > Used with the Volume-up™ Parallel Pin for multiple units or bridge.
- > Used with Point Drill (Ø2.1mm or less).
- > Laser marking identifiable from any position.
- * For the selection of the Healing Abutment, refer the pages 34, 63, 81 & 100.

Volume-up™ Parallel Pin

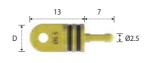

- > Used for bridge or multiple units with the Volume-up™ Gauge.
- > For bridge or multiple units.
- > For Ø3.5, Ø4.5 and Ø5.5, place the fixture and use the Healing Abutment instead of the Volume-up™ Parallel Pin.

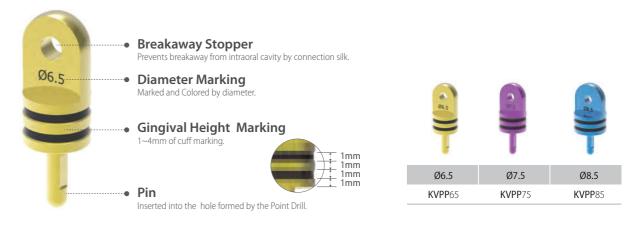

262 Volume-up™ Guide System

Volume-up™ Guide System 263

3. PROCEDURE

I. Single Implant




Set the Volume-up[™] Gauge on the implant site according to the diameter line marked on the Volume Up[™] Gauge.



Position the Point Drill in the drill insertion groove of the Volume-up™ Gauge.



Drill and place implant in accordance with the manufacturer's implantation sequence.

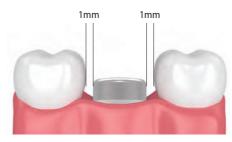

If implant placement torque is equal to or over 20~30 N.cm, connect the Healing Abutment. If not, connect the Cover Screw and do primary closure.

II. Multiple Implants & Bridge



Set the Volume-up[™] Gauge and position the Point Drill.



Insert the Volume-up™ Parallel Pin into the hole formed after point drilling.




Carry out the same as the previous step.


4. CLINICAL CASE



Preoperative view of the healed ridge.

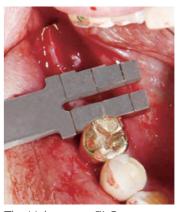

The Volume-up[™] Gauge was set to the 8.5 line and point drilling was carried out.

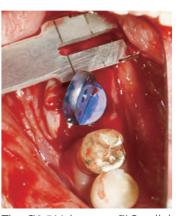

The Ø8.5 Volume-up™ Parallel Pin was inserted to the hole formed by point drilling and point drilling was done at the next site guided by the Volume-up™ Gauge.



The Ø8.5 Healing Abutments were placed after initial & finial drilling and fixture placement and bone grafting, and the site was sutured.



After 4 weeks, the contracted buccal alveolar bone & gingiva to the natural shape and width were restored.

Which will allow esthetically and functionally great prosthesis fabrication preventing food permeation.

^{*} For the selection of the Healing Abutment, refer the pages 34, 63, 81 & 100.

COWELL® REGENERATIVE SOLUTION

Inspire confidence through a comprehensive approach



human bone. Diabone has fast blood

optimal cell attachment and blood

absorption.

penetration by giving great hydrophilicity

and 3-Dimensional structure which allow



The products consisting of Titanium reinforced PTFE membrane (Wifi-Mesh),

PTFE membrane (PTFE-Mesh) and Titanium mesh (Ti-Mesh)

are non-resorbable membranes to be applied over intraoral

defects, especially, tooth extraction and bone augmented sites.

COWELL® BMP Osteoinductive Bone Graft rhBMP-2 + BCP

1. Composition of COWELL® BMP

- COWELL® BMP is bone graft material based on the E.rhBMP-2 (E.Coli derived Recombinant Human Bone Morphogenetic Protein type 2), developed for the first time in the world. COWELL® BMP is supported with 10 years of clinical data and over 40 studies.
- BCP as a carrier allows maintenance of space.

2. Features of COWELL® BMP

- Primary closure for soft tissue regeneration is not required.
- Regenerates adherent gingiva.
- Simplifies challenging bone grafting and soft tissue regeneration.
- Acts directly on stem cells.
- Induces bone regeneration without infection in extraction socket.
- Contains 1mg of bone morphogenic protein per 1g of the particle. (1g of autologous bone contains 2ng of bone morphogenic protein)

3. Application

A. Orthopedics

Bone grafts

- Fractures : Tibia, Radius, Ulna.
- Spine Fusion (Degenerative Disc Disease): Interbody cage, Posteolateral.

Injection Device

- Lengthening : Distraction osteogenesis.
- Osteoporosis: Hip joint fracture.
- Bone Defect : Bone cyst.
- Bone Fusion: Foot/shoulder revision.

B. Dentistry

Bone grafts

- Severely resorbed alveolar ridges.
- Tooth extraction socket.
- Alveolar bone loss.
- Maxillary sinus bone loss.
- Bone-inductive implant : Coated implant.
- Maxillofacial reconstruction.


C. Dermatology

Soft Tissue grafts


- Damaged skin regeneration.
- Diabetes ulcer.

4. Mechanism of Action of COWELL® BMP

A. Migration of Cells with lamellipodia

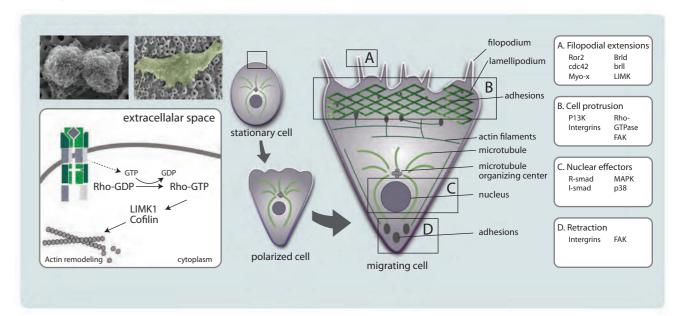

B. Cellular mechanisms

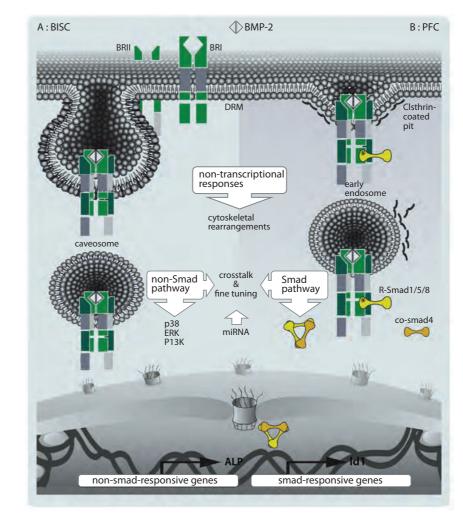

- BMP-2 adheres to the membrane of stem cell and induces expression of genes of nucleus. Then, BMP-2 migrates to recipient site.
- BMP-2 growth factor, Twist-2 transcriptional factor, and VEGF growth factor synthesize and secrete endogenous growth factor.
- Proliferation of osteoblast of osteocyte, and proliferation of fibroblast in dermis and keratinocyte of the skin.
- Twist-2 transcriptional factor induces tissue regeneration in osseous tissue and adherent gingival area.

268 COWELL® BMP

5. Product Type

COWELL® BMP (One vial)




• Dose and particle size of the COWELL® BMP



*A vial of 0.1g can be used for less than one extraction socket, while 0.25g/0.5g can be used for maxillary sinus or for the wide bone defect area.

COWELL® BMP Plus (Two vials)

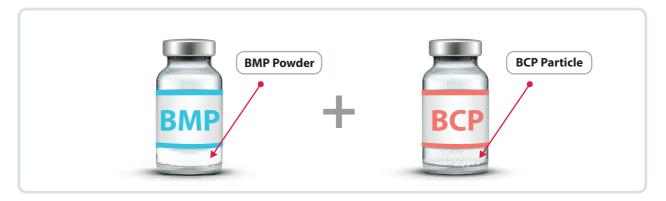

• Dose and particle size of the COWELL® BMP Plus.

BMP 0.1mg

Product Code	BMP Dose	Particle Dose	Particle Size
EBB0125	0.1mg	0.25g	0.41~1.0mm
EBB0105	0.1mg	0.5g	0.41~1.0mm
EBB1110	0.1mg	1g	0.41~1.0mm
EBB1220	0.1mg	2g	0.41~1.0mm

BMP 0.5mg

Product Code	BMP Dose	Particle Dose	Particle Size
EBB0525	0.5mg	0.25g	0.41~1.0mm
EBB0505	0.5mg	0.5g	0.41~1.0mm
EBB1150	0.5mg	1g	0.41~1.0mm
EBB1250	0.5mg	2g	0.41~1.0mm

BMP 2mg


Product Code	BMP Dose	Particle Dose	Particle Size
EBB2025	2mg	0.25g	0.41~1.0mm
EBB2050	2mg	0.5g	0.41~1.0mm
EBB2011	2mg	1g	0.41~1.0mm
EBB2012	2mg	2g	0.41~1.0mm

BMP 0.25mg


Product Code	BMP Dose	Particle Dose	Particle Size
EBB2525	0.25mg	0.25g	0.41~1.0mm
EBB2505	0.25mg	0.5g	0.41~1.0mm
EBB1125	0.25mg	1g	0.41~1.0mm
EBB1225	0.25mg	2g	0.41~1.0mm

BMP 1mg

Product Code	BMP Dose	Particle Dose	Particle Size
EBB1025	1mg	0.25g	0.41~1.0mm
EBB1050	1mg	0.5g	0.41~1.0mm
EBB1011	1mg	1g	0.41~1.0mm
EBB1012	1mg	2g	0.41~1.0mm

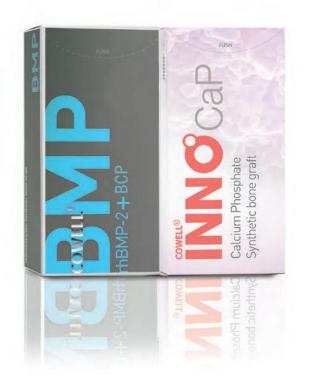

INNO GF Kit (Two vials + Saline Solution + Syringe)

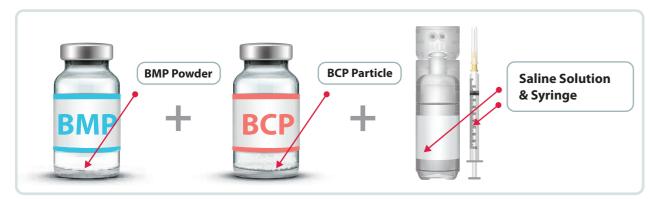

• Dose and particle size of the INNO GF Kit.

BMP 0.1mg

Product Code	BMP Dose	Particle Dose	Particle Size
IBB0125	0.1mg	0.25g	0.41~1.0mm
IBB0105	0.1mg	0.5g	0.41~1.0mm
IBB1110	0.1mg	1g	0.41~1.0mm
IBB1220	0.1mg	2g	0.41~1.0mm

Product Code	BMP Dose	Particle Dose	Particle Size
IBB0525	0.5mg	0.25g	0.41~1.0mm
IBB0505	0.5mg	0.5g	0.41~1.0mm
IBB1150	0.5mg	1g	0.41~1.0mm
IBB1250	0.5mg	2g	0.41~1.0mm

Product Code	BMP Dose	Particle Dose	Particle Size
IBB2025	2mg	0.25g	0.41~1.0mm
IBB2050	2mg	0.5g	0.41~1.0mm
IBB2011	2mg	1g	0.41~1.0mm
IBB2012	2mg	2g	0.41~1.0mm

BMP 0.25mg


Product Code	BMP Dose	Particle Dose	Particle Size
IBB2525	0.25mg	0.25g	0.41~1.0mm
IBB2505	0.25mg	0.5g	0.41~1.0mm
IBB1125	0.25mg	1g	0.41~1.0mm
IBB1225	0.25mg	2g	0.41~1.0mm

Product Code	BMP Dose	Particle Dose	Particle Size
IBB1025	1mg	0.25g	0.41~1.0mm
IBB1050	1mg	0.5g	0.41~1.0mm
IBB1011	1mg	1g	0.41~1.0mm
IBB1012	1mg	2g	0.41~1.0mm




6. User Guide COWELL® BMP


A. Method I

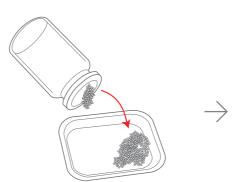


a. Transfer BCP graft material (Vial ${f I}$).

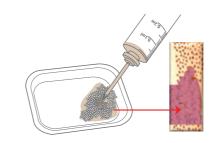

c. Mix BMP solution with BCP or plus autogenic / allograft and apply into the recipient site.

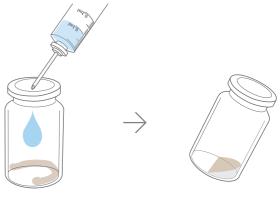


b. Inject distilled water into vial $\boldsymbol{I}\boldsymbol{I}$ with lyophilized rhBMP-2 power in it and mix with the powder.

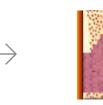


d. Cover the defect area using a barrier membrane or suture natural soft tissue without membrane.

B. Method II

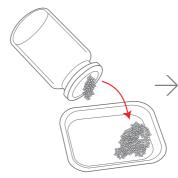


a. Transfer BCP graft material (Vial I) into a container.



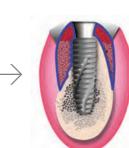

b. Apply BCP into the recipient site and cover the defect area using a barrier membrane or suture natural soft tissue without membrane.

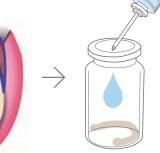


c. Inject distilled water into lyophilized rhBMP-2 powder (vial II).



d. Mix rhBMP-2 with distilled water (saline solution) and wait for 10 to 15 seconds before using.



e. Aspirate the mixture using a syringe.

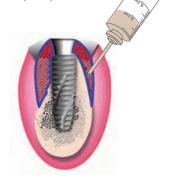


f. Inject BMP solution through soft tissue until needle of syringe reaches bone.


272 COWELL® BMP

C. Method III



a. Inject distilled water into vial II with lyophilized

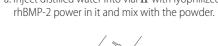

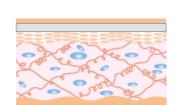

b. Aspirate the mixture using a syringe.

c. Hydrate BMP-2 solution into membrane.


d. Apply BMP-2 solution socked membrane to damaged site.

Dose of distilled water to make the mixture (BMP-2 Solution)

BMP Dose	Distilled Water Dose	BMP Dose	Distilled Water Dose
0.1mg	0.1ml	2mg	1.6ml
0.25mg	0.2ml	5mg	4ml
0.5mg	0.4ml	10mg	8ml
1mg	0.8ml	20mg	16ml

7. Study Result on COWELL® BMP


In Vivo Study

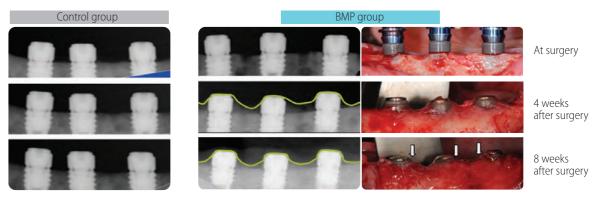

Jung-Bo Huh, et al., Alveolar ridge augmentation using anodized implants coated with Escherichia coli-derived recombinant human bone morphogenetic protein 2 (Beagle dog).

- Oral Surg Oral Med Oral Pathol Oral Radiol Endod 2011.

Histologic Findings: Tissue specimen collected approximately four months after the maxillary sinus grafting (human)

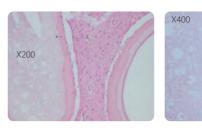

- New bone was formed around the graft material.

 - No inflammatory reaction was observed in connective tissue.
 - Proliferation of collagen fiber was observed.
 - Proliferation of fibrocyte was observed.
 - Osteoblast was observed on the surface of newly formed bone.

8. Clinical Data of COWELL® BMP

- Vertical height of surrounding bone was compared three months after grafting in extraction socket.
- The study was conducted at Seoul National University Bundang Hospital, Yonsei University Dental Hospital, and Korea University Guro Hospital.

Group		Average	SD	95%CI	†P value
	Control	-1.087	1.413	(-1.565, -0.609)	0.0006**
Height	Experiment	-0.059	0.960	(-0.384, 0.266)	
140 ld . ==0/ EG	Control	1.405	1.753	(0.812, 1.998)	0.346
Width at 75% ESL	Experiment	1.863	2.310	(1.081, 2.644)	
	Control	0.542	1.157	(0.15, 0.934)	0.016*
Width at 50% ESL	Experiment	1.239	1.249	(0.816, 1.662)	0.016
	Control	0.006	1.149	(-0.383, 0.395)	<0.0001**
Width at 25% ESL	Experiment	1.279	1.387	(0.81, 1.749)	<0.0001**

ESL: Extraction Socket Level

*:P<.05, **:P<.01, †: Student *t*-test

Jung-Bo Huh, et al., Multicenter, randomized clinical trial on the efficacy and safety of Escherichia-coli-derived rhBMP-2 with β -Tricalcium phosphate and hydroxyapatite in human extraction sockets.

- J Adv Prosthodont 2011;4-134.

274 COWELL® BMP

INNO-CaP Calcium Phosphate , Synthetic Bone Graft


Osteoconductive resorbable synthetic bone graft material


- INNO-CaP is a osteoconductive synthetic resorbable bone graft material consisting of Calcium Phosphate.
- INNO-CaP is completely resorpted and progressively replaced by normal-structured bone in the healing period.

Excellent Biocompatibility and Conductivity

• The characteristic biocompatibility and conductivity of the INNO-CaP represent the most safety.

Cell culture SEM images (14 days)

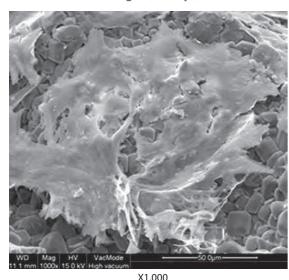

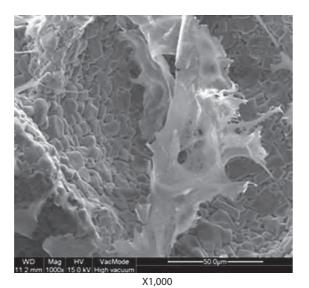






A porosity for new bone ingrowth

• The porosity promotes ingrowth of osteoblast, osteoclast and growth factors.

Particle surface SEM image

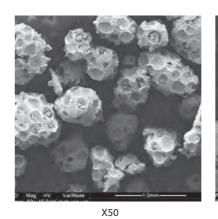

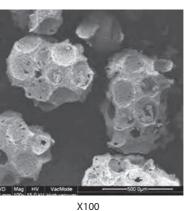

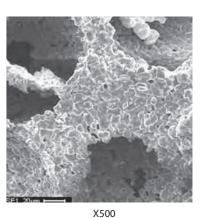

Indications

Sinus graft surgery

- For sinus graft, INNO-CaP is used alone or in combination with the other graft materials.
- Various healing period according to residual bone height.

residual bone height	less than 1mm	2~4mm	more than 4 mm
implant placement	post operation 9~12 months	post operation 6 months	simultaneous placement

GBR (Guided Bone Regeneration)

- Minimize the amount of autogenous bone.
- Sub-graft materials.
- Vertical and lateral augmentation.
- Using with the COWELL® BMP is highly recommended.

Dose and Particle Size

Product Code	Particle Size	Particle Dose
IG1025	0.4~1.0mm	0.25g
IG1050		0.5g
IG1001		1g
IG1002		2g
IG1425		0.25g
IG1450	1.0.14mm	0.5g
IG1401	1.0~1.4mm	1g
IG1402		2g

INNO OSS & INNO OSS Allo

Allograft

FDBA, Cortical 50% Cancellous 50%


Product Features

- This product is made up of human tissue from trusted donors whose gender, age, and medical history were checked to ensure that their tissue could be used safely.
- It is an ideal combination of 50% cortical powder and 50% cancellous powder for bone induction.
- The 50% cortical powder maintains the space of the transplanted area during the new bone formation due to the delayed absorption rate. [OsteoConduction]
- 50% cancellous powder is rich in minerals and collagen that promote cell adhesion, bone remodeling, and vascular re-formation.

INNO OSS Allo

- To prevent cross-infection by a different donor, the process is done by a single donor.
- Under the higher-level pharmacological standards (medical criteria) of the American Association of Tissue Banks (AATB), we sampled, processed and distributed the allograft tissue.
- We recommend use of this product with the COWELL® BMP.
- The difference between INNO OSS and INNO OSS Allo is classification of the product only. INNO OSS is classified as a HUMAN TISSUE and INNO OSS Allo is a MEDICAL DEVICE.

SEM Image

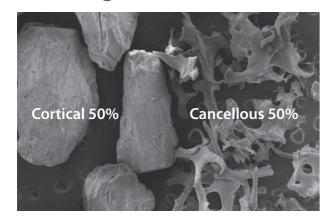

Dose and Particle Size


Inno-Oss

Type	Product Code	Particle Size	Particle Dose
INNO OSS	OSS3	0.3~0.8mm	0.3cc
11110 033	OSS6	0.3~0.8mm	0.6cc
INNO OSS Allo	OSS3P	0.3~0.8mm	0.3cc
INNO O33 Allo	OSS6P	0.3~0.8mm	0.6cc


Method of Use



Remove the syringe's rubber cap.

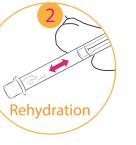

Hydrate it in saline solution.

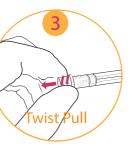

Turn and pull out the syringe cap to remove it.





Graft it in the desired area.


DiaBONE Bovine Bone Substitute

A Bone 100% fused to Natural Human Bone

- Fast blood penetration.
- · Great hydrophilicity.
- 3-Dimensional structure.
- Easy to handle.
- Maximizes bone fusion.
- Mutually connected porous structure.
- Optimal cell attachment and blood absorption.
- Stimulates activity of osteoclast and osteoblast.

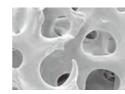

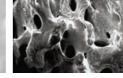

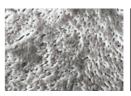





DIABONE

Human bone structure


Dia bone structure (x50) Dia bone structure (x1,500)


Hydrophilicity 1

Safe & Trustable Material

- Made of 100% bovine bone.
- Cleansing over 30 times to perfectly remove organic substances.
- Firmed bone formation as highly dense.
- 100% pure HA & 99.73% of bone crystallization.



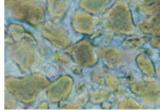

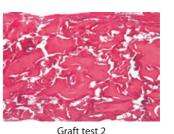

Graft test 1

(New bone formation clearly observed around grafted bone site)

Volume and Particle Size

Product Code	Particle Size	Volume
G2015	0.25~1.0mm	0.15g
G2025	0.25~1.0mm	0.25g
G2050	0.25~1.0mm	0.5g
G2100	0.25~1.0mm	1g
G2200	0.25~1.0mm	2g

Product Code	Particle Size	Volume
G5015	1.0~2.0mm	0.15g
G5025	1.0~2.0mm	0.25g
G5050	1.0~2.0mm	0.5g
G5100	1.0~2.0mm	1g
G5200	1.0~2.0mm	2g

278 INNO OSS & INNO OSS Allo DiaBONE 279

MEGA DERMTM PLUS Acellular Dermal Matrix


Product Features


- This product can carry out the functional blocks of the membrane (soft tissue penetration protection) due to its long absorption period, and has excellent manipulability.
- Unlike other imported complete products, this product is produced under the stringent standards of the FDA in Korea.
- The world's first E-Beam sterilization can induce safe and prompt engraftment.
- E-Beam is safe and can be effectively sterilized without destroying the collagen tissue structure.
- This product is the first in the world with the basement membrane layer removed (patent pending) to maximize the transplant engraftment rate.
- This shows the high engraftment rate after the transplant by maximizing the influx of fibroblasts and/or the neovascularization. (Patent Application No. 10-2012-0026616)


Application


- Mucogingival defect.
- Soft tissue formation around the implant area.
- Wide perforation in the Schneiderian membrane.

SEM Images (These have kept the collagen structure after the E-Beam sterilization.)

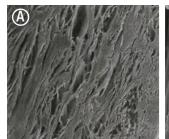

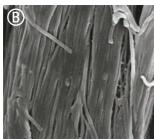

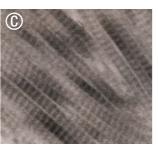

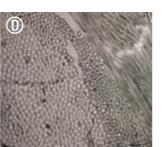

A. SEM (x200)

B. SEM (x20,000)


C. TEM (Transverse section)


D. TEM (Cross section)


Specifications

Product Code	Size	Thickness
D1520P	15x20mm	0.5~0.7mm
D1525P	15x25mm	0.5~0.7mm

MEGA DERM PLUS three-dimensional structure of the dermis

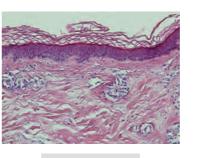

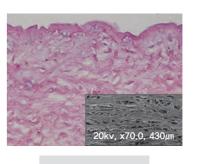

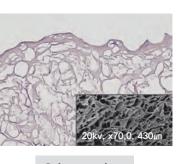



Normal skin

MEGA DERM PLUS

Other product

The world's first 'E-Beam' sterilization that does not destroy the collagen structure

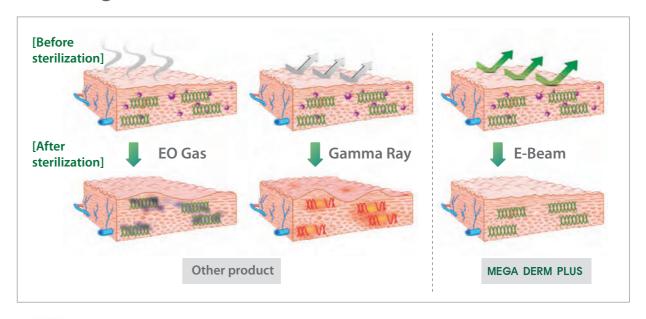

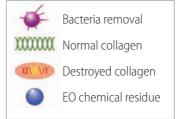

$\textbf{Diaderm}^{\texttt{®}} \ M \quad {}_{\textit{Biodegradable Atelocollagen Membrane}}$

GTR(Guided Tissue Regeneration) GBR(Guided Bone Regeneration) membrane

- Diaderm® is a dental membrane used for GTR (Guided Tissue Regeneration) and GBR (Guided Bone Regeneration) operation.
- Diaderm® helps restoration of alveolar bone and protect operation site from infiltration of an epithelial cell and exterior circumstances.
- Diaderm® made from high purity atelocollagen has high biocompatibility, mechanical strength, resistance to enzymatic degradation and flexibility.


Product Features

- High biocompatibility.
- Induces restoration of tissue.
- Closes wound site completely.
- Sustains space for bone reproduction.
- Easy to handle and operate.



Resistance against enzymatic degradation

• Diaderm® has higher resistance to enzymatic-degradation compared to products from other manufacturers.

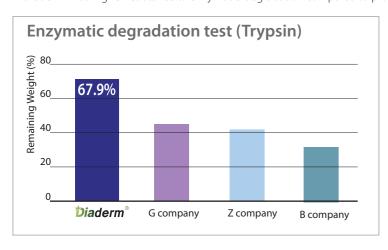

Storage and Shelf Life

- Store at 1-30°C
- Shelf Life: 3 years from the date of manufacture

Specifications

Product Code	Size
AS-007020	15X30mm

InnoGenic™ Non-resorbable Membrane

InnoGenic™ Wifi-Mesh and InnoGenic™ PTFE-Mesh

• The InnoGenic™ Wifi-Mesh, PTFE-Mesh and Ti-Mesh are non-resorbable barrier membrane to be applied over intraoral defects, especially, tooth extraction and bone augmented sites. The InnoGenic™ Wifi-Mesh and PTFE-Mesh are made of proprietary 100% PTFE, the polytetrafluoroethylene (teflon) sheet which is a biologically inactive and tissue compatible material and the InnoGenic™ Wifi-Mesh is reinforced with titanium frames (Titanium Gr II, ASTM F 67) embedded between two layers of PTFE sheets.

InnoGenic™ Wifi-Mesh


> Packing unit: 1ea






Product Code	Size	Thickness
BTP1424AA	14X24	0.25
BTP1424AB	14X24	0.25
BTP1525BB	15X25	0.25
BTP1725CA	17X25	0.25
BTP2030AB	20X30	0.25
BTP2530AB	25X30	0.25
BTP3040AB	30X40	0.25



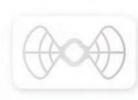



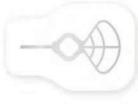

BTP1424AA

BTP1424AB

BTP1525BB

BTP1725CA







BTP2530AB

BTP3040AB

Clinical Case using the Wifi-Mesh







• Bone graft using INNO-OSS™ Allo



Shielding soft tissue penetration using Wifi-Mesh

3 months later





InnoGenic™ PTFE-Mesh

> Packing unit : 5ea





Product Code	Size	Thickness
TS 17251	17 x 25	0.1
TS24301	24 x 30	0.1
TS17252	17 x 25	0.2
TS24302	24 x 30	0.2

Features

- Non-resorbable: Made of 100% non-resorbable material for users to modulate healing period.
- Non-porous (0.0 μm) + Open Membrane Sheet Technique : Prevention of infection or other possible defects caused from passage or integration of bacteria through porosity of plaster and it even allows to apply Open Membrane Sheet Technique.
- Prevention of Displacement: Not only being sutured along with gingival but also using InnoGenic™ GBR Kit inserted and fixed into the hole of titanium frame allows displacement of the products.
- Close to Transparency: Observation of the healing of the underlying tissue through almost transparent PTFE surface allows more predictable result and helps determine removal time easier.
- Easy to be Customized: Easy to modify the shape according to shape and dimension of the defect.
- Easy to be Removed: Put a hook in the hole of the titanium frame of the InnoGenic™ Wifi-Mesh and in any center part of the InnoGenic™ PTFE-Mesh and remove.


Comparison to other similar products sold in market

Classification	Product A	Product B	InnoGenic™Wifi-Mesh & PTFE-Mesh
SEM Photograph	CA=157°	70	
Ultrastructure	Fiber	Filter	Sheet
Bacterial infection at exposure	Bacterial toxin penetration between filters at 50 µm intervals	Bacterial toxin penetration between filters at 2 µm intervals	No Bacterial toxin penetration thanks to non porous structure
Action on Exposure	Instant Removal	Removal on week 3 to 4	Safe for more than 6 weeks
Shielding Function against Fiber Cell	High	High	Extremely High
Shape-keeping Capability against External Force	Large Deformation	Shrinkable Deformation	No Deformation

InnoGenic™ Ti-Mesh

• The InnoGenic™ Ti-Mesh is made of stamping titanium sheet, also Titanium Gr II, ASTMF 67, which is 100% commercially pure titanium. The InnoGenic™ Ti-Mesh is non-resorbable surgical mesh to be applied over intraoral defects, especially, tooth extraction and bone augmented sites.



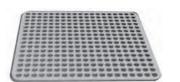

Product Code	Size	Thickness
TMP210	25 x 34	0.07
TMP211	25 x 34	0.1

Features

- Easy to be Customized: Easy to modify the shape according to shape and dimension of the defect.
- Prevention of Displacement: Prevents displacement of the InnoGenic™ Ti-Mesh using the InnoGenic™ GBR Kit inserted and fixed into the 1mm hole of the Ti-Mesh Frame.
- No Memory: The problem of Majority of Titanium Meshes in the market is resilience of the products after certain time. Due to this problem, patients go through serious pain. The InnoGenic™ Ti-Mesh is, however, made after many times of stamping process, the InnoGenic™ Ti-Mesh does not come back to the original shape after shape is formed.

MEMO	MEMO

Ver.28 Help your daily practice superior















Cowellmedi Co.,Ltd

Floor 6, Blue Fin Tower, 42, Seochojungang-ro, Seocho-gu, Seoul, Korea Tel. +82-2-3453-5085 Fax. +82-2-3453-5086 E-mail. cib@cowellmedi.com

Cowellmedi USA INC

218 Trianon LN Villanova PA 19085-1442 USA Tel. **1-623-939-1344** Fax. 1-623-939-1472

Cowell R&D Institute

48, Hakgam-daero 221beon-gil, Sasang-gu, Busan, 46986, Korea Tel. +82-51-314-2028 Fax. +82-51-314-2026





CWM-Ver.28